

- Assembly Instructions
- Schematic
- Parts List





To Register Your Product, visit us at [www.adctoday.com](http://www.adctoday.com) and follow the links


# DIAGNOSTIX™ 752M

## HOW TO ASSEMBLE



1. With base on its side connect upright outer pole to base using large (6mm) allen bolt. Tighten with large (6mm) allen key provided. Tighten hardware as securely as possible. (Note: To prevent loosening, hardware must be tightened securely and periodically checked). (Figure 1)
2. Slide stainless steel inner pole inside outer sleeve with open end up. (Figure 2)
3. Loosen (3mm) allen bolt on collar of cuff storage basket with small (3mm) allen key provided. Place cuff storage basket on top of stainless steel inner pole making sure the pole goes completely inside collar and tighten allen bolt with key. (Figure 3).
4. Attach manometer bracket to basket using (6mm) allen bolt and washers as shown. Tighten securely with large (6mm) allen key. (Figure 4).
5. Adjust height of manometer by loosening knob and raising or lowering inner pole until gauge is at desired height. Tighten knob by hand.
6. Connect coiled tubing to gauge inlet. (Note: The coiled tubing may be difficult to connect, moisten with soapy water if necessary). Refer to booklet entitled DIAGNOSTIX™ Aneroids Pocket, Palm, Clock Type booklet (9350N-00) for instructions on operating the unit.

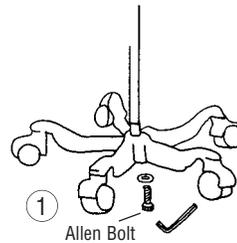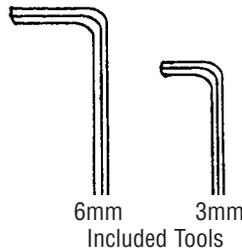


Connect outer pole (tube) to base assembly. Note: Tighten allen bolt securely by applying as much force to allen key as possible.

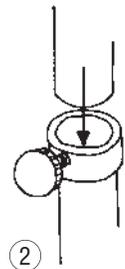

Slide inner pole inside outer pole.

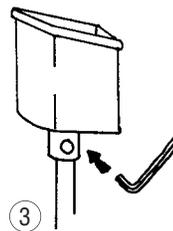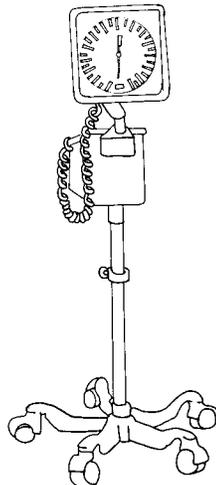


Insert pole into bottom of cuff storage basket and tighten allen bolt on side of the basket with small allen key.

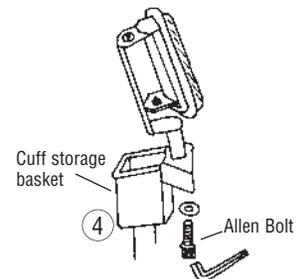

Attach manometer bracket to cuff storage basket.

# SCHEMATIC PARTS LIST

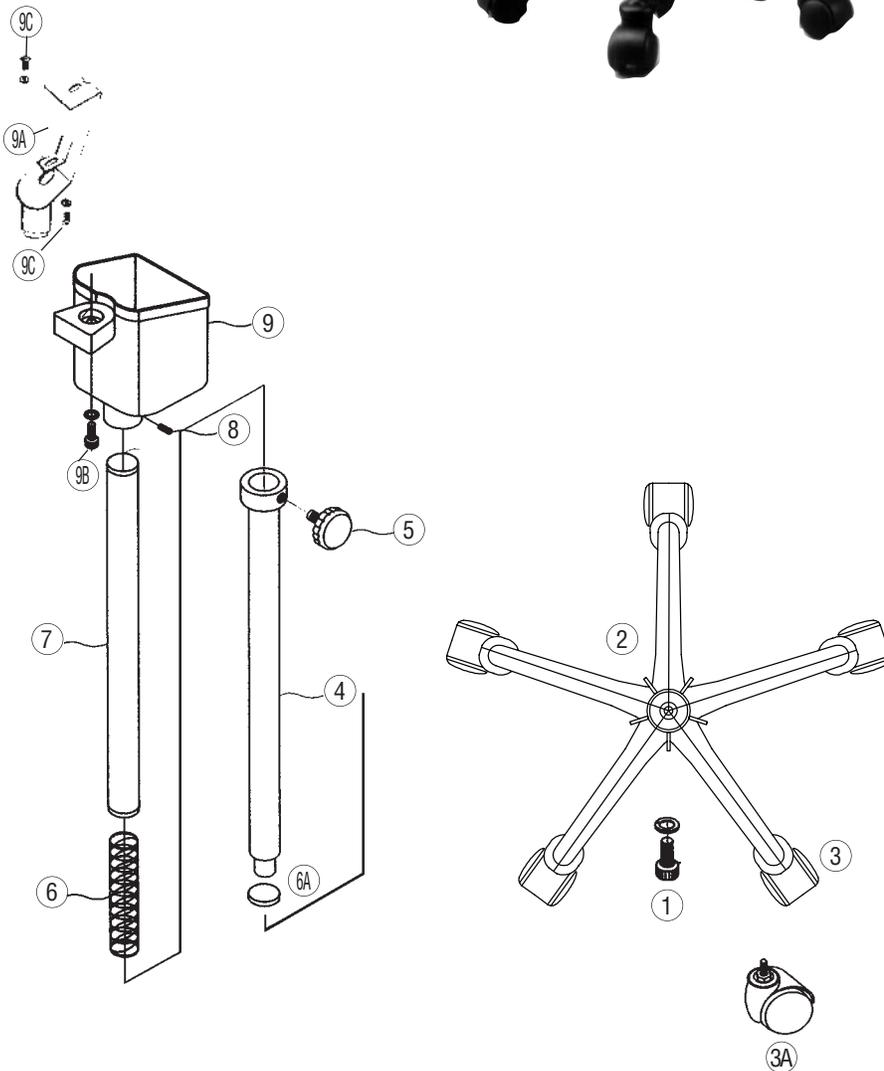

## ROD & BASE ASSEMBLY FOR 752M

Ref #	Item #	Description
1	972-101	Center Pole Screw and Washer (1)
2	972-080	Base
3	972-106	Casters (5)
3A	972-106-1	Single Caster
4	972-107	Outer Pole
5	972-108	Knob
6	972-109	Spring
6A	972-107A	Gasket
7	972-110	Inner Pole
8	972-111	Tightening Device
9	972-112	Basket
9A	752-113	Bracket
9B	752-114	Manometer Mounting Hardware
9C	752-115	Manometer Bracket Hardware