
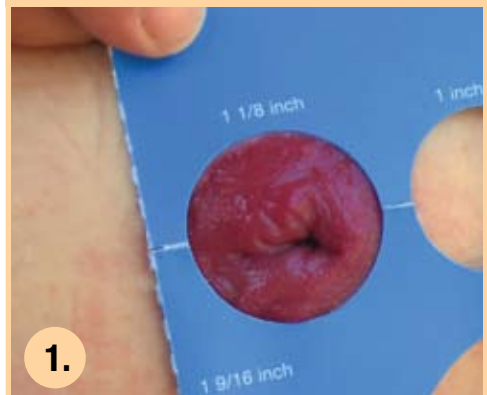
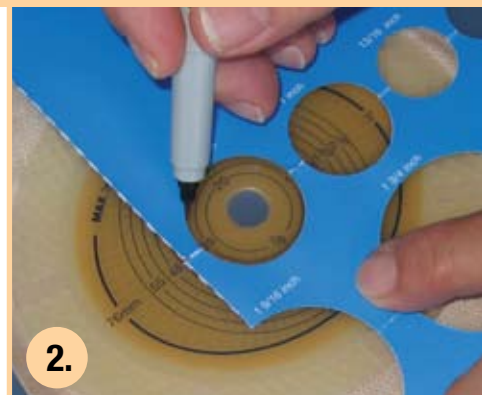
Preparation


1.
Measure the stoma.

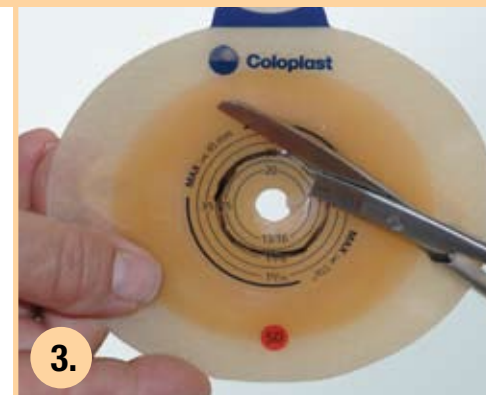2.
Trace measurement
onto back of the barrier.

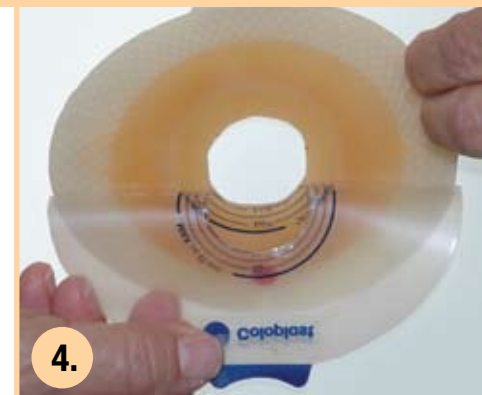
3.
Cut opening in the barrier.

4.
Remove protective backing by pulling the blue release tab away from the barrier.

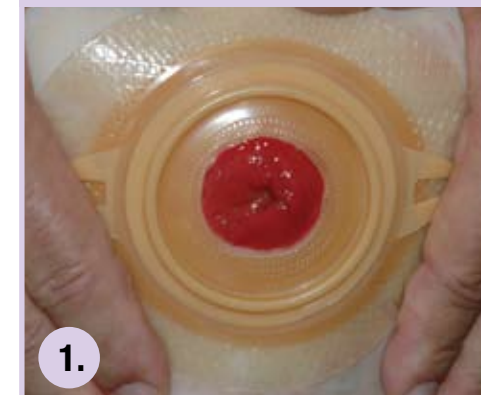Application

1.
Center barrier around stoma. Secure to skin by applying gentle pressure.

2.
Place pouch on the barrier starting at the bottom of the click lock security ring. Gently press pouch into place while moving toward the top of the barrier with both hands.

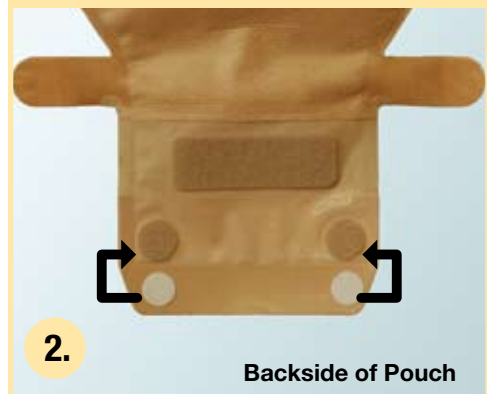
3.
Secure pouch by pinching the two ends of the click lock security ring tab together with your fingers. You will hear a click.

Emptying the pouch

1.
Open the outlet by lifting both tabs off the Velcro® plate. Unfold outlet.

Emptying the pouch

2.
Backside of Pouch
Fold back lower plate to avoid soiling when emptying. Attach Velcro® dots to hold plate in place.

3.
Empty pouch by pinching outer edges of the outlet open.

4.
Clean inside the outlet and wipe the plates with a fresh tissue.

Closing the pouch

1.
Release the lower plate from the Velcro® dots. Using the guide, fold the lower plate over the upper plate.

2.
Fold upwards. Seal the outlet by attaching the Velcro® hook tabs onto the Velcro® plate.

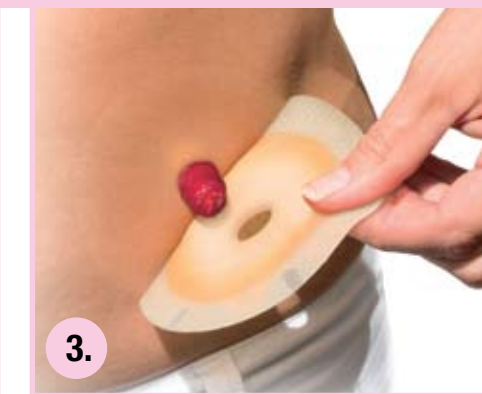
Removal

1.
Open the security ring by pressing on the front of the tab with your fingertip.


2.
Remove pouch from barrier. Pull from the top of the pouch with an up and over motion.

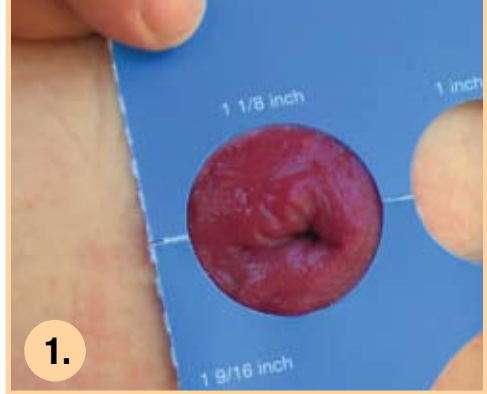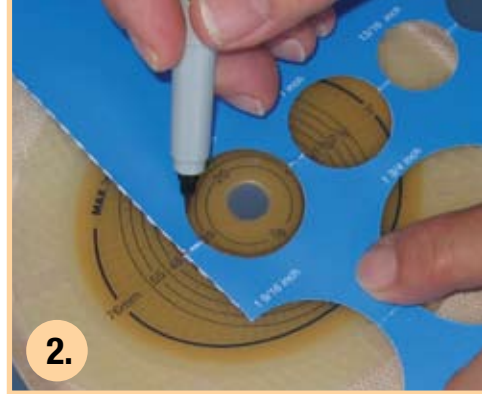
3.
Use the white tab to gently pull the barrier away from the skin using the "push-pull" technique.

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