

Assura : 1-piece colostomy pouches

Hole size Ø "							
Custom-cut 1/4-1 1/2" 1/4-2 1/4"	2120		Pediatric Closed Opaque		Mini Closed Opaque		Mini Closed Transparent
			Small Closed Opaque		Small Closed Transparent		Mini Cap Opaque
Pre-cut 1" 1 1/4" 1 1/2" 1 3/4" 1 7/8" 2"	2421						
Pre-cut 1" 1 1/4" 1 1/2" 1 3/4" 1 7/8" 2"	2431						
Pre-cut 1" 1 1/4" 1 1/2" 1 3/4" 1 7/8" 2"	2461						
Pre-cut 1" 1 1/4" 1 1/2" 1 3/4" 1 7/8" 2"	2471						
Pre-cut 1" 1 1/4" 1 1/2" 1 3/4" 1 7/8" 2"	2501						


Coloplast
Coloplast Corp.
1955 West Oak Circle
Marietta
Georgia 30062


Assura : 1-piece pouches




Coloplast

18814523. IFU inhouse. Assura is a registered trade mark of Coloplast A/S.
© 11.98. All rights reserved Coloplast A/S, DK-3050 Humlebæk, Denmark. Printed in Denmark by From & Co. A/S.



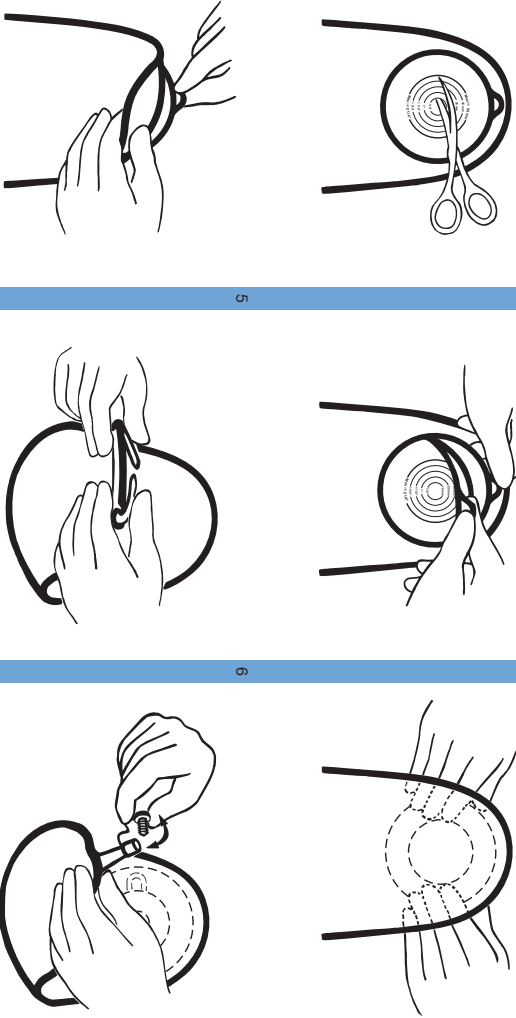

1. Cutting (if appropriate)
Place the template on the protective paper and make a sketch of the shape. Cut the hole in the adhesive to fit exactly.

2. Removal of protective paper
Remove the protective paper.

3. Application
Place the adhesive on the skin, ensuring a snug fit around the stoma. Press firmly against the skin starting at the bottom and smoothing upwards with the fingers.

4. Removal
The pouch is removed from the skin by peeling the pouch downwards slowly and carefully.

5. Drainable pouches
Before applying the pouch, close the open end by using the enclosed clamp. Remove the clamp from the sheet and place the adhesive side

down at the opening. Fold up 4-5 times, working toward the skin and then fold back the ends of the clamp. If the clamp is not placed correctly the first time, remove it and try again.

6. Urostomy pouches
Close the outlet by folding the tubing into the holder.

Hold the plug/end of outlet upwards and squeeze the outlet end of outlet, pull out the plug and relax your hold on outlet and drain the pouch. After complete drainage ensure that the plug is firmly closed. (Plug inserted)

Single use only
Coloplast accepts no liability for any injury or other loss that may arise in the use of this product in a manner contrary to Coloplast's current recommendations.

Hole size Ø "																								
Custom-cut 1/4-1 1/2" 1/4-2 1/4"	2115		Pediatric Drainable Transparent			Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent						
Extended wear 1/4-2 1/4"	13520		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
Pre-cut 1" 1 1/4" 1 1/2" 1 3/4" 1 7/8" 2"	12624		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12625		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12626		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12627		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12634		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12635		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12636		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12637		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12638		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12639		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12674		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12675		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12676		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12677		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12664		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12665		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12666		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	12667		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	13501		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	13502		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	13504		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	2540		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	2550		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						
	2570		Small Drainable Opaque			Small Drainable Transparent			Standard Drainable Opaque			Standard Drainable Transparent			Pediatric URO 150 ml Transparent			Small URO 150 ml Opaque						