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




18814644, IFU inhouse. Assura is a registered trade mark of Coloplast A/S. © 09.01.
All rights reserved Coloplast A/S, DK-3050 Humlebæk, Denmark. Printed in Denmark by L+J a/s.

Assura : **Convex**

Coloplast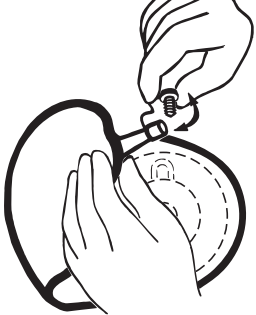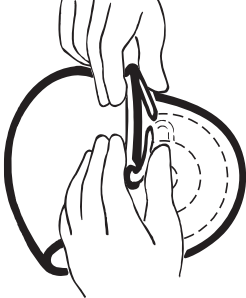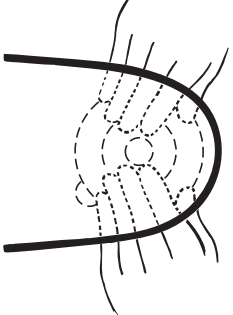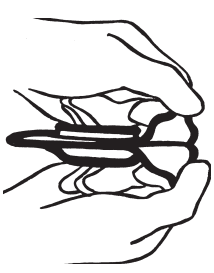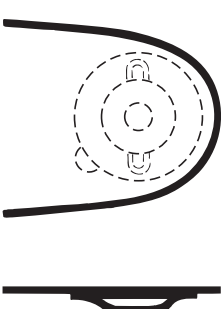

Convex products are for people with flush or retracted stomas, or stomas situated in skin folds. To fit properly, please be sure to select a product with the correct hole size.

1. Before application

Measure the diameter of the stoma using a stoma guide. Select a convex product with a hole size corresponding to, or slightly larger than, the diameter of the stoma. Gently clean and dry the skin around the stoma. Cut the adhesive from the starter hole so it fits the size of the stoma. This is easily done with a pair of curved scissors.

2. Removal of the protective film from the adhesive

Hold the small tabs of the adhesive and the protective film respectively with both hands. Be sure to support each side of the plate with your hands. Remove the protective film from the adhesive in one quick single movement.

3. Application of 1-piece pouch

Position the stoma opening of the adhesive over the stoma. Press firmly against the skin starting at the bottom and smoothing upwards with the fingers.

Continue to press the entire adhesive section until it adheres securely to the skin. The appliance comes with a belt attachment on either side. A Coloplast Universal Belt (code #0421) may be worn for extra support.

4. Removal of 1-piece pouch

To remove the ostomy pouch, you must remove the adhesive carefully from upper to lower level of your skin. With one hand keep your skin tight.

5. Open pouches

Make sure your pouch is closed before applying it. Apply the clip on the open side of the ostomy pouch. You can roll this flexible clip 3 to 4 times to the upper side, after it is placed on the pouch. Bend the clip at the ends so

they are placed on top of the pouch - not towards the body.

6. Drainage of urostomy pouches

Hold the plug/end of outlet upwards and squeeze the outer end of outlet. Pull out the plug, and relax your hold on outlet to drain the pouch. After complete drainage, ensure the plug is firmly closed. (Plug inserted).

Single use only.

Coloplast accepts no liability for any injury or other loss that may arise in the use of this product in a manner contrary to Coloplast's current recommendations.

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