
Gather supplies before starting the steps below.

SenSura™ 1-piece Pouch


Pictorial Instructions for Use

Preparation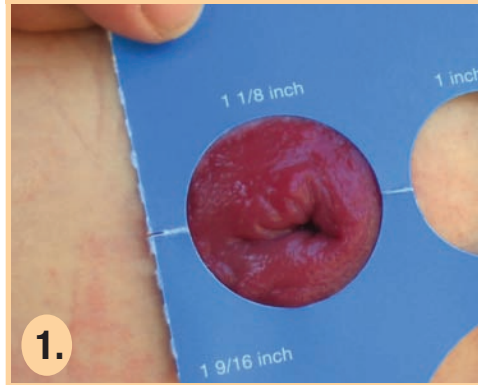


1. Measure the stoma.

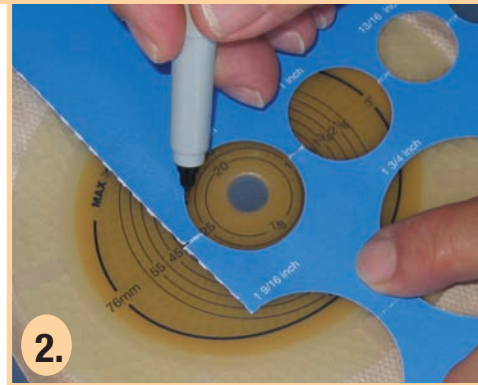2. Trace measurement
onto back of the barrier.

3. Cut opening in the barrier.

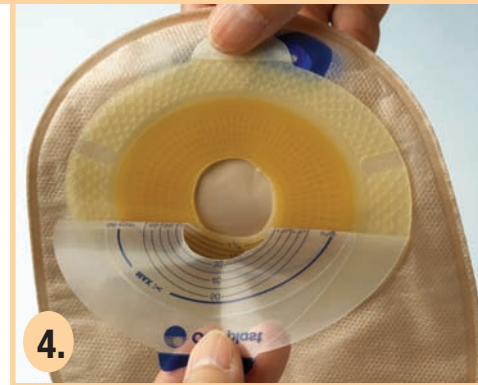4. Remove protective backing by pulling the blue release tab away from the barrier.

Application

1. Apply pouch
by centering barrier around stoma.

2. Secure to skin
by applying gentle pressure.

Emptying the pouch

1. Open the outlet
by lifting both tabs off the Velcro® plate. Unfold outlet.

2. Fold back lower plate to
avoid soiling when emptying.
Attach Velcro® dots to hold
plate in place.

Emptying the pouch cont.


3. Empty pouch
by pinching outer edges of
the outlet open.

4. Clean the outlet.

Closing the pouch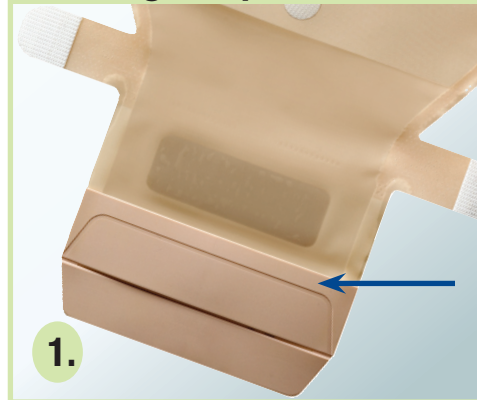

1. Release the Velcro® dots.
Using the guide, fold the
lower plate over the upper
plate.

2. Fold upwards.
Seal the outlet by attaching
the Velcro® hook tabs onto
the Velcro® plate.


3. Hide-Away:
Tuck sealed outlet up and
under the protective pocket.

Removal

1. To remove,
use the white tab on the
barrier to gently pull the
pouch away from the skin.

For product support call the Consumer Specialist Group at 1-888-726-7872.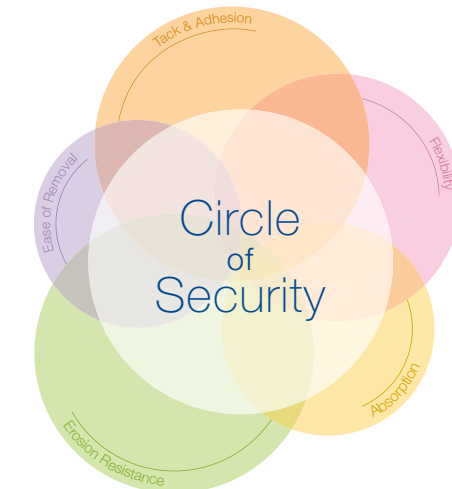

Coloplast Corp.
Minneapolis, MN 55411
Phone: 888-726-7872
usmedweb@coloplast.com
www.us.coloplast.com
www.coloplast.com

M1331N - 05/07

Coloplast® is a registered trademark of Coloplast A/S or related companies.
SenSura™ is a trademark of Coloplast A/S or related companies.
© 2007 Coloplast Corp. All rights reserved.
VELCRO® is a registered trademark of Velcro Industries B.V.

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