

7 Simple Steps to Independence.

Car/Boat/RV Usage

After you are familiar with operating your Inogen One G2 at home, you are ready to take your Inogen One G2 on the road.

As always, be a careful driver. DO NOT RUSH OR HURRY to set up your Inogen One G2 in the car. Take your time and follow the steps outlined to the right. The Inogen One G2 can operate either by battery or the Universal Power Supply while in your car, boat, or RV. If using the Universal Power Supply on DC power, the battery will charge.

CAUTION

Make sure your Inogen One G2 is secure before driving.



Plug the Universal Power Supply into the cigarette lighter or auxillary DC power supply.



Plug the Universal Power Supply power output cord into your Inogen One G2 and position your Inogen One G2 securely inside the car. Operate your Inogen One G2 as normal.

INOGEN ONE® G2



Getting Started

inogen®

326 Bollay Drive,
Goleta, CA 93117
1-877-4-INOGEN (Toll Free)
1-877-446-6436 (Toll Free)
1-805-562-0515 (International)
info@inogen.net
www.inogenone.com
www.OxygeNation.com

Your Equipment Provider:

Your Physician:

inogen®

INOGEN ONE® G2

Now that you have unpacked each box, you have everything you need to start using your Inogen One G2 (IO-200). The set-up of your Inogen One G2 is quick and simple. However, before using the Inogen One G2, please read the entire User Manual.



1 Place your Inogen One G2 in a well ventilated location. Ensure the particle filter is in place.



2 Insert the Inogen One G2 Battery by sliding battery into place and locking at bottom of concentrator.



3 Connect your Inogen One G2 Universal Power Supply AC cord to the Universal Power Supply. Connect the Universal Power Supply to your Inogen One G2.



4 Plug the Universal Power Supply AC cord into the nearest electrical outlet.



5 Connect the nasal cannula tubing into the metal nozzle on your Inogen One G2. Oxygen will only be delivered while you are breathing through the nasal cannula connected to the device.



6 Turn on the Inogen One G2 by pressing the  button.



7 Set the flow setting to the flow rate prescribed by your clinical professional. The flow setting can be changed by pressing the triangular flow setting buttons that are marked with "+" and "-" symbols. Position the nasal cannula on your face and breathe normally through your nose.

