



Under Buttocks Drape with Fluid Collection Pouch

(89414 depicted)




1. Orient Under Buttocks Drape with patient and insert right and left hands as marked.



2. Slide drape under patient's buttocks and open pouch with malleable wire.



3. Draping is complete.

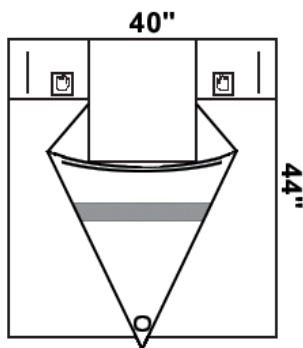

Features

- 40" x 44"
- Fluid Collection Pouch
- Screen and Suction Port
- Adhesive Pouch Closure

Draping Hints

- This drape has a pouch closure strip along the inside top edge of the pouch.

Stock

Description

Items Per Case

89415

Under Buttocks Drape with Fluid Collection Pouch, Sterile,
1 per package

40

79415 ♦

Under Buttocks Drape with Fluid Collection Pouch, Non-Sterile,
Bulk Pack

100

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.