

SURFACE DISINFECTION – CaviWipes1

IF GROSS DEBRIS IS PRESENT ON THE SURFACE:

CLEAN BEFORE DISINFECTING



1. Dispense CaviWipes1 towelette.



2. Clean surface with towelette to remove debris and bioburden.



3. Discard used towelette.

DISINFECT AFTER CLEANING



4. Dispense another CaviWipes1 towelette.



5. Disinfect precleaned surface with towelette. Leave surface wet for one (1) minute.



6. Discard used towelette.

IF NO VISIBLE GROSS DEBRIS IS PRESENT ON THE SURFACE:

ONE STEP CLEANING AND DISINFECTING



1. Dispense CaviWipes1 towelette.



2. Disinfect surface with towelette. Leave surface wet for one (1) minute.



3. Discard used towelette.

PRODUCT INFORMATION

PART NO. US / Canada	PRODUCT DESCRIPTION	PACKAGING
13-5120/11-5120	CaviWipes1, 120 count (6" x 9")	12 canisters/case
13-5150/11-5150	CaviWipes1 EXTRA LARGE, 65 count (9" x 12")	12 canisters/case
13-5155/11-5155	CaviWipes1 EXTRA LARGE Singles, 50 count (9" x 12")	6 boxes/case
13-5224	CaviWipes1 Flat Pack, 45 count (7" x 9")	20 packs per case
13-1175	CaviWipes Canister Wall Bracket	12 brackets/case

Note: Follow standard precautions for personal protection as appropriate.



CaviWipes1 Family
of surface disinfectants



For more information, visit [metrex.com/education](https://www.metrex.com/education), or call 800.841.1428.

Lit # 77-1075-1