

## User Guide 8200



**8200**

1. Remove the existing toilet seat and lid from the toilet frame by unscrewing the plastic seat screws located at the rear of the seat.
2. Place the toilet safety frame bracket between the toilet opening and the toilet tank with the rounded part of the bracket at the back of the toilet frame. The mounting holes on the bracket should line up with the mounting holes on the toilet frame.
3. Replace the toilet seat/lid and reinsert mounting bolts through the bracket and then through the frame.
4. Replace and retighten the mounting nuts to mounting screws.
5. Insert side frames into the bracket and adjust to the desired width. Adjust the brass push-buttons to the desired width. Make sure the push-buttons are fully engaged before moving to step 6.
6. Adjust the height of the legs by depressing the push-button and sliding the leg up or down to the desired height. Make sure both legs are fully engaged and adjusted to equal heights.
7. **RE-CHECK MOUNTING SCREWS TO MAKE SURE THEY ARE SECURE AND ALL PUSH BUTTONS ARE FULLY ENGAGED BEFORE USING.**

### **LIFETIME LIMITED WARRANTY**

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product.

NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent.

This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, handgrips, casters, and other parts subject to normal "wear and tear"

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.