



Welch Allyn Inc.

Medical Division

4341 State Street Road, Box 220

Skaneateles Falls, New York 13153-0220 USA

Tel: (315) 685-4445 FAX: (315) 685-4653

## *Decontamination* **Bulletin**

*This Bulletin references, and adds clarification to Product Instructions for:*

---

**Standard Laryngoscope Handles** (Lamp Type). .. 60200, 60300, 60305, 60400, 60703

---

**NOTE:** *Remove batteries before performing any of the following procedures.*

---

### CLEANING:

- Handles may be cleaned with a mild detergent/warm water solution.
- **DO NOT GET SOLUTION IN CONTACT AREA**

### DISINFECTION:

- Handles may be **wiped** with a 70% isopropyl alcohol.
- **DO NOT GET SOLUTION INTO CONTACT AREA**
- **DO NOT IMMERSER OR SOAK**

### STERILIZATION:

- May be gas sterilized with ETO concentration of 10-100%.
  - \* **DO NOT EXCEED** 4 hours exposure time and 132<sup>o</sup>F (55<sup>o</sup>C). Follow with 12-16 hours aeration at 120<sup>o</sup>F(49<sup>o</sup>C).
- Only handles marked **AUTOCLAVE** may be steam autoclaved using any standard autoclave cycle where the temperature does not exceed 280<sup>o</sup>F (138<sup>o</sup>C).
- **DO NOT FLASH AUTOCLAVE**