


6700-6704-6705 Premier Care Cliner Parts List

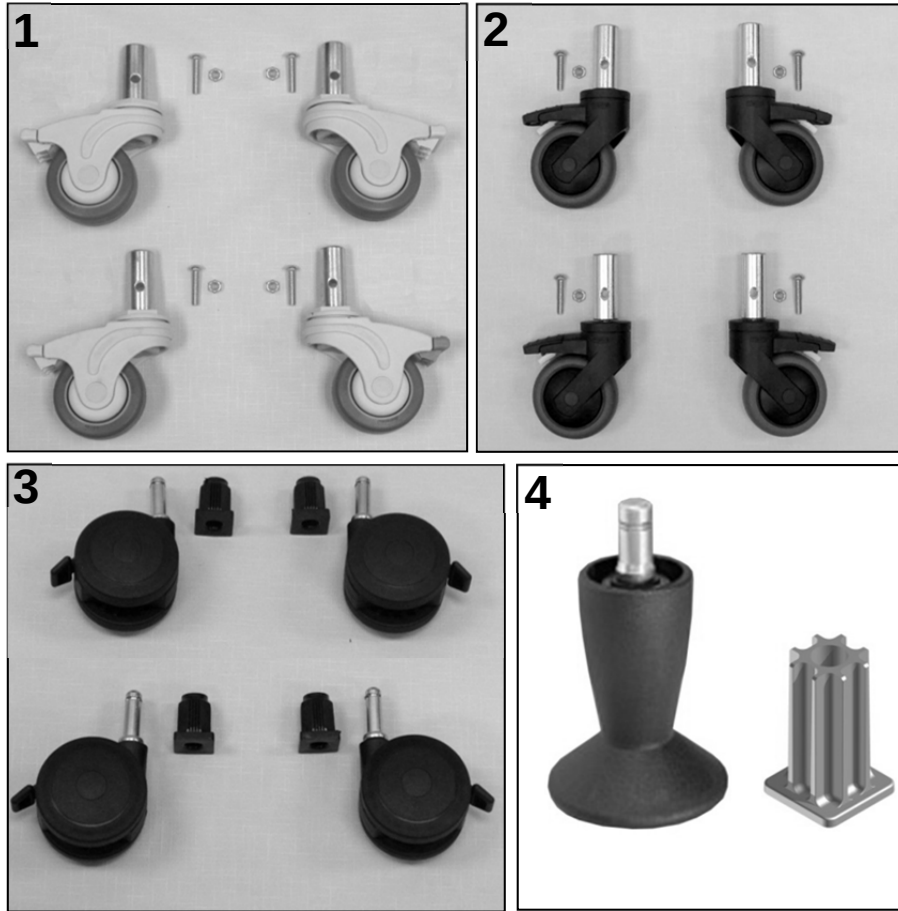



PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		400530	CASTER, NYLON 3" TOTAL LOCK, LIGHT GRAY	3
1	A	400540	CASTER, NYLON 3" DIRECTIONAL LOCK, LIGHT GRAY	1
1		566505	SCREW-MACH 1/4-20 X 1 1/4 RND PHILLIP M/S ZINC	4
1		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
2	B	400280	CASTER, NYLON 3" WHEEL BRAKE, BLACK	4
2		566505	SCREW-MACH 1/4-20 X 1 1/4 RND PHILLIP M/S ZINC	4
2		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
3	C	400270	CASTER, NYLON 3" TWIN WHEEL- LOCK, BLACK	4
3	C	702270	SOCKET, CASTER INSERT 1" SQ 1, GRIP RING STEM	4
4	D	700500	PEDESTAL FOOT, PLASTIC	4
4	D	702260	STEM SOCKET, GRIP NECK, SQ, 7/16" X 1-15/16"	4

NOTES





Note	Description
A	400540 (DARK GRAY BRAKE TAB) is installed on the LEFT REAR of chair frame.
B	STANDARD for chairs with serial # 670A100029 to PRESENT.
C	For 6705 chair models. Both parts (IMAGE 3) MUST be ordered to switch from 670 <u>0</u> to 670 <u>5</u>
D	For 6704 chair models. Both parts (IMAGE 4) MUST be ordered to switch from 670 <u>0</u> to 670 <u>4</u>
	<i>Left & Right is determined - as if sitting in the chair</i>

6700-6704-6705 Premier Care Cliner Parts List

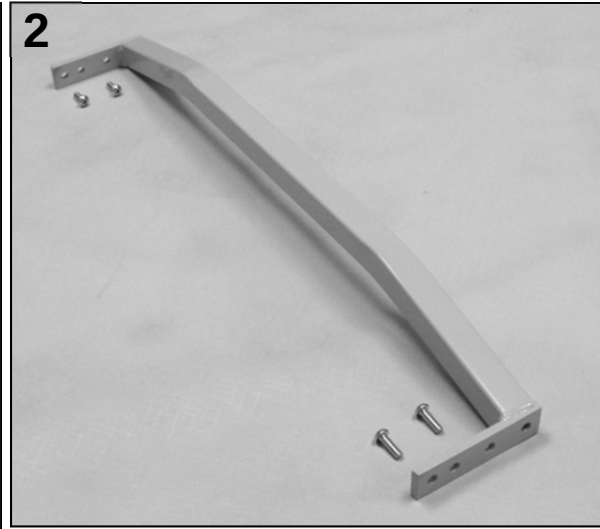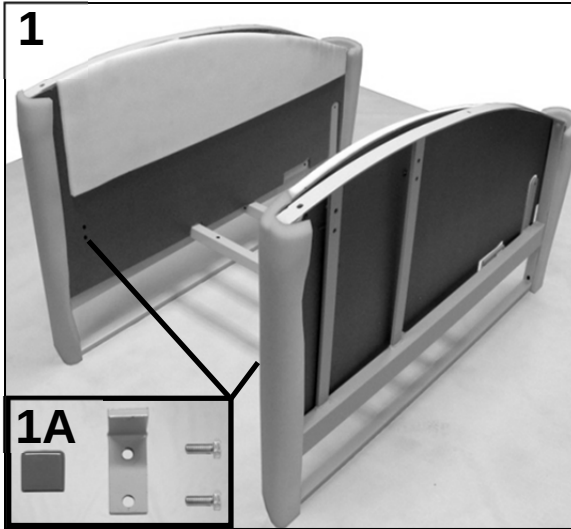

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1		N/A	557A MAIN FRAME	1
1A	C	216738	SEAT SUPPORT ANGLE BRKT	2
1A	C	566410	BOLT - 5/16 X 3/4 HEX HEAD	4
1A	C	700890	CAP, FLAT RECTANGULAR , GRAY, .25" X 1"	2
2	A	215580	WELDED REAR FRAME BRACE (557 style)	1
2		566504	SCREW-MACH 1/4-20 X 3/4 RND PHILLIP M/S ZINC	4
NOTES				
Note	Description			
A	Please contact Winco Customer Service/Tech Support (800-237-3377) for assistance with MAIN FRAME.			


6700-6704-6705 Premier Care Cliner Parts List

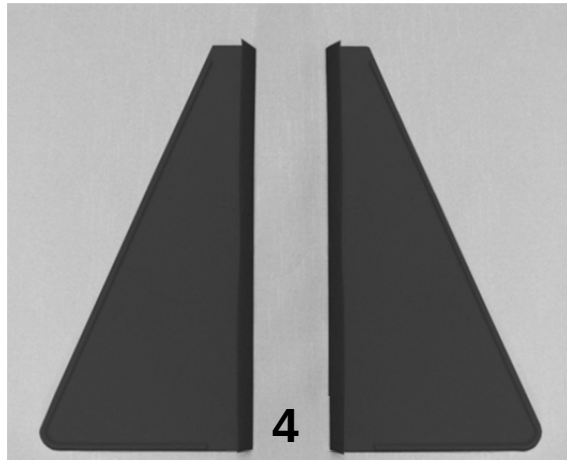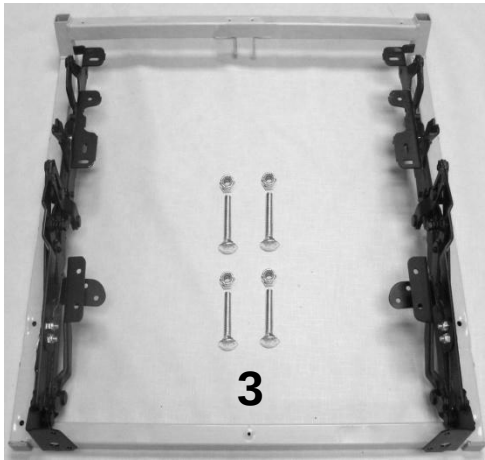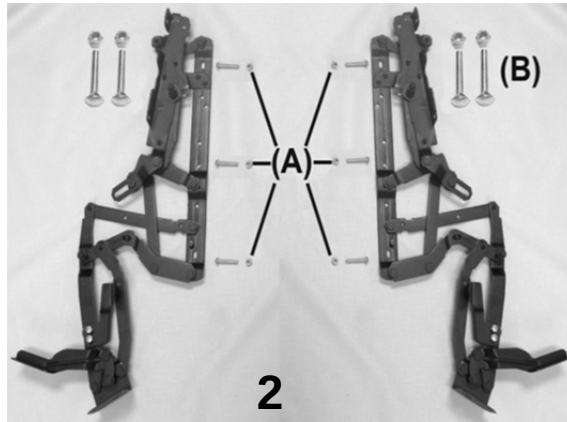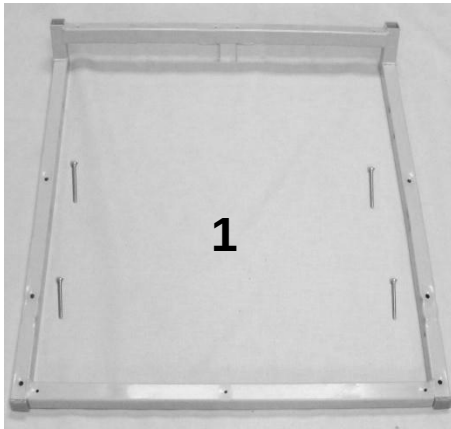

PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		216730K	557A STYLE SEAT FRAME KIT	1
1	A	216730	WELDED SEAT FRAME 557A	1
1	A	566515	SCREW-MACH 10-24 X 1 3/4 PAN PHILLIP M/S ZINC	4
1	A	700700	SQ 1" CAP, GRAY	2
2		200800K	653 MECHANISM KIT	1
2	A	400810	MECH 525/529/653 LEFT SIDE	1
2	A	400820	MECH 525/529/653 RIGHT SIDE	1
2	A	402670	MID OTTOMAN BRKT LEFT	1
2	A	402680	MID OTTOMAN BRKT RIGHT	1
2A	A	566506	SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC	6
2B, 3	A	566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
2	A	566504	SCREW-MACH 1/4-20 X 3/4 RND PHILLIP M/S ZINC	4
2A, 2B, 3	A	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	14
3	B	216740K	557A STYLE SEAT FRAME W/MECH KIT	1
4		704300	SAFETY SHIELD, BLACK (L&R PAIR)	1 PAIR
Note	Description			
A	It is NOT recommended that these parts be sold seperately however in special cases they can be.			
B	All parts listed above are included in this kit			
	<u>Left & Right is determined - as if sitting in the chair</u>			

6700-6704-6705 Premier Care Cliner Parts List

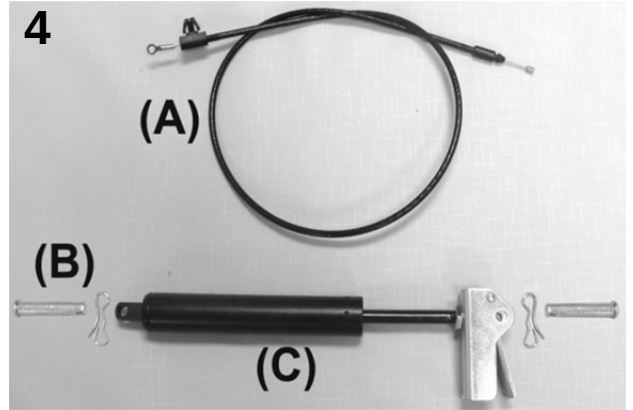



PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		215590W	BACK FRAME W/WEBBING (557-670 STYLE)	1
1, 4B	A	200025	CLEVIS/COTTER PIN ASSM. KIT	2
2		243246K	656 RELEASE HANDLE ASSM KIT	1
2	B	243246	656 RELEASE HANDLE	1
2	B	566512	SCREW-MACH 10-24 X 1 PAN PHILLIP M/S ZINC	1
2	B	566321	WASHER-SBUP 3/16 STEEL BACK WASHER	1
2	B	566219	CAP NUT, 10-24 STAINLESS STEEL	1
2	B	566310	WASHER-1/4 NYLON F/W (.275 X .50 X .0625)	3
2	B	566407	BOLT-1/4-20 X 1 1/4 HEX CAP SCREW GR 5 ZINC	2
2	B	566200	NUT-1/4-20 HEX FINISH NUT ZINC	2
3	C	U0216XX	671 UPHOLSTERED BACK	1
4A		203500	CONTROL CABLE REPLACEMENT KIT	1
N/A	D	566219	CAP NUT, 10-24 STAINLESS STEEL	1
4A	D	403500	CONTROL CABLE 34"	1
4B		200025	CLEVIS/COTTER PIN ASSM. KIT	2
4C	D	205210	CLINICAL CHAIR GAS CYL. W/OPERATOR ASSMD & ADJUSTED	1

NOTES

Note	Description
A	Parts are included with 215590W
B	It is NOT recommended that these parts be sold seperately however in special cases they can be.
C	XX- Use color code for specific part number i.e. 02 - Royal Blue
D	When ordering it is recommended that part # 200025 be ordered as well.
OPTIONS	For TB133 Add TB after Part Number i.e. U021602 TB
	For HM/HT option add HM or HT after TB option i.e. U021602 TBHM
	Use 00 if option is not needed i.e. U021602 00 HM for upholstered back without TB & with HM/HT

6700-6704-6705 Premier Care Cliner Parts List

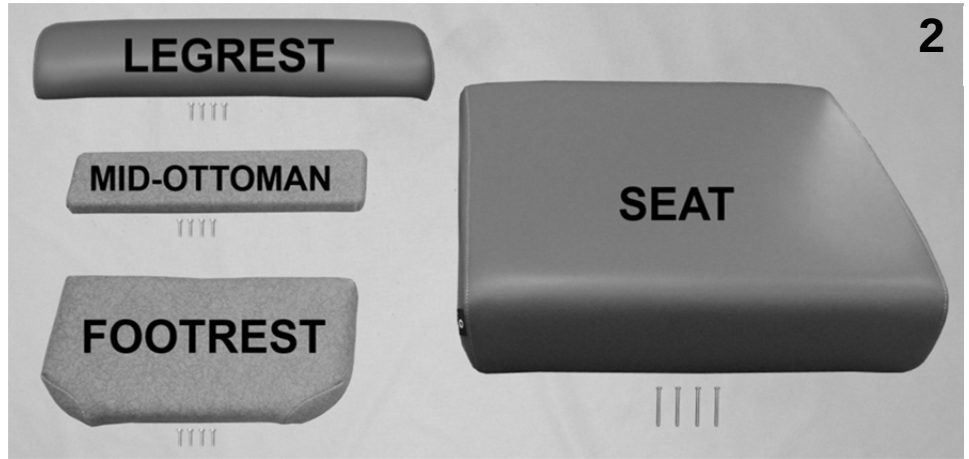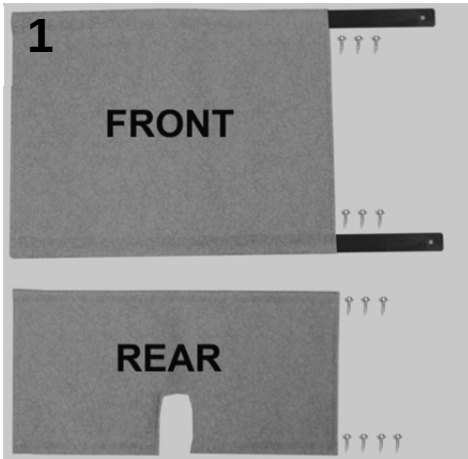


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1	A, B	U1403XX	FRONT APRON	1
1	A, B	U1503XX	REAR APRON	1
1		229910	APRON SLAT, FRONT (653 STYLE)	2
1		566534	SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC	13
2	A, B	U1011XX	UPHOLSTERED LEGREST	1
2	A, B	U1201XX	UPHOLSTERED MID-OTTOMAN	1
2	A, B	U1103XX	UPHOLSTERED FOOTREST	1
2		566511	SCREW-MACH 10-24 X <u>3/4</u> " PAN PHIL M/S ZINC	12
2	A, B	U0307XX	UPHOLSTERED SEAT W/ LIQUICELL	1
2		566515	SCREW-MACH 10-24 X <u>1 3/4</u> " PAN PHILLIP M/S ZINC	4

NOTES



Note	Description
A	XX- Use color code for specific part number i.e. 02 - Royal Blue
B	Screws needed to attach this product are included
OPTIONS	For TB133 Add TB after Part Number i.e. U0307XX TB
	For HM/HT option add HM or HT after TB option i.e. U0307XX TBHT
	Use 00 if option is not needed i.e. U0307XX 00 HTLC For Upholstered Seat without TB w/ HT & LC

6700-6704-6705 Premier Care Cliner Parts List

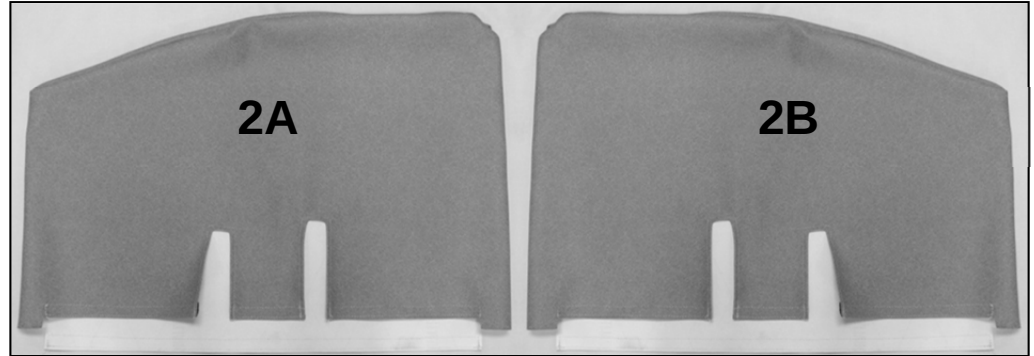

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1	A	208030LK	URETHANE ARM REPLACEMENT KIT (5400 STYLE) - LEFT	1
1	A	208030RK	URETHANE ARM REPLACEMENT KIT (5400 STYLE) - RIGHT	1
1	B	500115	SCREW, DOUBLE END, 1/4-20	6
N/A	B	680015	LOCTITE, THREADLOCKER, .02 OZ TUBE #262	1
1	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	6
2A		U0607XX	540-557-670 VINYL SIDE PANEL, LEFT	1
2B		U0707XX	540-557-670 VINYL SIDE PANEL, RIGHT	1
NOTES				
Note	Description			
A	Hardware needed to attach this product is included			
B	Parts are included in the above listed kits			
	<u>Left & Right is determined - as if sitting in the chair</u>			

6700-6704-6705 Premier Care Cliner Parts List

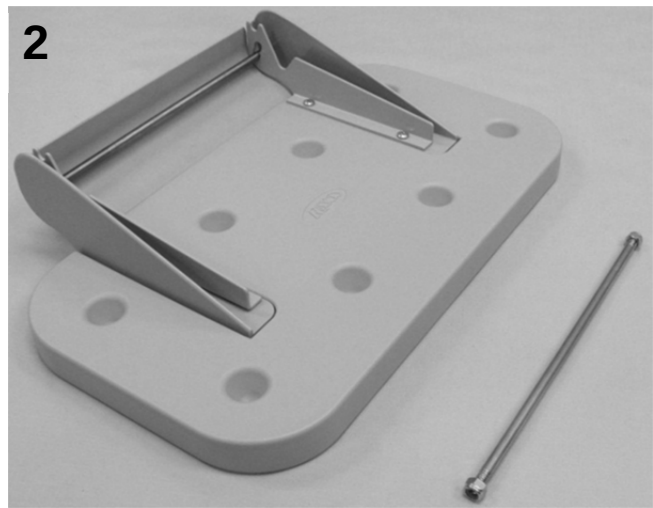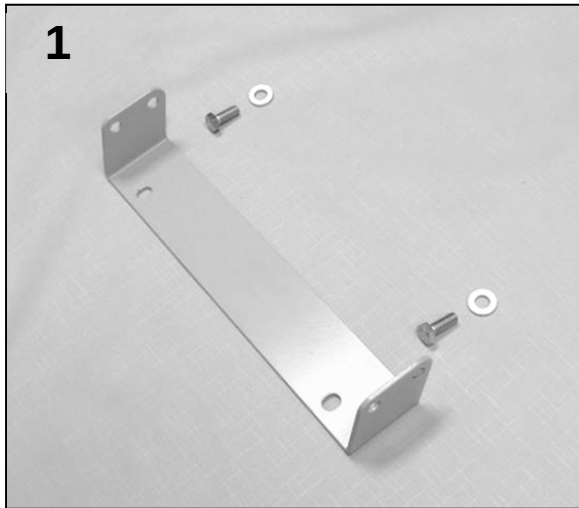

PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1 & 2	A	261640	SIDETABLE/COVER REPLACEMENT KIT BOLT-ON BRKT STYLE	2
1	B	402685	SIDE TABLE MOUNTING BRKT	2
1	B	566410	BOLT - 5/16 X 3/4 HEX HEAD	4
1	B	566304	WASHER-5/16 SAE FLAT WASHER ZINC	4
2	B	700425	SIDE TABLE, BLOW MOLDED (USED W/COVER)	2
2	B	700426	COVER, INJECTION MOLDED (FOR SIDE TABLE)	2
2	B	203150	SIDE TABLE HINGE BRKT (PR), GRAY	2
2	B	566561	#12 X 3/4" PANHEAD PHILLIPS TAP SCREW, ZINC	8
2	B	243270	5/16 SS ROD FOR SIDE TABLE	2
2	B	243273	THREADED ROD FOR SIDE TABLE KIT	2
2	B	566205	NUT-5/16-18 HEX NYLON LOCK NUT ZINC	4

NOTES

Note	Description
A	Kit includes ALL items listed below
B	It is NOT recommended that these parts be sold seperately however in special cases they can be.

For additional options please see:

[PIVOT TABLE OPTIONS PAGE](#)

[HM HT OPTIONS PAGE](#)

[IV POLE OPTIONS PAGE](#)

[OXYGEN TANK HOLDER OPTIONS PAGE](#)