
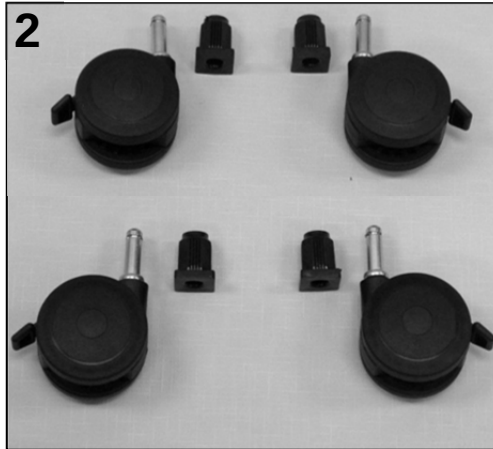


6710-6714-6715 XL Premier Care Cliner Parts List

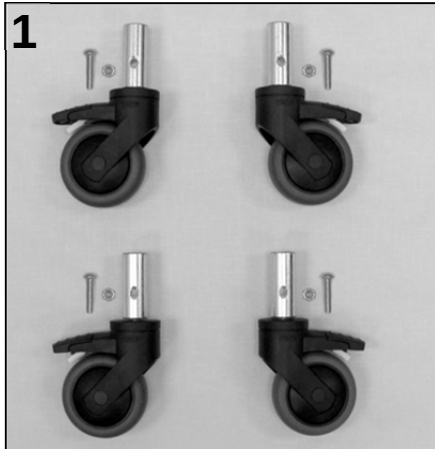



PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1	A	400280	CASTER, NYLON 3" WHEEL BRAKE, BLACK	4
1		566505	SCREW-MACH 1/4-20 X 1 1/4 RND PHILLIP M/S ZINC	4
1		566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	4
2	B	400270	CASTER, NYLON 3" TWIN WHEEL- LOCK, BLACK	4
2	B	702270	SOCKET, CASTER INSERT 1" SQ 1, GRIP RING STEM	4
3	C	700500	PEDESTAL FOOT, PLASTIC	4
3	C	702260	STEM SOCKET, GRIP NECK, SQ, 7/16" X 1-15/16"	4

NOTES

Note	Description
A	STANDARD for chairs with serial # 671100003 to PRESENT.
B	For 6715 chair models. Both parts (IMAGE 2) MUST be ordered to switch from 671 <u>0</u> to 671 <u>5</u>
C	For 6714 chair models. Both parts (IMAGE 3) MUST be ordered to switch from 671 <u>0</u> to 671 <u>4</u>
	<u>Left & Right is determined - as if sitting in the chair</u>

6710-6714-6715 XL Premier Care Cliner Parts List

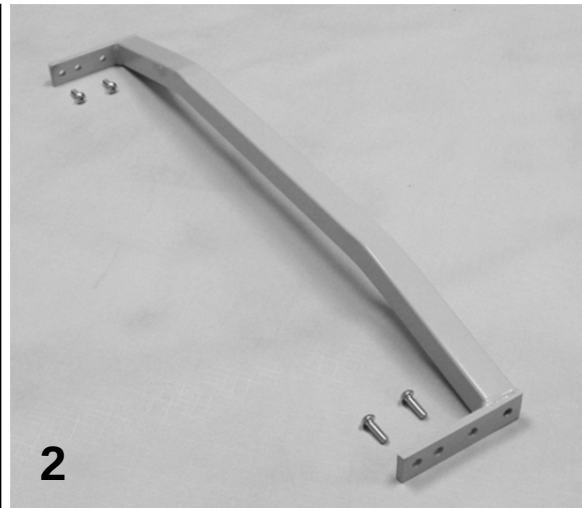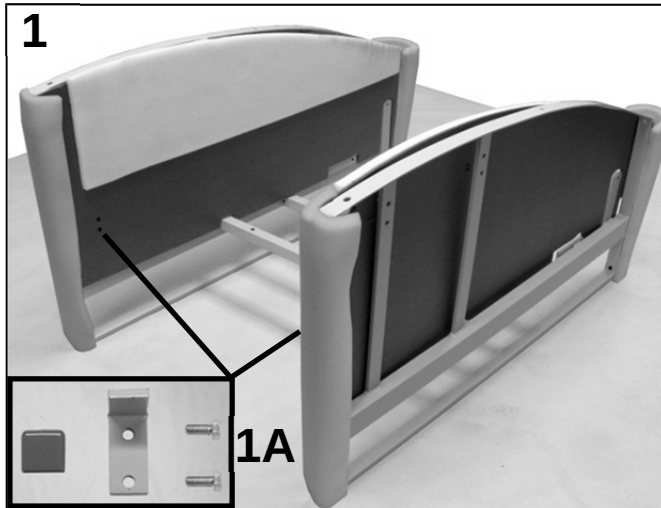


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1	A	N/A	671 MAIN FRAME	1
1A		216738	SEAT SUPPORT ANGLE BRKT	2
1A		566410	BOLT - 5/16 X 3/4 HEX HEAD	4
1A		700890	CAP, FLAT RECTANGULAR , GRAY, .25" X 1"	2
2		215580	WELDED REAR FRAME BRACE (557 style)	1
2		566504	SCREW-MACH 1/4-20 X 3/4 RND PHILLIP M/S ZINC	4

NOTES

Note	Description
A	Please contact Winco Customer Service/Tech Support (800-237-3377) for assistance with MAIN FRAME
	<u>Left & Right is determined - as if sitting in the chair</u>

6710-6714-6715 XL Premier Care Cliner Parts List

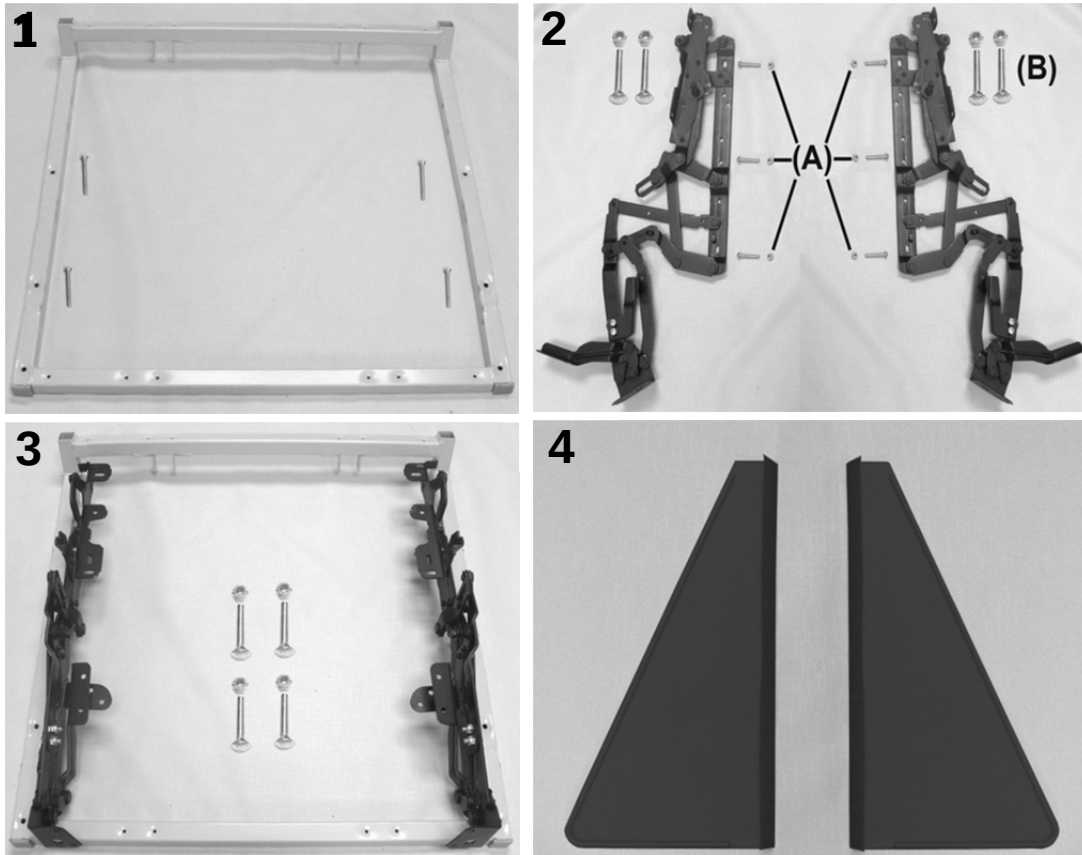


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		243211K	654 SEAT FRAME KIT	1
1	B	243211	529/654/657 SEAT FRAME WELDED	1
1	B	566515	SCREW-MACH 10-24 X 1 3/4 PAN PHILLIP M/S ZINC	4
1	B	700700	SQ 1" CAP, GRAY	2
2		200800K	525/529/653 MECHANISM KIT	1
2	B	400811	MECHANISM, RECLINER, LEFT 525/529/653	1
2	B	400821	MECHANISM, RECLINER, RIGHT 525/529/653	1
2	B	402670	MID OTTOMAN BRKT LEFT	1
2	B	402680	MID OTTOMAN BRKT RIGHT	1
2A	B	566506	SCREW-MACH 1/4-20 X 1 1/2 RND PHIL M/S ZINC	6
2B, 3	B	566401	BOLT-CARRIAGE 1/4-20 X 1 1/2 ZINC	4
2	B	566504	SCREW-MACH 1/4-20 X 3/4 RND PHILLIP M/S ZINC	4
2A, 2B, 3	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	14
3	A	243265	654 SEAT FRAME W/MECH KIT	1
4		704300	SAFETY SHIELD, BLACK (L&R PAIR)	1 PAIR
4		566536	SCREW-TAP 10 X 1 3/4 RND STD WASHER SQ T/S ZINC	2

NOTES

Note	Description
A	All parts listed in the previous 2 kits are included in this kit
B	It is NOT recommended that these parts be sold seperately however in special cases they can be.
	<u>Left & Right is determined - as if sitting in the chair</u>

6710-6714-6715 XL Premier Care Cliner Parts List

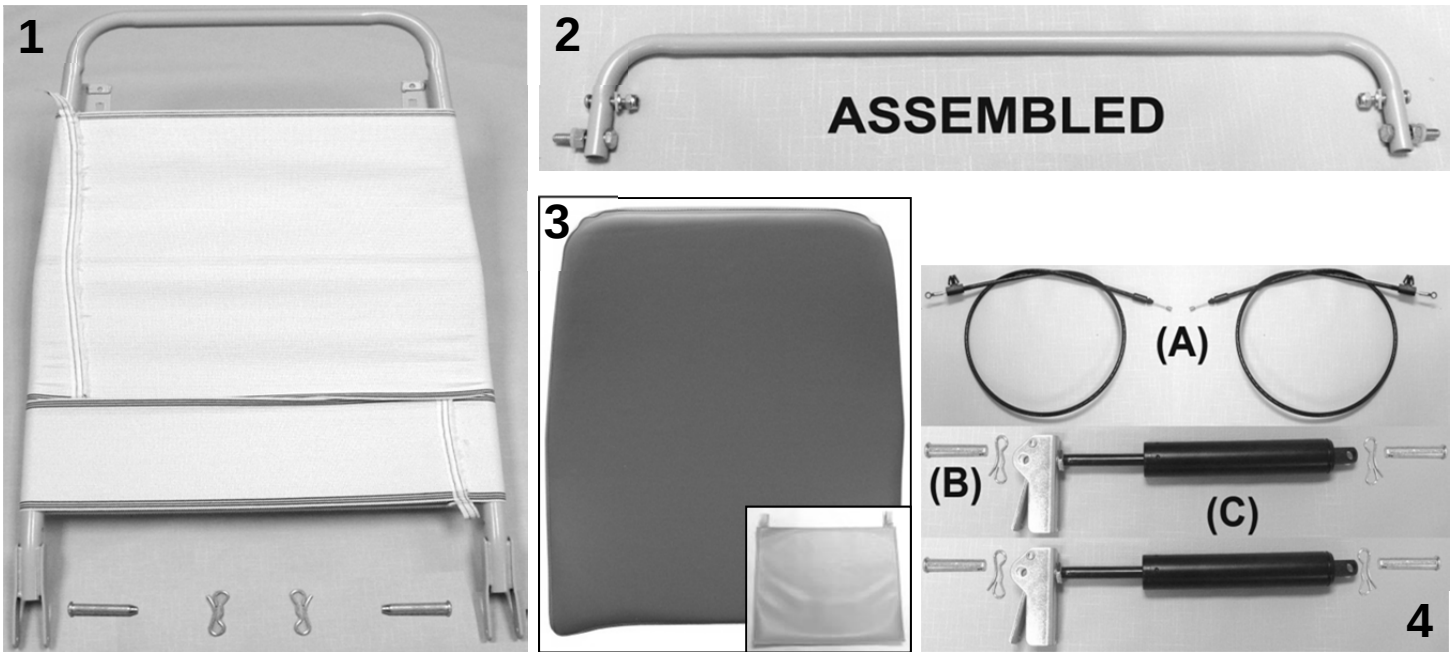



PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1		226720W	BACK FRAME W/WEBBING (671 STYLE)	1
1, 4B	A	200025	CLEVIS/COTTER PIN ASSM. KIT	2
2		226740K	671 RELEASE HANDLE ASSM KIT	1
2	B	226740	RELEASE HANDLE (671 style)	1
2	B	566512	SCREW-MACH 10-24 X 1 PAN PHILLIP M/S ZINC	2
2	B	566321	WASHER-SBUP 3/16 STEEL BACK WASHER	2
2	B	566219	CAP NUT, 10-24 STAINLESS STEEL	2
2	B	566310	WASHER-1/4 NYLON F/W (.275 X .50 X .0625)	4
2	B	566407	BOLT-1/4-20 X 1 1/4 HEX CAP SCREW GR 5 ZINC	2
2	B	566200	NUT-1/4-20 HEX FINISH NUT ZINC	2
3	C	U0217XX	671 UPHOLSTERED BACK	1
3	C,E	HC02XX	XL HEADREST COVER (671)	1
4A		203500	CONTROL CABLE REPLACEMENT KIT	2
N/A	D	566219	CAP NUT, 10-24 STAINLESS STEEL	2
4A	D	403500	CONTROL CABLE 34"	2
4B		200025	CLEVIS/COTTER PIN ASSM. KIT	4
4C	D	205210	CLINICAL CHAIR GAS CYL. W/OPERATOR ASSM'D & ADJUSTED	2

NOTES

Note	Description
A	Parts are included with 226720W
B	It is NOT recommended that these parts be sold seperately however in special cases they can be.
C	XX- Use color code for specific part number i.e. 02 - Royal Blue
D	When ordering it is recommended that part # 200025 be ordered as well.
E	HEADREST COVER is a separate item & is NOT included with the uph. back when ordering
OPTIONS	For TB133 Add TB after Part Number i.e. U021702 TB
	For HM/HT option add HM or HT after TB option i.e. U021702 TBHM
	Use 00 if option is not needed i.e. U021702 00 HM for upholstered back without TB & with HM/HT

6710-6714-6715 XL Premier Care Cliner Parts List

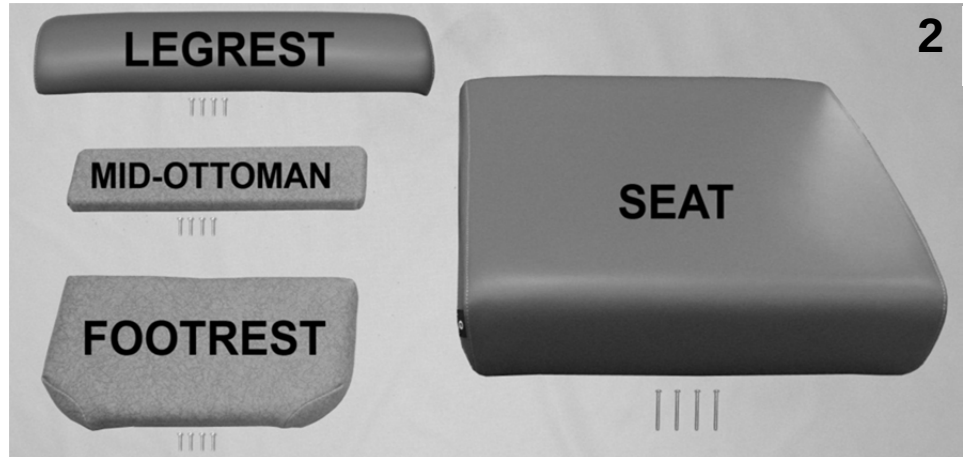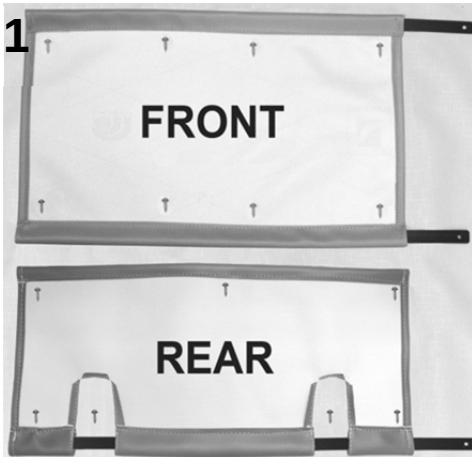


PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1	A, B	U1402XX	FRONT APRON	1
1	A, B	U1504XX	REAR APRON	1
1		229910	APRON SLAT, FRONT (653 STYLE)	2
1		566534	SCREW-TAP 10 X 3/4 RND WASHER SQ T/S ZINC	13
2	A, B	U1012XX	UPHOLSTERED LEGREST	1
2	A, B	U1301XX	UPHOLSTERED XL OTTOMAN	1
2	A, B	U1106XX	UPHOLSTERED FOOTREST	1
2		566511	SCREW-MACH 10-24 X <u>3/4</u> " PAN PHIL M/S ZINC	12
2	A, B	U0309XX	UPHOLSTERED SEAT W/ LIQUICELL	1
2		566515	SCREW-MACH 10-24 X <u>1 3/4</u> " PAN PHILLIP M/S ZINC	4

NOTES



Note	Description
A	XX- Use color code for specific part number i.e. 02 - Royal Blue
B	Screws needed to attach this product are included
OPTIONS	For TB133 Add TB after Part Number i.e. U0309XX TB
	For HM/HT option add HM or HT after TB option i.e. U0309XX TBHT
	Use 00 if option is not needed i.e. U0309XX 00 HTLC For Upholstered Seat without TB w/ HT & LC

6710-6714-6715 XL Premier Care Cliner Parts List

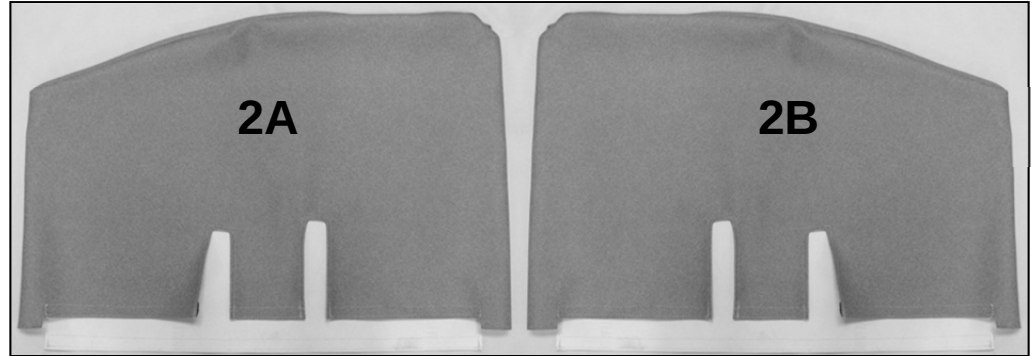

PARTS LIST				
Image Number	Note	Part Number	Description	QTY per Product
1	A	208030LK	URETHANE ARM REPLACEMENT KIT (5400 STYLE) - LEFT	1
1	A	208030RK	URETHANE ARM REPLACEMENT KIT (5400 STYLE) - RIGHT	1
1	B	500115	SCREW, DOUBLE END, 1/4-20	6
N/A	B	680015	LOCTITE, THREADLOCKER, .02 OZ TUBE #262	1
1	B	566204	NUT-1/4-20 HEX NYLON LOCKNUT ZINC	6
2A		U0607XX	540-557-670 VINYL SIDE PANEL, LEFT	1
2B		U0707XX	540-557-670 VINYL SIDE PANEL, RIGHT	1
NOTES				
Note	Description			
A	Hardware needed to attach this product is included			
B	Parts are included in the above listed kits			
	<i>Left & Right is determined - as if sitting in the chair</i>			

6710-6714-6715 XL Premier Care Cliner Parts List

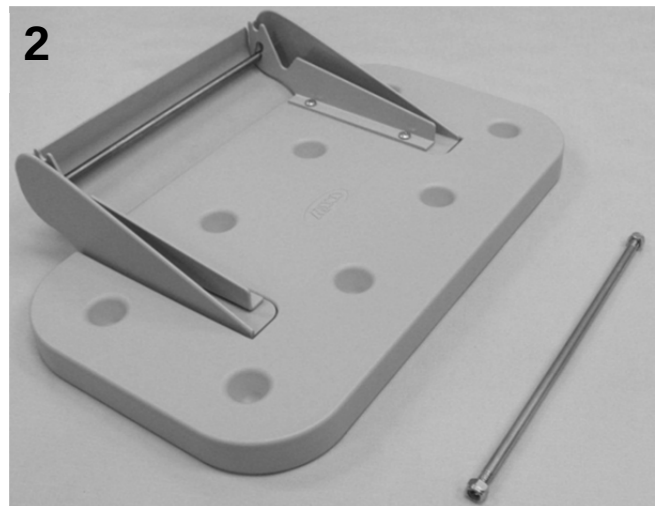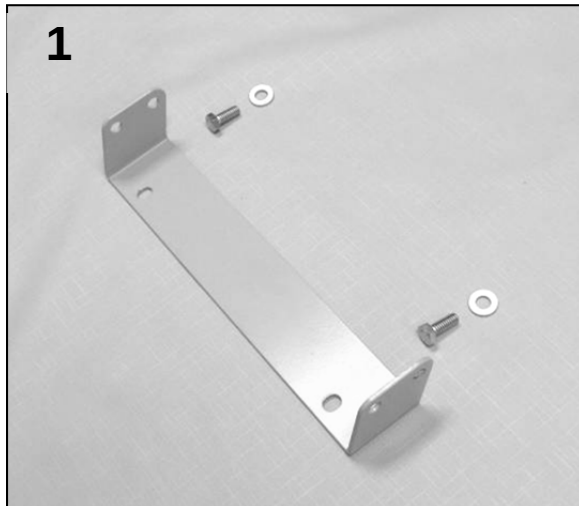

PARTS LIST

Image Number	Note	Part Number	Description	QTY per Product
1 & 2	A	261640	SIDETABLE/COVER REPLACEMENT KIT BOLT-ON BRKT STYLE	2
1	B	402685	SIDE TABLE MOUNTING BRKT	2
1	B	566410	BOLT - 5/16 X 3/4 HEX HEAD	4
1	B	566304	WASHER-5/16 SAE FLAT WASHER ZINC	4
2	B	700425	SIDE TABLE, BLOW MOLDED (USED W/COVER)	2
2	B	700426	COVER, INJECTION MOLDED (FOR SIDE TABLE)	2
2	B	203150	SIDE TABLE HINGE BRKT (PR), GRAY	2
2	B	566561	#12 X 3/4" PANHEAD PHILLIPS TAP SCREW, ZINC	8
2	B	243270	5/16 SS ROD FOR SIDE TABLE	2
2	B	243273	THREADED ROD FOR SIDE TABLE KIT	2
2	B	566205	NUT-5/16-18 HEX NYLON LOCK NUT ZINC	4

NOTES

Note	Description
A	Kit includes ALL items listed below
B	It is NOT recommended that these parts be sold seperately however in special cases they can be.

For additional options please see:

[PIVOT TABLE OPTIONS PAGE](#)

[HM HT OPTIONS PAGE](#)

[IV POLE OPTIONS PAGE](#)

[OXYGEN TANK HOLDER OPTIONS PAGE](#)