



Alere™ Influenza A & B Test

The Alere™ Influenza A & B Test provides a simple, rapid method as an aid in the differential diagnosis of influenza A & B using nasal swab specimens. This easy-to-use format and rapid results allow for point-of-care testing where it can provide information to assist with treatment and patient management decisions.

Features of the Alere™ Influenza A & B Test

- Results in 10 minutes
- Nasal swab sample
- CLIA Waived
- Dual reimbursement¹

Benefits of Rapid Diagnosis

- Guide appropriate therapy
- Avoid unnecessary antibiotic use
- Empower timely treatment decisions

 [Learn more](#)




Actual size

To learn more about rapid detection of **influenza A & B**, contact your local Account Executive.

Alere™ Influenza A & B Test Performance*

Format: Dipstick

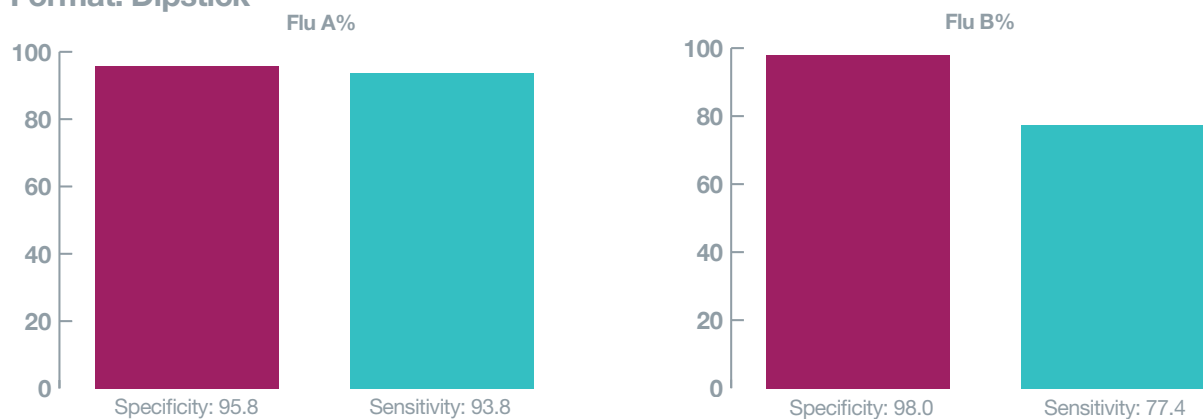


Easy to Use

- 1 Empty all of the liquid in an R1 vial into coated Reaction Tube.



- 2 Place the swab in the Tube and mix it in the liquid for about 10 seconds.

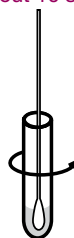

- While squeezing the Tube around the swab head, remove the swab from the Tube.



- 3 Add a Test Strip.




- 4 Wait 10 minutes, then read the result.



For complete instructions for use, see the package insert.

Easy to Read

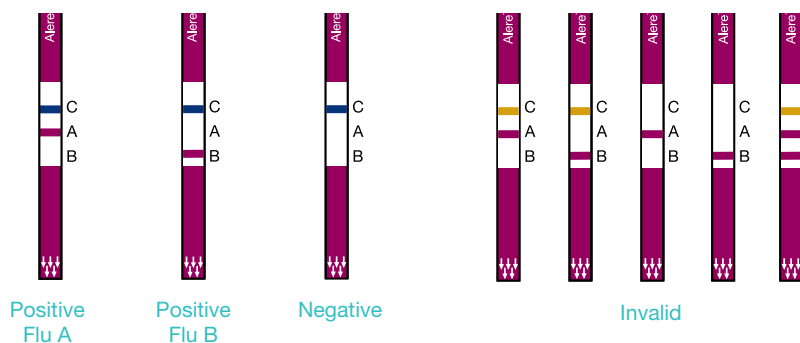

Order Information

Alere™ Influenza A & B Test
CLIA Waived
25 Test Kit
Order No. **412-000**
CPT: 87804/QW

Call **1.877.441.7440** or visit **alere.com** today

1. Actual reimbursement may vary.

*Versus viral culture.

© 2012 Alere. All rights reserved. The Alere Logo and Alere are trademarks of the Alere group of companies. 12000333E-01 09/13