



# 758C

## DIGITAL WEIGHT INDICATOR



**PATIENT ID  
PRINTING**

Detecto's enhanced model 758C indicator offers the best solution to patient weighing. Whether you require a simple display of patient weight or the full benefits of printing or sending patient ID and weight

data to a PC, the 758C is your best choice. With BMI, automatic display lock, pushbutton and keypad tare, the 758C is excellent for doctors' offices, hospitals, weight loss clinics, and long term care facilities.

### **SERIAL CONNECTIVITY PROVIDES:**

**Output To Optional Printers** - print patient ID, weight, time, date, and BMI

**Send Data To A PC** - Excellent for transmitting data to medical records software

# The New 758C Digital Weight Indicator



11:23, 02/15/05  
ID: 12548796  
BMI: 28.6  
Height: 6' 0.1"  
Weight: 212.4 lb G




Model P220 Thermal Label Printer



Model P185 Ticket Printer

## SIMPLE TO USE

For weighing only use, the yellow "quick keys" (pictured to the right) provide fast, easy operation, even by first time users. The full complement of displayed information makes it easy to read the data at a glance and the display hold function can be set for manual operation or automatic activation on stable weight.

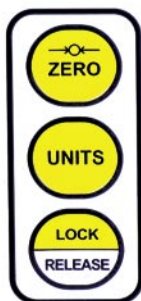

The new serial port allows output of the 758C's data directly to a printer by simply pressing the PRINT/ENTER key.



## PATIENT ID

Enter up to an 11 digit patient ID that can be printed on a label or ticket or sent with the weight data to a PC. Detecto can provide the optional P220 thermal label printer or P185 ticket printer to professionally document your patients' weight by time and date for medical files. You can also print the patient height if it is required for their records.



The 758C is **battery powered** for use with portable scales, so you can take the scale to the patient. Optional AC adapter available.

## BODY MASS INDEX

Body Mass Index (BMI) measurements have been shown to be an accurate estimate of health risk when used as part of a comprehensive health assessment. The 758C quickly calculates BMI by simply entering the patient's height while weighing.



## 758C SPECIFICATIONS

<b>Display:</b>	Liquid Crystal, Reflective Type
<b>Weight:</b>	5 digit, 7 segment 0.5" high
<b>BMI:</b>	3 digit, 7 segment 0.5" high
<b>Height-ID:</b>	11 digits entered - 4 digits displayed, 7 segment 0.3" high
<b>Bar Graph:</b>	8 segment 0.3" high
<b>Enclosure:</b>	Painted steel
<b>Keypad:</b>	19 key, color-coded, sealed membrane
<b>BMI Range:</b>	15.0 - 60.0
<b>Height Range:</b>	3 to 7 ft / 90 to 215 cm
<b>Sensitivity:</b>	1.2 $\mu$ V/d
<b>Resolution:</b>	5,000 divisions, maximum
<b>Accuracy:</b>	Class III
<b>Power:</b>	6 "C" size Alkaline or Nickel-Cadium cells (not included) or optional AC adapter.
<b>Temperature Range:</b>	+14 to +104 °F / -10 to +40 °C



The 758C is now standard on Detecto's complete line of digital clinical scales.

Cardinal Scale reserves the right to improve, enhance or modify features and specifications without prior notice. All registered trademarks are the property of their respective owners.

## SOLD BY:



A Division of Cardinal Scale Manufacturing Co.

203 E. Daugherty, Webb City, MO 64870 USA

Ph: 417-673-4631 or 1-800-641-2008 Fax: 417-673-5001

www.DetectoScale.com