

New & Improved

Therma Moist Heating Pad

10895	Petite, 7" x 15", 1/ea
10890	Standard, 14" x 27", 1/ea
10893	Medium, 14" x 14", 1/ea

FEATURES:

- The positioning strap with Velcro-type closure ensures a secure, proper fit for maximum treatment.
- The easy to use hand control features four heat settings and four time settings.
- Ideal in the treatment of arthritis, back pain, sprains and strains, muscle soreness, neck pain, headaches and sports injuries.
- Moist heat therapy is more effective than dry heat because moisture transmits heat more efficiently than air.
- The Therma Moist Heating Pad has an outer flannel cover that draws the air's humidity and retains it. As the heating pad warms, the moisture is released the flannel cover and onto the treated area.
- The heating pad will automatically shut off after treatment.
- Three sizes are available to provide added convenience.
- 1 Year Limited Warranty.

TECHNICAL SPECIFICATIONS:


- Heat settings: 115/130/150/165 degrees Fahrenheit, 45/55/65/75 Celsius.
- Time settings: 15/30/45/60 minutes.
- Power consumption: 90-130 Volts, 40 - 70 Hz.
- UL/cUL/TUV Listed
- Avoid sharp folds in pad.
- Do not expose pad to liquids.
- Do not place pad directly over cuts, abrasions or open wounds.
- To clean, remove cover from heating pad. Machine or hand wash. Use mild detergent in cool water. Hang to dry or use set on cool gentle cycle in dryer.



10890



10895

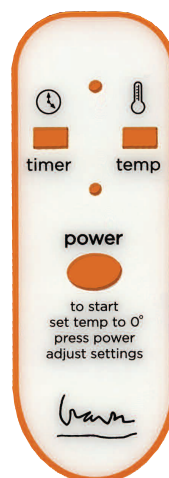

NEW & IMPROVED

TO PREVENT ACCIDENTS, HAND CONTROL WILL NOT ALLOW UNIT TO TURN ON UNLESS THE TEMPERATURE IS SET AT 0°



10893



Therma Moist Heating Pad Floor Display

10899

- Contains floor display and two (2) each standard (item # 10890) medium (item #10893) and petite (item # 10895) moist heating pads.



personal care

drive
WWW.DRIVEMEDICAL.COM

To order call toll free: 877.224.0946

Important Notice: Item numbers that display an asterisk () reflect an item that will be discontinued or package change.