

Cystoscopy "T" Drape

(89548 depicted)



1. To apply leggings, grasp legging where toe is marked with left hand while placing right hand within marked pocket.



2. Holding onto toe of legging, slip drape over stirrup. Repeat for the other leg.




3. Orient the "Head" Stamp on the drape with the patient.




4. Next, remove the adhesive strip liner and position the drape on the patient's lower abdomen.



5. Unfold the drape laterally and then downward, between the patient's legs, exposing the perineal fenestration and specimen collection screen.

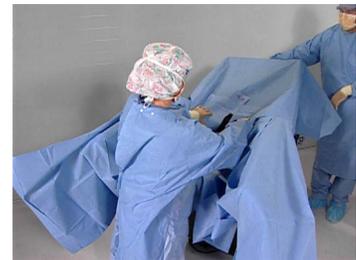

6. Continue to unfold the drape toward the patient's head. Adjust the perineal fenestration and specimen collection screen.

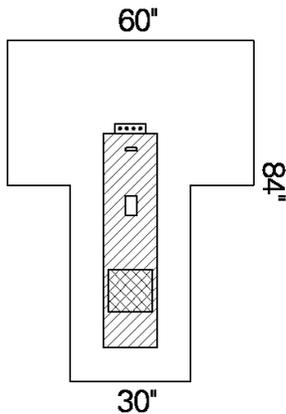

Features

- 60" x 30" x 84"
- 3" x 6" Fenestration
- 9" x 9" Screen
- CONTROL* Fabric Reinforcement
- Tube Holders

Stock

79540 ♦

Description

Cystoscopy "T" Drape, Non-Sterile, Bulk Pack

Items Per Case

50

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Pack Stock #
88611

Description
Cystoscopy Pack I

Packs/Case
12

Component Stock #

Sterile Non-Sterile ♦

79448

Contents:

1 Back Table Cover, Zone-Reinforced, 44" x 78"

89701 79720

1 Absorbent Towel, 15" x 26"

89408 79408

2 Leggings, with 6" Cuffs

79540

1 Cystoscopy "T" Drape, 60" x 30" x 84"

- 3" x 6" Perineal Fenestration
- 9" x 9" Screen
- CONTROL* Fabric Reinforcement
- Tube Holders

Pack Stock #
88631

Description
Cystoscopy Pack III

Packs/Case
9

Component Stock #

Sterile Non-Sterile ♦

79448

Contents:

1 Back Table Cover, Zone-Reinforced, 44" x 78"

95511 70511

1 ULTRA Specialty Surgical Gown, Large, with Towel

89408 79408

2 Leggings, with 6" Cuffs

79540

1 Cystoscopy "T" Drape, 60" x 30" x 84"

- 3" x 6" Perineal Fenestration
- 9" x 9" Screen
- CONTROL* Fabric Reinforcement
- Tube Holders

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.

Copyright 2002 Kimberly-Clark Corporation. All Rights Reserved. Your visit to this site and use of the information hereon is subject to the terms of our Legal Statement. *Registered Trademark or Trademark of Kimberly-Clark Corporation or its affiliates.

<See the following page for Pack Options>