

U-Drape

(No video available)



1. Remove adhesive strip liner. Locate "Base of Incision" mark on drape and orient as such.




2. Unfold laterally, then toward patient's feet to expose tails of drape.



3. Remove adhesive strip liner on tails, and conform tails lateral to operative site.



4. A Bar Drape will be needed to seal off operative site. Insert surgical lines and tubing through the tube holders to complete draping.

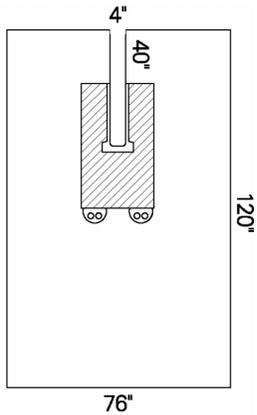

Features

- 76" x 120"
- 4" x 40" Split
- CONTROL* Fabric Reinforcement
- Tube Holders

Draping Hints

- This drape is recommended for any procedures except extremities.

Stock

Description

Items Per Case

89301

U-Drape, Sterile, 1 per package

17

79301

U-Drape, Non-Sterile, Bulk Pack

19

† As with any drape or pack recommendation, it is the hospital's decision as to the selection of the appropriate draping technique.

Pack Stock #	Description	Packs/Case
88311	<i>U-BAR* Pack I</i>	8

Component Stock #

Sterile	Non-Sterile ♦	Contents:
89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"
89701	79720	1 Absorbent Towel, 15" x 26"
89731	79731	4 Utility Drapes, 26" x 15", with Tape
		1 Suture Bag
89301	79301	1 U-Drape, 76" x 120", 4" x 40" Split <ul style="list-style-type: none"> • CONTROL* Fabric Reinforcement • Tube Holders
	79370	1 Bar Drape, 96" x 44" <ul style="list-style-type: none"> • CONTROL* Fabric Reinforcement • Tube Holders

Pack Stock #	Description	Packs/Case
88321	<i>U-BAR* Pack II</i>	5

Component Stock #

Sterile	Non-Sterile ♦	Contents:
89618	79498	1 Back Table Cover, Zone-Reinforced, 44" x 90"
95211	70211	2 ULTRA Fabric-Reinforced Surgical Gowns, Large with Towels
89601	79601	1 Mayo Stand Cover, Reinforced, 23" x 54"
89731	79731	4 Utility Drapes, 26" x 15", with Tape
		1 Suture Bag
89301	79301	1 U-Drape, 76" x 120", 4" x 40" Split <ul style="list-style-type: none"> • CONTROL* Fabric Reinforcement • Tube Holders
	79370	1 Bar Drape, 96" x 44" <ul style="list-style-type: none"> • CONTROL* Fabric Reinforcement • Tube Holders

♦ Except as noted, all non-sterile products are for custom trays and available only through your custom tray supplier.