

# 5300 ERGOGLIDE



The ErgoGlide one way glide is used to help maintain good sitting position by allowing for easy repositioning of patients in chairs. The friction reducing surface minimizes forces required to achieve a proper seated position for patients. The locking material reduces the risk of the patient slipping forward.

**17.75" x 17.75" (45 x 45cm)**

**ERGOSAFE PRODUCTS**  
A PRISM MEDICAL COMPANY



# 5300 ERGOGLIDE

## USE INSTRUCTIONS:

### If the patient is placed in wheelchair by a mechanical lift

1. Place the ErgoGlide with the label visible and the arrow pointing toward the back of the chair to ensure proper placement of Glide.
2. When the patient is placed all the way in the chair, the ErgoGlide will keep the patient from sliding down in the chair.

### If the patient is getting in the chair on their own and can not get to the back of the chair

1. With the ErgoGlide's label visible and the arrow pointing toward the back of the chair place the ErgoGlide a few inches from the back of the chair.
2. Once patient is in sitting position, the patient can be assisted back into the chair with minimal effort.

Regular assessment of the patient's skin is essential if using the ErgoGlide for extended periods of time. Consultation with a Registered Occupational or Physical Therapist is advised if the ErgoGlide is being considered for use with patients who have or are at risk of developing a pressure ulcer.

## LAUNDRY INSTRUCTIONS:

The ErgoGlide® can be machine-washed at 80°C/170°F. Hang to dry or tumble dry on low heat at 40°C/104°F.

For more information on this and other ErgoSafe® products visit [www.ergosafe-products.com](http://www.ergosafe-products.com)

© 2009 Prism Medical 45 Progress Parkway, Maryland Heights, MO. 63043

**ERGOSAFE PRODUCTS**  
A PRISM MEDICAL COMPANY