

osom[®]

Ultra hCG Combo Rapid Test

FOR THE DETECTION OF HUMAN CHORIONIC GONADOTROPIN IN URINE OR SERUM



FEATURES:

- ✓ **ACCURATE;** 100% & 100% (Sensitivity & Specificity) on Urine; 100% & 99% on Serum
- ✓ **FAST;** Results in 3-5 minutes
- ✓ **SAMPLE;** Single step procedure with <30 seconds hands on time
- ✓ **OBJECTIVE;** Detects β -core fragments[®]
- ✓ **CUSTOM CARE;** Technical support staffed with certified medical technologists



OSOM®

Ultra hCG Combo Rapid Test

Sekisui Diagnostics, LLC (AMERICAS)
4 Hartwell Place Lexington, MA 02421
Phone: 800 332 1042
Fax: 800 762 6311
Email: osomrapidtests@sekisui-dx.com
ISO 13485 Certified

Sekisui Diagnostics (UK) Limited (INTERNATIONAL)
Liphook Way, Allington, Maidstone, Kent, ME16 0LQ, UK
Phone: +44 1622 607800 Fax: +44 1622 607801
Email: info@sekisui-dx.com
ISO 9001 Certified, ISO 13485 Certified

TEST PROCEDURE / RESULTS

STEP 1

For each test, open one OSOM hCG Serum/ Urine pouch, and label the OSOM device with the patient ID.



STEP 2

Add 3 drops of sample into the Sample Well.

STEP 3

Read the results at 3–5 minutes.



Positive

Two pinkish-purple lines, one each at the Test Position (T) and at the Control Position (C) indicates a positive test result.



Negative

Only one pinkish-purple line, at the Control Position (C).



Invalid

A distinctive colored line at the Control Position (C) should always appear. The test is invalid if no Control line forms.

DESCRIPTION	CONFIGURATION	CATALOG #	STORAGE	CPT CODE
OSOM ULTRA HCG COMBO TEST	25 TESTS	1004	ROOM TEMP	URINE: 81025 SERUM: 84703
OSOM HCG URINE CONTROL SET	2 X 10-ML VIALS (POS & NEG)	134	REFRIGERATED	
OSOM HCG SERUM CONTROL SET	2 X 5-ML VIALS (POS & NEG)	138	REFRIGERATED	

Reference
* Data on file

OSOM® is a registered U.S. trademark of Sekisui Diagnostics, LLC. © 2015 Sekisui Diagnostics, LLC. All rights reserved.

