

Products designed with a purpose...
...to help you help them.



Surshield™ Safety Winged Infusion and Blood Collection Sets

***Help shield your staff against accidental
Needle-sticks with Surshield***

Terumo Surshield Safety Winged Needle Sets provide:

- Simple, one-handed activation
- Minimal steps for activation
- No change in needle insertion technique required
- Confirmation that needle is securely locked

Surshield Safety Winged Needle Sets have the soft, flexible, interlocking wings that readily conform to body's contour. The needles are bonded to the wings at just under a 3° downward angle, for less risk of vessel damage or occlusion of the lumen. The coil-resistant tubing reduces recoil during use and is both Latex and DEHP free.

LATEX FREE

 **TERUMO®**

TERUMO MEDICAL PRODUCTS

Surshield™ Safety Winged Infusion and Blood Collection Sets


- One-handed*, easy-to-activate safety device
- Minimize the risk of needlestick injuries

*Note: OSHA bloodborne pathogen standard 1910.1030(d)(2)(vii)(B)


Terumo Safety Winged Blood Collection and Infusion Sets with an added safety feature

- No change in needle insertion technique required
- Simple, built-in safety design is easy to use
- Latex-Free, DEHP Free

How to Activate

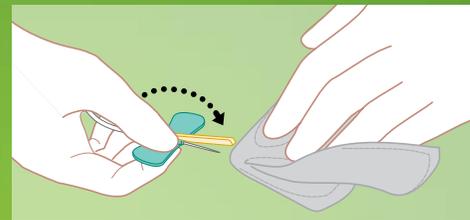

1. Flip safety shield toward needle. Remove the needle and apply digital pressure to the site.

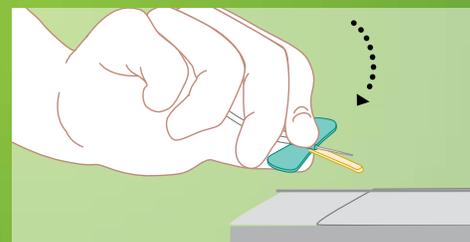

2. With the wing and shield between your thumb and index finger, press the safety shield against a hard surface such as a bedside table until an audible click is heard.



3. Visually confirm activation of the safety feature.

TerumoTMP.com/SurShield

US Office

2101 Cottontail Lane
Somerset, NJ 08873
Phone: 732.302.4900
Fax: 732.302.3093
Toll free: 800.283.7866
Fax for Orders: 800.411.5870

Canada Office

Terumo Medical Corporation
C/O UPS - Supply Chain Solutions
1165 Creditstone Road
Vaughan, Ontario L4K 4N7
Phone: 800.668.5489
Fax: 905.760.1215

 and TERUMO are trademarks owned by Terumo Corporation, Tokyo, Japan, and they are registered with the U.S. Patent & Trademark Office. Surshield is a trademark owned by Terumo Medical Corporation, Elkton, MD, U.S.A.

Surshield Safety Winged Blood Collection Sets (with attached multi-sample luer adapter)

Packed: 50/box, 10 boxes/case (500)

Product Code	Description
MN*SVS21B30	21G x 3/4" Needle and 12" Tubing
MN*SVS23B30	23G x 3/4" Needle and 12" Tubing
MN*SVS25B30	25G x 3/4" Needle and 12" Tubing
MN*SVS21B18	21G x 3/4" Needle and 7" Tubing
MN*SVS23B18	23G x 3/4" Needle and 7" Tubing

Surshield Safety Winged Infusion Sets (without multi-sample luer adapter)

Packed: 50/box, 10 boxes/case (500)

Product Code	Description
SV*S19BL	19G x 3/4" Needle and 12" Tubing
SV*S21BL	21G x 3/4" Needle and 12" Tubing
SV*S23BL	23G x 3/4" Needle and 12" Tubing
SV*S25BL	25G x 3/4" Needle and 12" Tubing
SV*S19BLS	19G x 3/4" Needle and 3 1/2" Tubing
SV*S21BLS	21G x 3/4" Needle and 3 1/2" Tubing
SV*S23BLS	23G x 3/4" Needle and 3 1/2" Tubing
SV*S25BLS	25G x 3/4" Needle and 3 1/2" Tubing

Multi-Sample Luer Adapter

Packed: 100/box, 30 boxes/case (3,000)

Product Code	Description
MN*2000T	VENOJECT Multi-Sample Luer Adapter (male). For use with Standard Tube Holder and SampLok® Tube Holder. 100/box, 30 boxes/case (3000/case)

Venoject® II Tube Holder

Packed: 50/package, 20 packages/case (1,000)

Product Code	Description
P-1316R	VENOJECT II (Standard) Tube Holder. Use with the multi-sample luer adapter and Diversion Blood Sampling Arm® and Blood Sampling Arm® blood bags to collect samples. 50/bag, 20 bags/case (1000/case)