

Products designed with a purpose...
...to help you help them.



SurGuard2™ Safety Needles & Syringes

SAFETY

- Unique needle locking mechanism helps reduce risk of needlestick injury
- An audible click* indicates device activation

EASE OF USE

- One-handed activation
- Simple, intuitive, no complicated mechanisms

CONVENIENCE

- Small, compact packaging reduces waste
- Broad product line, includes 25Gx1½" and 30Gx½" safety needle

Low cost

- Offered as needle-only or combination product

LATEX FREE

 **TERUMO**®

TERUMO MEDICAL PRODUCTS

SurGuard2™ Safety Needles & Syringes

Needle Protection Systems Avoid a Stick with a Single Click

The perfect solution to your safety compliance issues, SurGuard2 meets OSHA regulation for a sharps device with a built-in safety feature.

To learn more about SurGuard2 or to acquire samples for evaluation, call our customer support team at 800.283.7866 ext. 4919 or visit our Web site: www.TerumoTMP.com/SurGuard2.

Prior to Activation: Sleek, thin, more streamlined sheath. Ratchet mechanism allows for positioning of sheath so it does not obscure injection site. Standard hub fits all Luer Lock and Luer Slip syringes. Bevel-up, sheath right needle orientation allows for low-angle injections.

After Activation: Needle remains fully engaged under locking mechanism.

PRODUCT PREPARATION



1. Securely attach SurGuard2 needle to syringe with a push and clockwise twist motion. For syringe with needle, be sure to tighten assembly firmly before use.



2. Move sheath away from the needle to an approximate 90° angle. Sheath will remain as set.



3. Remove needle cover carefully by pulling straight off. Do not twist.

PRODUCT ACTIVATION



POSITION

1. Position sheath about 40°-45° to a hard, flat surface.



LOCK

2. With a FIRM, quick motion, press down against a flat surface until an audible click is heard.

*NOTE: Audible click may not be heard on small needle sizes; visual confirmation is required.



CONFIRM

3. Visually confirm that the sheath is fully engaged.

ORDERING INFORMATION

SurGuard2 Safety Needle

Packed: 100/box, 8 boxes/case (800)

SIZE REORDER NO.

Product Code	Description (with Sheath)
SG2-1825	18G x 1" needle
SG2-1838	18G x 1 1/2" needle
SG2-1925	19G x 1" needle
SG2-1938	19G x 1 1/2" needle
SG2-2025	20G x 1" needle
SG2-2038	20G x 1 1/2" needle
SG2-2125	21G x 1" needle
SG2-2138	21G x 1 1/2" needle
SG2-2225	22G x 1" needle
SG2-2238	22G x 1 1/2" needle
SG2-2325	23G x 1" needle
SG2-2338	23G x 1 1/2" needle
SG2-2516	25G x 5/8" needle
SG2-2525	25G x 1" needle
SG2-2538	25G x 1 1/2" needle
SG2-2613	26G x 1/2" needle
SG2-2713	27G x 1/2" needle
SG2-3013	30G x 1/2" needle

SurGuard2 Syringe with Safety Needle

Packed: 100/box, 4 boxes/case (400)

SIZE REORDER NO.

Product Code	Description
SG2-01T2516	1cc 25G x 5/8" needle
SG2-01T2609	1cc 26G x 3/8" needle
SG2-01T2713	1cc 27G x 1/2" needle
SG2-03L2025	3cc 20G x 1" needle
SG2-03L2038	3cc 20G x 1 1/2" needle
SG2-03L2125	3cc 21G x 1" needle
SG2-03L2138	3cc 21G x 1 1/2" needle
SG2-03L2225	3cc 22G x 1" needle
SG2-03L2238	3cc 22G x 1 1/2" needle
SG2-03L2325	3cc 23G x 1" needle
SG2-03L2516	3cc 25G x 5/8" needle
SG2-03L2525	3cc 25G x 1" needle
SG2-05L2025	5cc 20G x 1" needle
SG2-05L2038	5cc 20G x 1 1/2" needle
SG2-05L2138	5cc 21G x 1 1/2" needle
SG2-10L2025	10cc 20G x 1" needle
SG2-10L2038	10cc 20G x 1 1/2" needle
SG2-10L2125	10cc 21G x 1" needle

TerumoTMP.com/SurGuard2

US Office

2101 Cottontail Lane
Somerset, NJ 08873
Phone: 732.302.4900
Fax: 732.302.3093
Toll free: 800.283.7866
Fax for Orders: 800.411.5870

Canada Office

Terumo Medical Corporation
C/O UPS - Supply Chain Solutions
1165 Creditstone Road
Vaughan, Ontario L4K 4N7
Phone: 800.668.5489
Fax: 905.760.1215

Ⓟ and TERUMO are trademarks owned by Terumo Corporation, Tokyo, Japan, and they are registered with the U.S. Patent & Trademark Office.