User's Guide

Cardiac Stress Treadmills





CARDIAC STRESS TREADMILLS USER'S GUIDE

70-00297-01 A



Copyright © 2008 Cardiac Science Corporation. All rights reserved.

Notice: Due to rapid changes in computer technology, the specifications provided in this manual are subject to change without notice.

Federal law restricts this device to sale by or on the order of a physician.

Trademarks

Quinton, Burdick, Q-Stress, and Quest are trademarks or registered trademarks of Cardiac Science Corporation.

All other product and company names are trademarks or registered trademarks of their respective companies.

For sales and technical support outside the USA, contact your local sales and technical support representative.

Liability Notice

Failure to follow the conditions set forth below shall limit, to the extent allowed by law, Cardiac Science Corporation responsibility for the safety, reliability, and performance of this equipment:

The operator manual must be read in full by each operator before the product is first used.

Assembly operations, extensions, readjustments, modifications, or repairs must be carried out only by personnel trained or authorized by Cardiac Science Corporation.

The electrical wiring within the treadmill's setting and the electrical installation of the treadmill must comply with the applicable local or provincial requirements.

The equipment must be used in accordance with the instructions for use.



Cardiac Science Corporation

ii

3303 Monte Villa Parkway Bothell, WA 98021, USA (800) 426 0337 (USA and Canada) (425) 402 2000 techsupport@cardiacscience.com www.cardiacscience.com



EC REP

MDSS GmbH Schiffgraben 41 D-30175 Hannover Germany

Contents

Chapter 1: Safety Requirements

	Safety Terms and Definitions	1-2
	Warnings and Cautions	1-3
	Safety Notes	1-6
	Symbol Definitions	1-7
	Cleaning	1-9
	Disinfection	1-9
	Daily	1-9
	Weekly	1-9
	Periodic	1-9
	EMC Declaration Tables	-10
	Electromagnetic Emissions	-10
	Electronic Immunity	-11
	Recommended Separation Distances 1	-13
Ch	apter 2: Introduction	2.2
	Overview	
	Treadmill Configuration	
	Treadmill Controls	
	Power	
	Operation	2-5
	Emergency Stop Button	2-5
	Indicators	2-5
	Accessories and Options	2-5
Ch	apter 3: Operating the Treadmill	
	Guidelines for Safe Operation	3-2
	Instructing the Patient	3-3
	Starting the Exercise	3-4
	Emergency Stop Button (option)	
	Ending the Exercise.	3-5

Chapter 4: Maintenance/Troubleshooting	
Daily Visual Inspection	4-2
Cleaning	4-3
Exterior	4-3
Interior	4-3
Disinfection	4-3
Adjustments	4-4
Belt Tension	4-4
Belt Tracking	
Rear Roller Guards	
Electrical Testing	4-8
Moving the treadmill	4-9
Storing the Treadmill	4-10
Troubleshooting	4-11
Appendix A: Specifications	
Cardiac Stress Treadmills	A-2
Power Requirements	A-3
Speed Vs. Weight Range	A-4
% Grade vs. Angle Relationship	A-5
Appendix B: Receiving and Installation	
Receiving	B-2
Installation Notice	B-2
Site Requirements	B-3
Installation	
Tools	
Procedure	B-4
Connecting to the Monitoring System	B-6
Testing the Treadmill	
Glossary	

1 Safety Requirements

Contents

•	Safety Terms and Definitions	1-2
•	Warnings and Cautions	1-3
•	Safety Notes	1-6
•	Symbol Definitions	1-7
•	Cleaning	1-9
•	EMC Declaration Tables	1-10

This user guide contains the user and patient safety requirements, operating instructions and maintenance requirements for the Cardiac Stress treadmills, models ST55, TM55, ST65, and TM65. The user guide is intended for use by trained clinicians working in a clinical setting. It is expected that the clinician will instruct the patient in the proper use of the treadmill and its accessories. Before using the treadmill, read the user guide carefully, noting the safety requirements in this chapter.

There are no user-serviceable parts in the treadmill. Any attempt by non-authorized personnel to service the equipment may void the warranty. Upon request, we will provide a technical document containing block-level theory of operation, troubleshooting, removal and replacement instructions (by module), maintenance, and other information that will assist appropriately-trained personnel to repair those parts of the equipment designated as repairable.



Caution. Use only approved equipment.

Accessory equipment connected to analog, digital, or power interfaces must be either equipment offered for sale by Cardiac Science Corporation or equipment that, when connected to such interfaces, maintains the safety and specified performance of the overall system and the individual devices. For example, such safety is maintained if the individual equipment and overall system complies with relevant safety requirements found in International Electrotechnical Commission (IEC) standards.

Safety Terms and Definitions

The symbols shown below identify potential hazard categories. The definition of each category is as follows:



WARNING!

This alert identifies hazards that may cause serious personal injury or death.



1-2

Caution

This alert identifies hazards that may cause minor personal injury, product damage, or property damage.

Warnings and Cautions

Before operating any treadmill, ensure the following warnings and cautions are understood and followed.



WARNING!

The treadmill must be on an appropriate, dedicated electrical circuit with a power rating that meets the electrical specification on the treadmill serial number label. Nothing else should be connected to the circuit.



WARNING!

As long as the treadmill is plugged into a powered outlet and the treadmill circuit breaker is set to ON, the treadmill is receiving power, even when the monitoring system is turned off. Do not place hands beneath the treadmill while it is plugged in.



Caution

Read this manual in full before operating the treadmill.



Caution

Before each use of this equipment, check the power receptacle for signs of damage. Do not operate the equipment if the integrity of these items is in question.



Caution

Be sure the connection for the optional Emergency Off switch is secure. Test the switch to verify proper operation. Do not operate the equipment if the integrity of these items is in question.



Caution

Keep the treadmill area clear. Maintain a minimum open space of 1.5 feet (0.5 meter) on each side and 6 feet (2 meters) at the rear.



Caution

The Cardiac Stress Treadmill should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Cardiac Stress Treadmill should be observed to verify normal operation in the configuration in which it will be used.



Caution

Do not operate this equipment in the presence of flammable anesthetic mixtures.



Caution

Regularly inspect cables and treadmill belts for wear or damage. Do not operate the equipment if the integrity of these items is in question.



Caution

Increased risk due to leakage current can result if this equipment is not grounded properly.



Caution

To avoid potential safety and electrical problems, use parts and accessories that meet specifications as noted in this user guide and the user guide for your stress monitor.



Caution

When connecting auxiliary equipment approved for use with the monitoring system, be certain the summation leakage current does not exceed local or provincial standards.



Caution

Failure to follow these guidelines can produce a serious or possibly fatal electrical shock hazard. Consult a qualified electrician as required.



Caution

Patients and clinicians should secure long hair and loose clothing before using and/or operating the treadmill.



Caution

Do not start the treadmill when someone is standing on the belt.



Caution

Keep speed and grade at the lowest settings when getting on and off the treadmill belt.



Caution

Never place chairs or other objects on treadmills.



Caution

Be aware of a moving treadmill belt.



Caution

Allow sufficient room for patients to maneuver around the system and to safely mount and dismount the treadmill.



1-4

Caution

Patients should wear shoes when using the treadmill.

Rapid Deceleration

If you have purchased the Rapid Deceleration option:



Caution

This treadmill is equipped with the rapid deceleration profile, it is specifically designed to come to a rapid stop. This feature requires that attendant(s) are available to support and assist the patient user. Attendants MUST be in a position to support and assist the patient when the belt stops.

Safety Notes

1-6

This equipment is classified Class I, Type B, ordinary equipment, not protected against fluid ingress. It is rated for continuous operation.

Note: Use of accessories or cables other than those specified, with the exception of accessories or cables sold by the manufacturer as replacement parts for internal components, may result in increased emissions or decreased immunity of the treadmill.

Note: Keep the area underneath and around the treadmill clear. Make sure cables are clear of the treadmill.

The following are safety guidelines for operating the treadmill.

Ensure the patient understands the proper treadmill mount and dismount procedure. Properly train new staff.

Symbol Definitions

Your treadmill may display one or more of the following symbols and warning labels for your protection. No single product displays all.

Caution	Attention: Consult accompanying documents		Earth ground (protective)
Off (power disconnected from mains)		*	Type B equipment— provides adequate protection against electric shock, particularly regarding allowable leakage current; reliability of the protective earth connection (when present)
	On (power connected to mains)	☀	Type BF equipment— contains an F-type isolated patient applied part providing a high degree of protection against electric shock
\sim	Alternating current	4 ½ F	Type BF equipment with defibrillation protection
4	High voltage		Type CF equipment— contains an F-type isolated patient applied part and provides a degree of protection against electric shock higher than that for type BF equipment regarding allowable leakage currents
<u></u>	Earth ground (functional)	4 (Type CF equipment with defibrillation protection
	Replace fuse only as marked	-	Fuse

	Mains power	♦	Equipotentiality
	Down		Up
\$	Faster	Ÿ ₩	Slower
Warning	Warning	Т	Timed fuse (slo-blo)
	USB Connector		Fragile: Do not lift in this location
Hz	Hertz	V	Volts
A	Amperes	VA	Volt Amperes



This is the CE marking of conformity indicating that the device having this symbol on its immediate label meets the applicable requirements of the European Medical Device Directive.

Cleaning



Caution

Turn off the treadmill and unplug the power cord from the wall outlet before cleaning.



Caution

Do not let liquid enter the interior of the treadmill. If it does, the equipment must be inspected and tested for safety by an approved technician before it can be used again.

Never wipe the deck beneath the belt, even when changing the belt. Wiping can damage the wax finish.

Do not use detergents or cleaning agents on any part of the deck.

Disinfection

If it is necessary to disinfect the treadmill, follow the procedures established by your institution.

Do not use liquids on the deck surface.

Daily

Keep the treadmill free of dust and debris. Clean the exterior and walking belt with a damp sponge; do not soak surfaces. Dry all surfaces thoroughly.

Weekly

Elevate the treadmill to maximum grade and vacuum the floor under it to prevent excess dust and dirt from interfering with operation.

Periodic

Depending upon the treadmill environment, dust and or lint can accumulate under the hood. Periodic internal cleaning by an authorized technician is recommended (see Service Manual).

EMC Declaration Tables

This section lists EMC Declaration Tables.

The Cardiac Stress Treadmill requires special precautions regarding EMC and must be installed and put into service according to the guidelines of the EMC declaration tables.

Portable and mobile RF communications equipment may affect the Cardiac Stress Treadmill and the recommended separation distances in the EMC declaration tables should be observed.

Electromagnetic Emissions

Cardiac Stress Treadmills are intended for use in the electromagnetic environment specified below. The customer or user of this equipment must ensure it is used in such an environment.

Table 1-1: Guidance and Manufacturer's Declaration - Electromagnetic Emissions

Emissions Test	Compliance	Electromagnetic Environment - Guidance	
RF emissions CISPR 11	Group 1	The Cardiac Stress Treadmills use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
RF emissions CISPR 11	Class A	The Cardiac Stress Treadmills are	
Harmonic emissions IEC 61000-3-2	Class A	suitable for use in all establishments other than domestic and those directly connected to the public low-voltage	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	power supply network that supplies buildings used for domestic purposes.	

Note: Tests verified with shielded input/output cables only.

Electronic Immunity

Cardiac Stress Treadmills are intended for use in the electromagnetic environment specified below. The customer or user of this equipment must ensure it is used in such an environment.

Table 1-2: Guidance and Manufacturer's Declaration - Electromagnetic Immunity

Immunity Test	IEC 60601 test level	Compliance Level	Electromagnetic Environment - Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+ 6kV contact + 8 kV air	+ 6kV contact + 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	+2 kV for power supply lines +1 kV for input/output lines	+2 kV for power supply lines +1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment
Surge IEC 61000-4-5	+ 1 kV differential mode +2 kV common mode	+ 1 kV differential mode +2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	$<5\% \ U_T$ $(>95\% \ dip \ in \ U_T \ for \ 0.5$ $cycle$ $<40\% \ U_T$ $(>60\% \ dip \ in \ U_T \ for \ 5 \ cycle$ $<70\% \ U_T$ $(>30\% \ dip \ in \ U_T \ for \ 25 \ cycle$ $<5\% \ U_T$ $(>95\% \ dip \ in \ U_T \ for \ 5 \ sec$	<5% U _T (>95% dip in U _T for 0.5 cycle <40% U _T (>60% dip in U _T for 5 cycle <70% U _T (>30% dip in U _T for 25 cycle <5% U _T (>95% dip in U _T for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the ST/TM55 or ST/TM65 requires continued operation during power mains interruptions, it is recommended that the ST/TM55 or ST/TM65 be powered from an uninterruptible power supply or a battery.

Table 1-2: Guidance and Manufacturer's Declaration - Electromagnetic Immunity

Immunity Test	IEC 60601	Compliance	Electromagnetic
	test level	Level	Environment - Guidance
Power frequency (50/ 60 Hz) Magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment

Note: U_T is the A.C. mains voltage prior to application of the test level.

Note: Tests verified with shielded input/output cables only.

Table 1-3: Guidance and Manufacturer's Declaration - Electromagnetic Immunity

Immunity Test	IEC 60601	Compliance	Electromagnetic Environment -
	test level	Level	Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 V	Portable and mobile RF communications equipment should be used no closer to any part of the ST/TM55 or ST/TM65, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: 150 kHz to 80 MHz: $d = 1.2\sqrt{P}$

Table 1-3: Guidance and Manufacturer's Declaration - Electromagnetic Immunity

Immunity Test	IEC 60601	Compliance	Electromagnetic Environment -
	test level	Level	Guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	80 MHz to 800 MHz: $d = 1.2\sqrt{P}$ 800 MHz to 2.5 GHz: $d = 2.3\sqrt{P}$ where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: $\left(((\bullet)) \right)$

Note: At 80 MHz and 800 MHz, the higher frequency range applies.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Note: Tests were verified with shielded input/output cables only.

Recommended Separation Distances

This section describes recommended separation distances between portable and mobile RF communications equipment and the Cardiac Stress Treadmill.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ST/TM55 or ST/TM65 is used exceeds the applicable RF compliance level above, then the ST/TM55 or ST/TM65 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the ST/TM55 or ST/TM65.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

The Cardiac Stress Treadmills are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Cardiac Stress Treadmills can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Cardiac Stress Treadmills as recommended below, according to the maximum output power of the communications equipment.

Table 1-4: Recommended Separation Distances

	Separation distance according to frequency of transmitter (m)			
Rated maximum output power of transmitter (W)	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$	
0.01	0.12	0.12	0.23	
0.1	0.38	0.38	0.73	
1	1.2	1.2	2.3	
10	3.8	3.8	7.3	
100	12	12	23	

Note: For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note: At 80 MHz and 800 MHz, the higher frequency range applies.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Note: Tests were verified with shielded input/output cables only.

2 Introduction

Contents

Overview	2-2
Treadmill Configuration	2-3
Treadmill Controls	2-4
Operation	2-5
Emergency Stop Button	2-5
Indicators	2-5
Accessories and Options	2-5

This chapter describes the Cardiac Science stress treadmills.

Overview

The Cardiac Stress Treadmills are heavy-duty medical treadmills designed specifically for stress test applications. Four models are available and can be used with the following cardiac stress testing systems:

Treadmill Model	Where Used
TM55	Quinton Q-Stress Cardiac Stress System
TM65	Burdick Quest Exercise Stress System Burdick HeartStride Cardiac Stress System
ST55	Quinton Q-4500 Stress Test Monitor System
ST65	Quinton Q710 Stress Monitor & Resting ECG System

All models feature variable speed and grade and a 20-inch-wide walking surface.

Treadmill Configuration

The following illustration shows a typical treadmill configuration.

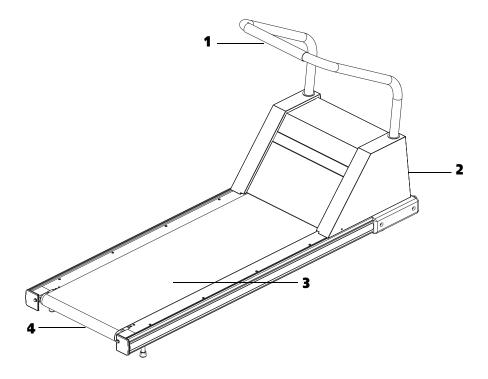


Figure 2-1: Example of a Treadmill

- 1 Handle bar
- 2 Front of hood
- 3 Walking belt
- 4 Rear of treadmill

Treadmill Controls

Power

The circuit breaker located on the front of the treadmill hood, controls the power to the treadmill (see item 1 in Figure 2-2). The circuit breaker must be set to ON for the treadmill to run. As long as the treadmill is plugged into a powered socket and the circuit breaker is set to ON, the treadmill is receiving power, even when the monitoring system power is turned off.



2-4

WARNING! Danger of electric shock.

Turn off the treadmill circuit breaker before connecting or disconnecting the treadmill from the power outlet.

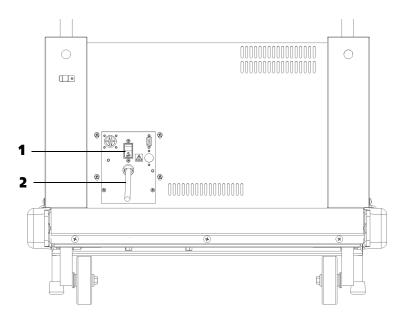


Figure 2-2: Example of Hood configuration Plate for Treadmill

- 1 Circuit breaker
- 2 Power cord

Operation

All patient data displays on the monitoring system screen. Use the operator manual supplied with the monitoring system for instructions on how to control the treadmill.

Emergency Stop Button

If the treadmill is equipped with an optional emergency stop button, it is located on the front handle bar. Pressing the emergency stop button causes the treadmill to stop sending power to the belt drive motor and the grade motor.

Note: The emergency stop button is intended only for emergency use. You may have to restart the stress test if the emergency stop button is pressed.



WARNING! Connect Emergency Stop Button.

If your treadmill is equipped with the Emergency Stop Button feature, the treadmill will not run unless the emergency button cable is plugged into the connector on the front of the hood. Once installed, this cable should not be removed except by an authorized technician.

Indicators

There are no visual indicators on the treadmill. All patient responses and warnings for low power, motor overload, and other responses appear on the stress test monitoring system screen.

Accessories and Options

The following accessories and options are available for order:

Part No.	Description	
036734-002	Hand grip, Blue Vinyl	
013802-003	Hand grip Blue Vinyl, Emergency Stop	
032758-001	Side Handrail Kit, Short, TM55/ST55	
032758-002	Side Handrail Kit, Short, TM65/ST65	
032759-001	Side Handrail Kit, Long TM55/ST55	
032759-002	Side Handrail Kit, Long TM65/ST65	
019331-001	Low Handrail Kit	
037082-xxx	Rapid Deceleration Kit (specify language)	
037080-xxx	Emergency Stop Button Kit (specify ST or TM version)	

3 Operating the Treadmill

The stress test monitor controls all treadmill operation. Please read the following treadmill-specific information, then refer to the operator manual supplied with your stress test monitor for operational procedures.

Guidelines for Safe Operation

Keep the treadmill area clear. Maintain a minimum open space of 1.5 feet (0.5 meter) on each side and 6 feet (2 meters) at the rear.

Before using the treadmill, verify that the correct cable connects the treadmill to the monitoring system.

Do not start the treadmill when someone is standing on the belt.

Never place chairs or other objects on treadmills.

Do not leave a patient unattended on the treadmill.

Patients and clinicians should secure long hair and loose clothing before using and operating the treadmill.

Keep speed and grade at minimum settings when patients are getting on and off the treadmill.

Use the optional emergency stop button only for emergency.

Instructing the Patient

- 1. Before each test, describe the complete procedure to the patient:
 - Remind the patient to not use the handrail for support during the exercise unless absolutely necessary. Explain that gripping the handrail can affect the accuracy of the test measurements.
 - Describe how you will change the belt speed and treadmill grade during the test.
- **2.** Demonstrate how to get on and off the moving belt:
 - **a.** Stand next to the treadmill belt, facing the front of the treadmill.
 - **b.** Place both hands on the handrail.
 - **c.** Step onto the side of the deck or straddle the belt.
 - **d.** (Optional) Place one foot forward on the belt and let it move backwards, as though walking with one foot. Repeat until comfortable.
 - **e.** Carefully step onto the belt and begin walking.
 - **f.** When able to walk unassisted, let go of the handrail.
 - **g.** While walking on the treadmill:
 - Face forward; avoid looking down.
 - Maintain speed by keeping a constant distance from the handrail.

Starting the Exercise

To start the exercise:

1. Start the treadmill walking belt while the patient stands on the side of the deck or straddles the walking belt.



WARNING! Risk of injury.

Do not start the belt while someone is on the treadmill belt.

- **2.** Set the speed and grade to the minimum settings.
- **3.** Have the patient step onto the treadmill as previously instructed.
- **4.** Before changing speed or grade, inform the patient of the change.

Emergency Stop Button (option)

If the treadmill is equipped with an optional emergency stop button, pressing this button causes the treadmill to stop sending power to the belt drive motor and the grade motor. The belt stops moving and the grade cannot change. The button locks into place and the treadmill does not function until the button is unlocked. To unlock the button, turn it counter-clockwise.

Note: The emergency stop button is intended only for emergency use. You may have to restart the stress test if the emergency stop button is pressed.

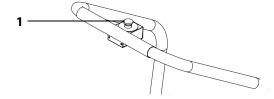


Figure 3-1: Example of Emergency Stop Button

1 Emergency Stop Button

Ending the Exercise

- 1. Tell the patient when the test is about to end.
- **2.** Decrease the speed and grade gradually to minimum settings.
- **3.** Before stopping the walking belt, tell the patient to grasp the handrail for balance.
- **4.** Stop the walking belt.
- **5.** Tell the patient to hold the handrail and step off to the side.



WARNING! Unsteady patient.

The patient may be slightly unsteady for several seconds after stepping off the walking belt.

4 Maintenance/Troubleshooting

Contents

Daily Visual Inspection	4-2
Adjustments	4-4
Belt Tension	4-4
Belt Tracking	4-6
Rear Roller Guards	4-7
Electrical Testing	4-8
Moving the treadmill	4-9
Storing the Treadmill	4-10
Troubleshooting	4-11

The treadmill should be visually inspected, cleaned, and adjusted regularly and as needed.

Daily Visual Inspection

Check the following items daily:

Inspect the treadmill power cord and walking belt for wear.

Check the position of the walking belt; be sure it is not rubbing against the frame. It should be centered on the deck within 0.25 inch of the sides; adjust if necessary.

Check the handle bar and optional attachments to be sure they are fastened securely.

Remove potential hazards from the treadmill area.

Cleaning

Exterior



WARNING! Electric shock hazard.

Turn off the treadmill and unplug the power cord from the wall outlet before cleaning.

Daily

Keep the treadmill free of dust and debris. Clean the exterior and walking belt with a damp sponge; do not soak surfaces. Dry all surfaces thoroughly.



Caution. Damage to exterior.

Never wipe the deck beneath the belt, even when changing the belt. Wiping can damage the wax finish.

Do not use detergents or cleaning agents on any part of the deck.

Weekly



Caution

Do not let liquid enter the interior of the treadmill. If it does, the equipment must be inspected and tested for safety by an approved technician before it can be used again.

Elevate the treadmill to maximum grade and vacuum the floor under it to prevent excess dust and dirt from interfering with operation.

Interior

Depending upon the treadmill environment, dust and or lint can accumulate under the hood. Periodic internal cleaning by an authorized technician is recommended (see Service Manual).

Disinfection

If it is necessary to disinfect the treadmill, follow the procedures established by your institution.

Note: Do not use liquids on the deck surface.

Adjustments

Tools Required:

Hex wrenches

Phillips screwdriver

Torque wrench



WARNING! Catching of clothing.

Secure long hair and loose clothing before using the treadmill or working near the treadmill walking surface or pulleys.

Belt Tension

Adjust the belt tension whenever the belt slips or moves unsteadily during operation:

- 1. Use the controls on the monitoring system to start the treadmill at minimum speed and grade, then increase the speed to 2.5 mph.
- **2.** Hold onto the handrail for balance and walk heavily on the treadmill by marching flat-footed.
 - **a.** If the belt hesitates or lags noticeably, tighten the belt as in steps 3 and 4.

Note: Do not use all your weight to resist the belt movement. Too much resistance applied too long (more than two seconds) will shut down the system. If this should occur, recycle power to resume normal operation.

- **b.** Increase the speed to 4.5 mph (7.1 km/h) and jog on the treadmill. If the belt hesitates or lags noticeably, tighten the belt as in steps 3 and 4.
- **c.** Increase the speed to 6.5 mph (10.3 km/h) and run on the treadmill. If the belt hesitates or lags noticeably, tighten the belt as in steps 3 and 4.
- **3.** Locate the two adjustment bolts at the rear of the treadmill (see Figure 4-1).

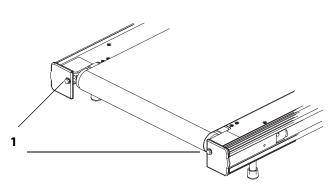


Figure 4-1: Adjustment bolts

1 Adjustment bolts

4. Turn both adjustment bolts clockwise 1/4 turn. Test the belt tension as in Step 2. Repeat if necessary until the belt runs smoothly without slipping. If more than three adjustments are necessary, call an authorized technician.



Caution. Belt damage.

Do not overtighten the walking belt. Overtightening can damage the belt and rollers. Do not torque adjustment screws beyond 80 in/lb (9 Nm) maximum.

- **5.** Stop the treadmill.
- **6.** Check the position of the rear roller guards and adjust if necessary (see Figure 4-2).

Belt Tracking

Perform this procedure whenever the belt moves to one side or the other.



WARNING! Falling hazard.

Stay off the belt when adjusting the tracking. Do not touch or grab the walking belt while it is moving.

To adjust the belt tracking:

- 1. Use the controls on the monitoring system to start the treadmill at minimum speed and grade.
- **2.** Increase speed to 6 mph (9.5 km/h) and make the following adjustment (See Figure 4-1 for the location of the bolts):
 - If the belt moves to the right, rotate the right tension bolt clockwise 1/4 turn.
 - If the belt moves to the left, rotate the left tension bolt clockwise 1/4 turn.
- **3.** After making an initial adjustment, run the treadmill for several minutes and observe how the belt tracks; adjustments to belt tracking take several minutes to become apparent. If the belt continues to move off center, adjust accordingly until it is properly centered.

If more than three adjustments are necessary, call an authorized technician.



4-6

Caution. Damage to belt and rollers.

Do not overtighten the walking belt. Overtightening can damage the belt and rollers. Do not torque adjustment screws beyond 80 in/lb. (9N/m) maximum.

- **4.** Stop the treadmill.
- **5.** Check the position of the rear roller guards and adjust if necessary (see Figure 4-2).

Rear Roller Guards

Check the position of the rear roller guards each time you readjust the belt tracking or the belt tension. The roller guards are located at the rear of the treadmill between the rear roller and the deck (see Figure 4-2).

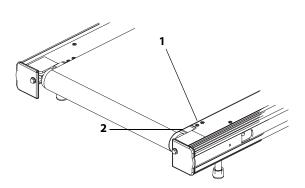


Figure 4-2: Roller Guard

- 1 Adjustment Screws
- 2 Roller Guard

Adjust the guards when the gap between the roller and the guard exceeds 3/8 (0.375) inch (9.5 mm).

- 1. Disconnect the treadmill power cord from the power source.
- **2.** Loosen, but do not remove, the two screws attaching the rear roller guard to the deck (hold the wing nuts on the underside with your finger).
- **3.** Slide the rear roller guard towards the rear roller until the gap between the two is approximately 0.1 inch (2.5 mm or the thickness of two quarters).
- **4.** Tighten the mounting screws loosened in Step 2.
- **5.** Connect the power cord.
- **6.** Use the controls on the monitoring system to start the treadmill belt. If there is a scraping noise by the rear roller guards, redo steps 1 through 6.

TM 55/65 Treadmill 4.0 70-00297-01 A **4-7**

Electrical Testing

4-8

Electrical testing is to be done by the facility's biomedical department as required. Check the leakage current of the treadmill periodically—at least every nine months—to be sure it does not exceed local or provincial standards.

Moving the treadmill



WARNING!

Moving the treadmill requires two people.

To move the treadmill:

- 1. Set the treadmill to 3 to 5% grade.
- **2.** To avoid electric shock, turn off the treadmill circuit breaker and the monitoring system, then remove the power cord from the power source.
- **3.** Disconnect all cables that are attached to the treadmill or any other equipment which is attached to the treadmill.



WARNING! Pinch hazard.

As long as the treadmill is plugged into a powered outlet and the treadmill circuit breaker is set to ON, the treadmill is receiving power, even when the monitoring system is turned off. Do not place hands beneath the treadmill while it is plugged in.

4. Together, lift the rear of the treadmill, then roll it to the new site using the wheels on the front of the treadmill.

Note: You may stand the treadmill on end to maneuver around corners.

TM 55/65 Treadmill 4.0 70-00297-01 A **4-9**

Storing the Treadmill

When storing for prolonged periods, cover the treadmill with a dust cover. Do not store in damp areas.



4-10

WARNING! Falling hazard.

Do not store the treadmill on its end as it could fall on someone.

Before re-using the treadmill after moving or storage, check the power cord and all attachments to be sure they are undamaged and securely connected, then test the system for proper operation.

Troubleshooting

If the walking belt does not run, check the monitoring system for an error code. Record the code, if present, then call an authorized technician. If no error message is present, recycle the power by powering the monitoring system off and on again. If this does not eliminate the fault, check the interface cables for proper connection.

If problems persist, refer to the Troubleshooting Guide that follows. Problems beyond the scope of this table can require service assistance to isolate and correct. Contact an authorized technician or call 800-426-0337.



WARNING! Electric shock hazard.

Do not remove the treadmill hood: Dangerous voltages are present. There are no operator-serviceable components.



Caution. Authorized service only.

Servicing should be done only by an authorized technician who should consult the service manual before attempting any in-depth troubleshooting.

TM 55/65 Treadmill 4.0 70-00297-01 A 4-11

Use the following table to diagnose problems with the treadmill.

Table 4-1: Troubleshooting Guide

Problem	Possible Cause	Remedy		
	Treadmill or monitoring system power cord is not plugged in.	Plug in power cords.		
	Interface cable disconnected or damaged	Plug in and secure interface cable. Replace if damaged.		
	Monitoring system power off.	Turn on power. Check AC power, if necessary.		
	Circuit breaker on treadmill hood reads OFF.	Set the circuit breaker switch on the treadmill to ON.		
Treadmill will not run	 Start key not selected. Not in proper menu. 	Press proper key on monitoring system control panel. Consult stress test system user guide.		
	No power at outlet	Check building circuit breaker.		
	Voltage below nominal	Provide dedicated line for treadmill.		
	Emergency Stop button has been unplugged.	Plug into connector on front of hood.		
	Emergency Stop button activated.	Turn emergency stop button counter-clockwise to deactivate.		
	Internal problem.	Service required. Contact an authorized technician.		
Walking belt too far left or right	Improper belt tracking.	Adjust tracking (see "Belt Tracking" on page -6). If problem persists, contact an authorized technician.		
Walking belt slips, but front roller turns.	Improper belt tension	Adjust belt tension (see "Belt Tension" on page -4). If problem persists, contact an authorized technician.		
Walking belt hesitates; adjusting walking belt tension is ineffective	Internal drive belt slipping.	Service required. Contact an authorized technician.		
Treadmill will	Excess weight on treadmill.	See Appendix B for maximum load.		
not change grade.	Internal problem.	Service required. Contact an authorized technician.		

Table 4-1: Troubleshooting Guide (continued)

Problem	Possible Cause	Remedy	
Circuit breaker trips during normal operation.	Power fault.	Service required. Contact an authorized technician.	
Error messages on stress test monitor	Refer to monitoring system user guide.	Refer to monitoring system user guide.	
	Internal problem	Service required. Contact an authorized technician.	

TM 55/65 Treadmill 4.0 70-00297-01 A **4-13**

4-14

A Specifications

Contents

Cardiac Stress Treadmills	A-2
Power Requirements	A-3
Speed Vs. Weight Range	A-4
% Grade vs. Angle Relationship	A-5

This appendix contains specifications for Cardiac Science stress treadmills.

TM 55/65 Treadmill 4.0 70-00297-01 A **A-1**

Cardiac Stress Treadmills

A-2

Cardiac Stress Treadmills have the following specifications:

Performance					
Maximum Rated Load	500 lb (227.3 kg) (with restrictions see Speed Vs. Weight graph next page)				
Belt Speed Range ± 0.2 mph (continuously adjustable)	0.8 to 9.6 mph (1.3 to 15.4 km/h)				
Grade Range ± 0.5%	0 to 25%				
Physical					
Weight	ST/TM55: 352 lb (160 kg) ST/TM65: 375 lb (170 kg)				
Nominal Walking Surface	ST/TM55: 20 x 55 in. (51 x 140 cm) ST/TM65: 20 x 65 in. (51 x 165 cm)				
Treadmill Area	ST/TM55: 29.9 x 80.2 in. (76 x 204 cm) ST/TM65: 29.9 x 90.2 in. (76 x 229 cm)				
Walking Surface Height	7.0 in. (18 cm) from floor				
Handrail Height	47.2 in. (120 cm) from floor				
Environmental					
Temperature	Operating: 50 to 104 °F (10 to 40 °C) Storage: -40 to 158 °F (-40 to 70 °C)				
Humidity (non- condensing)	Operating: 15 to 95% relative Storage: 5 to 95% relative				

Power Requirements

Listed below are the power requirements for your treadmill

Nominal Voltage Range (min - max) / Hertz*	Current Draw (Amps)	Min. Branch Circuit Amps		
100-120 V, 50/60 Hz	20**	20		
200-240 V, 50/60 Hz	10**	10		


^{*} The nominal voltage range is listed on the serial number name plate, which can be found on the hood under the circuit breaker switch.

TM 55/65 Treadmill 4.0 70-00297-01 A **A-3**

^{**} Full-load current is computed as described in section 430-24 of the National Electrical Code.

Speed Vs. Weight Range

The following graph illustrates the relationship between speed and weight for the treadmills

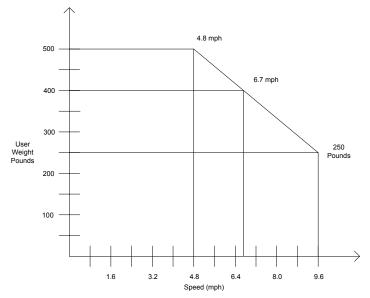

Performance Envelope 115V or 230V Operation

Figure A-1: Speed vs. Weight graph

A-4

% Grade vs. Angle Relationship

The following graph and table indicate the relationship between grade and angle.

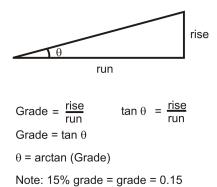


Figure A-2: %Grade vs. Angle Graph

Grade (%)	Angle (°)								
0.0	0.00	5.0	2.86	10.0	5.71	15.0	8.53	20.0	11.31
0.5	0.29	5.5	3.15	10.5	5.99	15.5	8.81	20.5	11.59
1.0	0.57	6.0	3.43	11.0	6.28	16.0	9.09	21.0	11.86
1.5	0.86	6.5	3.72	11.5	6.56	16.5	9.37	21.5	12.13
2.0	1.15	7.0	4.00	12.0	6.84	17.0	9.65	22.0	12.41
2.5	1.43	7.5	4.29	12.5	7.13	17.5	9.93	22.5	12.68
3.0	1.72	8.0	4.57	13.0	7.41	18.0	10.20	23.0	12.95
3.5	2.00	8.5	4.86	13.5	7.69	18.5	10.48	23.5	13.22
4.0	2.29	9.0	5.14	14.0	7.97	19.0	10.76	24.0	13.50
4.5	2.58	9.5	5.43	14.5	8.25	19.5	11.03	24.5	13.77
5.0	2.86	10.0	5.71	15.0	8.53	20.0	11.31	25.0	14.04

TM 55/65 Treadmill 4.0 70-00297-01 A **A-5**

70-00297-01 A TM 55/65 Treadmill 4.0

A-6

B Receiving and Installation

Contents

Receiving	B-2
Installation Notice	B-2
Site Requirements	B-3
Installation	B-4
Testing the Treadmill	B-8

This section describes the receiving and installation procedures.

TM 55/65 Treadmill 4.0 70-00297-01 A **B-1**

Receiving

When the carrier delivers your order, verify that the number of items received equals the number listed on the freight bill or express receipt.

Inspect the containers for damage. Itemize discrepancies and damage on the waybill and have the agent sign it. Failure to describe external evidence of loss adequately may result in the carrier refusing to honor your claim. Do not discard the packing materials until you have verified physical condition and proper operation.

Installation Notice

The treadmill and controller must be installed correctly before being used. We recommend that you contact your treadmill dealer or representative when your equipment arrives. The representative will help unpack, install, and demonstrate it, to ensure that:

Equipment is free from shipping damage.

hazard. Consult a qualified electrician as required.

The treadmill is connected correctly to the appropriate AC power source.

Installation and operation are in accordance with standards stated in this manual.

Incorrect installation by unauthorized personnel can lead to equipment damage and may void the warranty.



WARNING! Dedicated circuit.

The treadmill must be on a dedicated branch circuit. No other device should be connected to that circuit.

Excessive risk current (leakage) can result if the equipment is not properly grounded. Failure to follow these guidelines will produce a serious or possibly fatal electrical shock

Site Requirements

The treadmill requires a dedicated AC power line (refer to "Power Requirements" on page -3). To ensure electrical safety, the treadmill is equipped with a three-wire power cord and three-pronged plug. To maintain ground reliability, the plug must be connected to an equivalent receptacle.



Caution. Operate in conditioned environment.

The treadmill is designed to operate in a typical clinical environment with adequate heat dissipation (1850 Watts maximum).

Place the treadmill on a flat surface, free of moisture and debris. Maintain a minimum clearance of 1.5 feet (0.5 m) on each side and 6 feet at the rear.

If the preamp for the Q-Stress system is to be attached to the handrail, maintain a clearance of 1.5 feet (0.5 m) at the front of the treadmill.

TM 55/65 Treadmill 4.0 70-00297-01 A **B-3**

Installation

If you choose to install your treadmill without the assistance of your representative, follow the procedure given below:

Tools

Hex wrench (supplied in kit)

Procedure

To install the treadmill:

- **1.** Ensure that the handrail gaskets have been placed in their proper location (see Figure B-1).
- **2.** Insert the front handrail into the holes on each side of the hood.
- **3.** Use the two hex-head socket screws recessed within the front of the hood to secure the handrail:
 - **a.** Using the hex wrench supplied in the ship kit, turn the socket screws clockwise.
 - **b.** Cover the screws with the caps supplied in the ship kit.

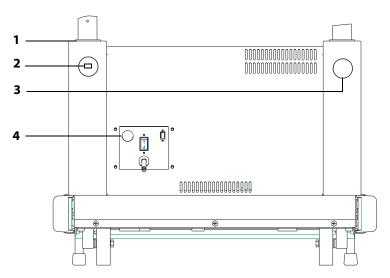


Figure B-1: Example of Treadmill front

- 1 Handle bar gasket
- 2 Socket screw
- 3 Cap in place
- 4 Emergency Stop switch Connector (optional)

4. Plug the cable for the optional Emergency Stop Button, if present, into the connector on the front of the hood (see Figure B-1).

Note: If the optional Emergency Stop Button is present, it must remain connected and secured at all times. It should be disconnected only by an authorized technician.

- **5.** (High Voltage units only) Plug the female end of the power cord into the connector on the front of the treadmill hood. Be sure the plug is connected securely, then tighten the clamp using the hex wrench provided.
- **6.** If you want to attach the Q-Stress preamp to the handrail, follow the installation instructions included with the preamp.

TM 55/65 Treadmill 4.0 70-00297-01 A **B-5**

Connecting to the Monitoring System

To connect the treadmill to the monitoring system:

- **1.** Be sure the treadmill and the monitoring system are disconnected from the power source.
- **2.** Inspect the monitoring system interface cable connector, located on the front of the hood (see Figure B-2). If it appears damaged, contact an authorized technician before continuing this installation.
 - If the connector is undamaged, plug the male end of the monitoring system interface cable into the connector. Tighten the connector thumb screws until finger tight.
- **3.** Plug the female end into the treadmill connector on the back panel of the monitoring system.
- **4.** Verify that the outlet voltage matches the voltage on the nameplate located on the front of the treadmill hood.
- **5.** If the voltage is adequate, plug in the power cord. Do not use this outlet if the voltage is not adequate.

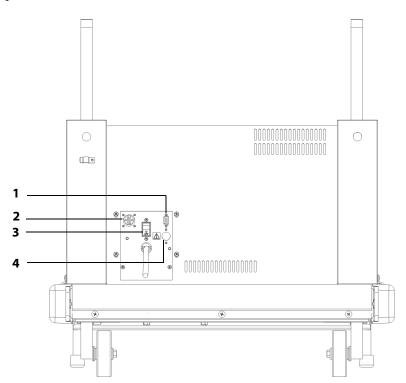


Figure B-2: Example of Monitoring System Connections/

- Monitoring System Interface Cable Connector
- 2 Emergency Stop Switch Connector
- 3 Circuit Breaker Switch
- 4 USB Monitoring System Interface Cable Connector

- **6.** Check the circuit breaker switch on the treadmill hood to be sure it is set to ON.
- **7.** Test the treadmill as described below by using the controls on the monitoring system.

TM 55/65 Treadmill 4.0 70-00297-01 A **B-7**

Testing the Treadmill



WARNING! Falling hazard.

DO NOT permit anyone to stand on the treadmill when it is started.

To test the treadmill:

- **1.** Connect the treadmill power cord to the correct power outlet. Turn on treadmill power.
- **2.** Start the walking belt from the monitoring system: the treadmill starts running immediately at low speed.
- **3.** Set the treadmill to minimum speed and zero grade and confirm that the treadmill meets these values.
- 4. Set the treadmill to maximum speed and confirm that the treadmill meets that value.
- 5. Set the treadmill to maximum grade and confirm that the treadmill meets that value.
- **6.** Let the treadmill run for at least 15 minutes.
 - **a.** Watch the walking belt carefully to ensure that it does not drift left or right. Adjust if necessary using the adjustment procedures in *Adjustments* on page 4-4.
 - **b.** Listen for unusual noises, such as squeals or squeaks. The treadmill should run quietly.
 - **c.** Reduce speed and grade until the treadmill belt is moving at a reasonable speed for walking.
 - **d.** Walk on the moving belt and verify proper operation at representative speeds and grades. If the walking belt slips, but the front roller turns, adjust the walking belt tension.
- **7.** When the test is finished:
 - **a.** Set the treadmill to zero grade.
 - **b.** Stop the walking belt.
 - **c.** Turn off the monitoring system.

If your treadmill is not running smoothly, contact an authorized technician before using it.

Glossary

Controller Press-button panel that lets the user control the treadmill and view exercise progress.

Deck Treadmill surface under walking belt.

Grade Incline, or slope, of the treadmill deck measured in percent grade.

Monitoring System A device that receives ECG signals from a patient and displays and/or records them for review. Used with cardiac stress treadmills for exercise testing.

Protocol The specific treadmill workload sequence used for a particular stress test.

Roller Guard Metal plate at the rear of the treadmill deck, designed to prevent people from placing fingers between the walking belt and rear roller.

Stress Test A procedure used to test cardiovascular and pulmonary response. Consists of a patient performing controlled exercise while heart and/or lung activity is measured with precision instrumentation. Also called a treadmill exercise test or exercise tolerance test (ETT).

Track The path that the treadmill belt follows.

Walking Belt Walking surface of the treadmill—moves according to the speed set on the controller.

TM 55/65 Treadmill 4.0 70-00297-01 A C-1

 $\textbf{Cardiac Science Corporation • 3303 Monte Villa Parkway, Bothell, WA 98021 USA • 425.402.2000 • US toll-free 800.426.0337 • Fax: 425.402.2001 • info@cardiacscience.com$

Orders and Customer Care (US and International) • 425.402.2000 • US toll-free 800.426.0337 • Fax: 425.402.2001 • care@cardiacscience.com

 $\textbf{Technical Support} \bullet (US) \ Fax: \ 425.402.2022 \bullet technical support@cardiacscience.com \bullet \ http://websupport.cardiacscience.com/webchat/ \bullet (International) \ international support@cardiacscience.com$

Cardiac Science International A/S • Kirke Vaerloesevej 14, DK-3500 Vaerloese, Denmark • +45.4438.0500 • Fax: +45.4438.0501 • international@cardiacscience.com

United Kingdom • The Manse, 39 Northenden Road, Sale, Manchester, M33 2DH, United Kingdom • +44.161.926.0000 • uk@cardiacscience.com
France • Parc de la Duranne, 565, Rue René Descartes, F-13857 Aix-en-Provence Cedex 3, France • +33.4.88.19.92.92 • france@cardiacscience.com
Central Europe (D, A, CH) • Oskar-Schindler-Strasse 3, D-50769 Köln, Germany • +49.0.221.33734.300 • centraleurope@cardiacscience.com
China • 6/F South Building, 829, Yi Shan Road, Shanghai 200233, China • +86.21.6495.9121 • china@cardiacscience.com

Cardiac Science, the Shielded Heart logo, Quinton, Burdick, Q-Stress, and HeartStride are trademarks of Cardiac Science Corporation. Copyright © 2008 Cardiac Science Corporation. All Rights Reserved.



70-00297-01 A

