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English

Coaxial Bipolar Adapter REF:133001 & 133002

Description:
Coaxial Bipolar electrosurgical generator adapter
Non-sterile / Re-usable

Intended Use:
The CONMED® Coaxial Bipolar Adapter converts the bipolar output receptacle of a specific CONMED® electrosurgical generator to a coaxial bipolar output receptor for use with compatible bipolar accessories.
Maximum voltage output: 600Vpk

Contraindications:
These devices should never be used when:

- There is visible evidence of damage to the exterior of the device, such as cracks, cuts, punctures, nicks, abrasions, discoloration or connector damage.
- In the presence of flammable gases, flammable prep solutions or drapes, oxidizing gases, such as nitrous oxide, or in oxygen-enriched environments.

Warnings:

- Electrosurgery is potentially hazardous for patients with pacemakers, AICDs, neurostimulators and other active implants. High frequency energy may damage or adversely affect active implants. Ventricular fibrillation may occur.
- Do not permit cables connected to associated electrosurgical accessories to be parallel to or in close proximity to the leads of other electrical devices.
- MODIFICATION OF THE DEVICE BY THE USER:** CONMED® warns explicitly against modifying the device. Any modification voids warranty and exempts CONMED® from all liability.
- DO NOT STERILIZE.** The ability to effectively sterilize this device has not been established and subsequent sterilization processing may adversely affect the performance and safety of the device.

Precautions:

- Refer to the applicable CONMED® electrosurgical generator operator's manual instructions prior to use.
- Do not immerse or saturate adapter in fluid.
- Do not bend or damage the connectors.
- The electrosurgical generators should be used only by surgeons/ healthcare practitioners experienced in electrosurgical techniques and safety.

Inspection:
These devices should be inspected before use. Visually examine the devices for obvious physical damage and replace if any defect is found:

- Broken, cracked, nicked, punctured, abraded, bent, distorted, discolored insulation, connector or tip.

Instructions for Use:

- Before turning on generator power, insert the 3-prong connector into the BIPOLAR receptacle of the CONMED® electrosurgical generator.
- Plug appropriate bipolar accessories into adapter receptacle.
- Ensure all connections are tight.
- Turn generator power on.
- Turn off generator power before disconnecting accessories.
- Clean off outer plastic housing with mild detergent solution and soft damp cloth. Do not submerge in liquid or allow connectors to get wet. Allow adapter to air dry.

USA Federal law restricts this device to sale by or on the order of a physician.

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© CONMED Corporation，美国印制。

For International orders or inquiries, please contact
CONMED International Sales
(315) 797-8375 • FAX (315) 735-6235

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