

# lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China


glider walker

drive  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

## instructions for handle adjustments

1. The Glider Walker can be adjusted to fit each individual's requirements. Both the height and depth of the handles can be adjusted to adjust the height of the handle (A). Loosen the adjustment knobs (B) on both top and bottom brace, slide handle bar to the desired height and re-tighten adjustment knobs.
2. To adjust the depth of the handle (A), loosen the adjustment knobs (B) on both the top and bottom brace, slide the handle forward to the desired position and re-tighten the adjustment knobs.

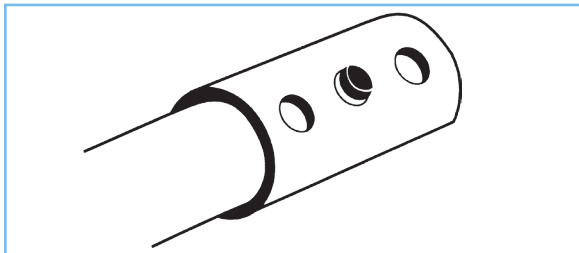

## adjusting leg height

Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration above. To be sure legs are locked into place by pushing down on or by pulling legs.

Item # 10219C-2  
Shown



Weight Capacity: 175 lbs.