
MICHAEL GRAVES  
solutions™

# Therma Moist Heating Pad

## USER MANUAL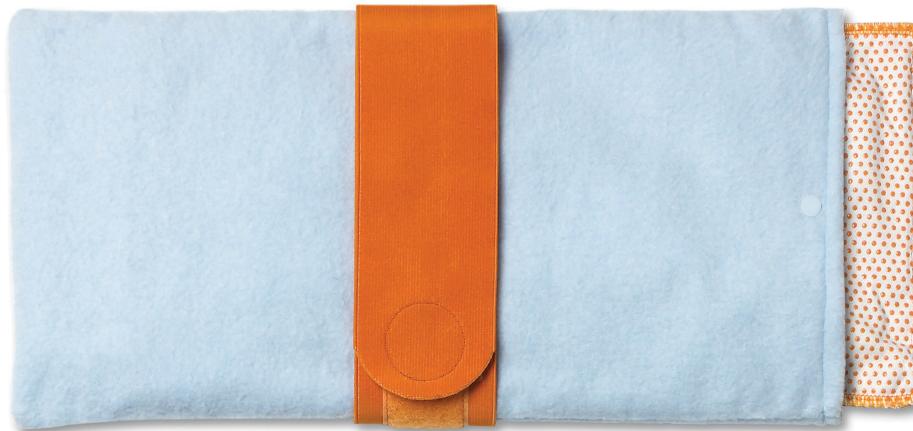

**drive**  
MEDICAL DESIGN & MANUFACTURING

# Therma Moist Heating Pad

## **MOIST HEATING PAD THERAPY**

The application of moist heat is a commonly prescribed therapy in medical practice today, moist heat is specially usefully in treating pain. And the pain is usually related to muscle strain and tension which results in muscle spasm.

## **HOW TO USE:**

1. Place the moist heating pad into the pad cover (provided).  
Be sure the pad is lying flat inside the cover to ensure uniform heating. Be sure cover is clean prior to use.
2. Insert the plug into any 110 volt electric power outlet. Switch the TEMP knob to the "O" position.
3. Press the ON/OFF switch. The green light indicators power on. Adjust to the desired time and temperature for treatment.  
Time and Temp. selection  
a.Time: 15/30/45/60 min.  
b.Temp: 110/130/150/165 Fahrenheit  
45/55/65/75 Celsius
4. Place the moist heating pad flat on the area of the body to be treated.
5. The recommended treatment time is 30 minutes.
6. After the treatment press the on/off switch to turn off the power.  
Pull the plug from the electrical outlet.
7. The Therma Moist Heat Pad will automatically shut off after the timed treatment.
8. Warranty: 1 year

# Therma Moist Heating Pad

## **FLANNEL COVER WASHING INSTRUCTIONS**

Remove cover from heating pad. Machine or hand wash. Use mild detergent in cool water. Hang to dry or machine dry on cool gentle cycle.

## **IMPORTANT SAFETY INSTRUCTIONS**

1. Read all instructions carefully.
2. Do not use while sleeping.
3. Do not use on an infant.
4. This pad is not to be used on a unconscious person or a person with poor blood circulation unless carefully attended.
5. Check skin under pad frequently.
6. Do not use on sensitive skin.
7. Never use pad without cover in place.
8. Do not use pins or other metallic means to fasten this pad in place.
9. Do not sit on or crush pad.
10. Never pull the power cord of the pad and do not use the cord as a handle.
11. Carefully examine the inner cover before using it.
12. Discard the pad if the inner cover shows any signs of wear or deterioration.
13. Skin burns can occur regardless of control setting. Check skin under pad frequently to avoid burning and blistering.
14. Unplug when not in use.
15. Do not wet.

# Therma Moist Heating Pad

---

## **CAUTION**

- Unplug when not in use.
- DO NOT use while sleeping.
- DO NOT use on an infant or small child.
- DO NOT apply over sensitive skin or in the presence of poor circulation.
- Burns can occur regardless of control settings. Check skin under pad frequently to avoid burning and blistering.
- Never use pad without cover in place.
- DO NOT sit on or crush pad. Avoid sharp folds in pad.
- DO NOT expose heating pad to liquids.
- Carefully examine inner cover before each use.  
Discard the pad if inner covering shows any signs of wear or deterioration.
- DO NOT use pins or other connecting means to fasten pad in place.
- Never pull pad by the supply cord and do not use cord as a handle.
- Individuals with circulation problems should consult with a physician before using this product.
- DO NOT sit or lie on the pad.
- DO NOT use pad directly over cuts, abrasions or open wounds.
- Exercise extreme caution when using pad on non-communicative individuals.
- Handle control mechanism with care.
- DO NOT use pad as a bed warmer or foot warmer.

# Therma Moist Heating Pad

## WARNING

- Read all instructions.
- Never use the Thermaaahhh Moist Heating Pad without a clean flannel pad cover.
- Keep heating pad out of the reach of children.
- Use of this heating pad is not intended as a substitute for consultation with your physician.
- DO NOT expose the heating pad to temperature extremes (below -10 C/14 F or over 50 C/122 F) or excessive humidity (>90% RH).
- Special precautions should be taken with patients having desensitized skin.

## MOIST HEATING PADS SPECIFICATIONS

Display	Numerical knob
Button	Plastic
Control	Analog switch
Sensor	Thermal sensor
Circuit	Micro processor
Protection	Thermostat switch
Cover	Cotton & polyester
Output	Fomentation heating
Heating	115/130/150/165 F or 45/55/65/75 C
Time	15/30/45/60min.
Power consumption	AC 90-130V 40-70Hz: 10890 2.2A/290W max. 10893 1.1A/145W max. 10895 0.6A/70W max.
Pad sizes	10890 / 14" x 27" / 68 x 36cm, 10893 / 14" x 14" / 36 x 36cm, 10895 / 7" x 15" / 38 x 18cm
Operating condition	Temperature / 68-104 F / 20-40 C Humidity / 30-75%RH
Storage condition	Temperature / 50-122F / 10-50C Humidity / 10-90%RH

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# 1 Year Limited Warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for 1 year of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 1 year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

## Classification

Class II Equipment

Operating condition: Temperature: 20-40 C/68-104 F

Humidity: 30-75%RH

Storage condition: Temperature: 10-50 C/50-122 F

Humidity: 10-90% RH

Type BF Applied Part

Do not expose the Heating-Pad to temperature extremes (below 5 C/41 F or over 50 C/122 F) nor excessive humidity(>90%RH).

Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

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