drive
WWW.DRIVEMEDICAL.COM

bariatric bath bench

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banc de bain renforcé

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
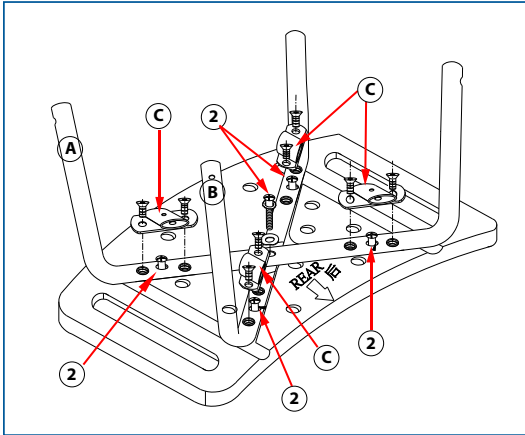
EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

drive
WWW.DRIVEMEDICAL.COM


installation instructions

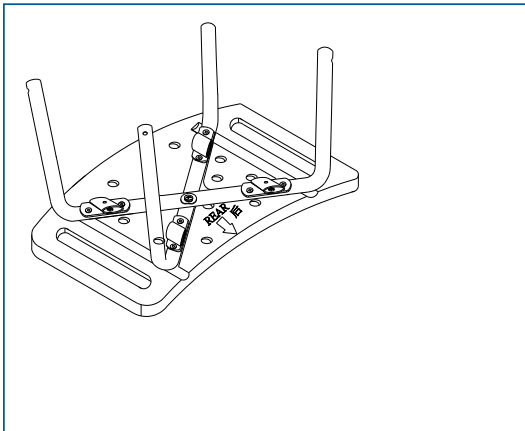1. Place cross braces (A&B) in seat channel.
2. Secure cross screw (2) with washer in center of cross brace and tighten.
3. Secure frame brace (C) into position with frame brace screws.
4. Secure flat head screws through frame brace (C) to secure legs to bench.

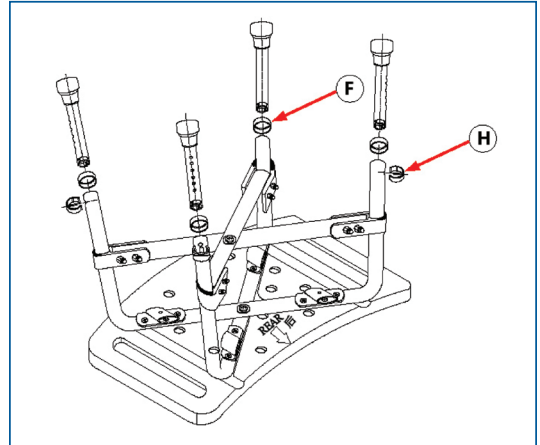
1. Align support bar (D&E) into side brace (F).
2. Tighten with center screw and bolt and at each side brace (F) with screws.

1. Place round washer over tube and leg at each side. Place bow shaped washer (H) around tube and tighten with screw.

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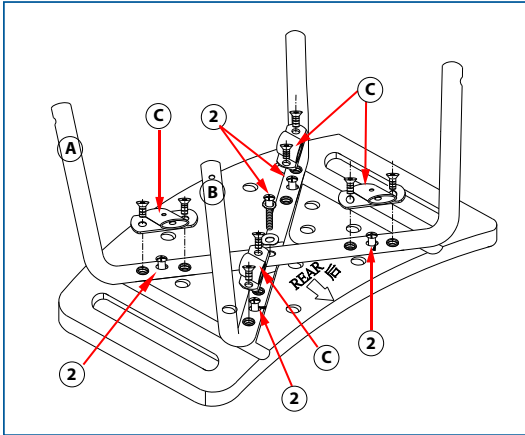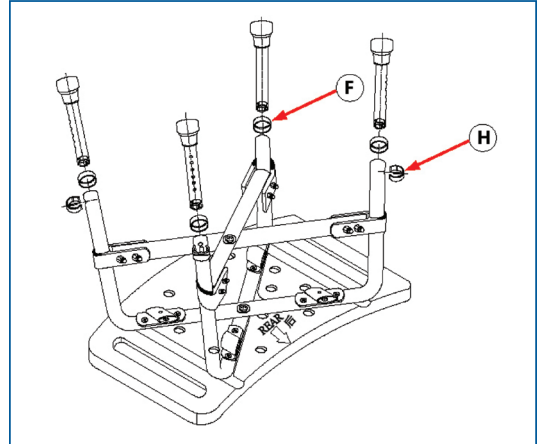
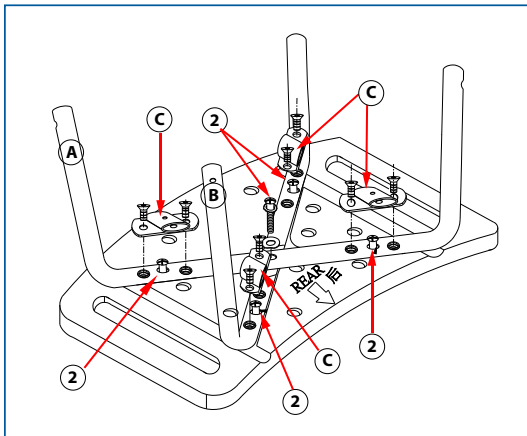
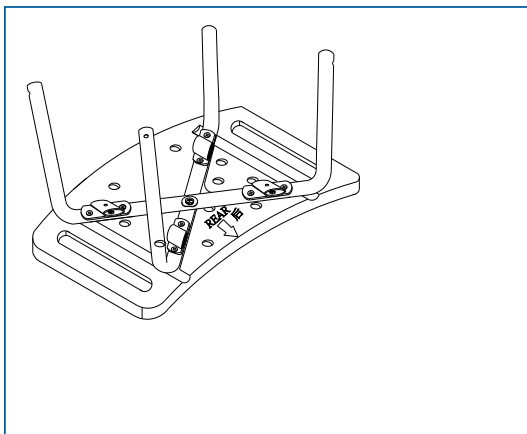
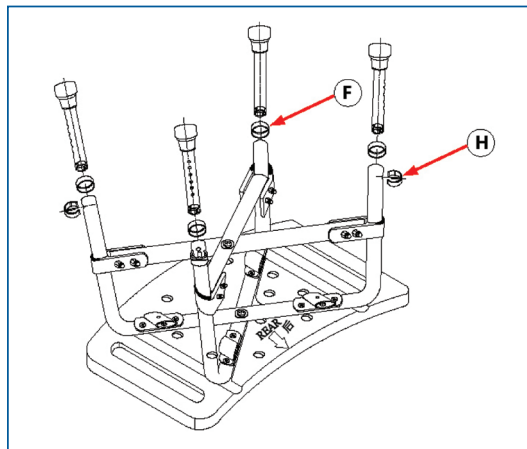
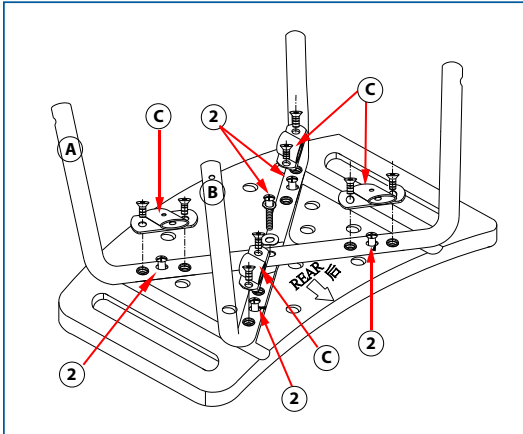
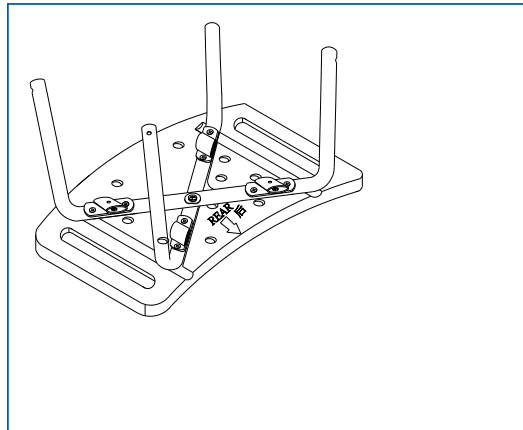
lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.
© 2006 Medical Depot, Inc. Alle Rechte vorbehalten.
Drive ist ein Markenzeichen der Medical Depot, Inc.