www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

portable shower bench
banco portátil para ducha
planche de bain

EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation instructions

1. Remove contents from box.
2. Attach red handle by threading plastic screws through the bottom of the bench and tightening.
3. Attach rubber grips by threading the plastic screws through the top of the bench and into the rubber grips, making sure the grey rubber side faces outward. This will be the side that makes contact with the bath tub.
Do not tighten.
4. Place the shower bench in the tub with the red handle on the outside. This handle provides support into and out of the tub.
5. Move rubber grips flush against the side of tub and tighten screws.
6. Make sure bath bench is secure before each use.

NOTE: to maintain to the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe shower bench dry after each use.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2005 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.