lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2002 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

mayo instrument stand double post

item # 13045



www.drivemedical.com

assembly

CONTENTS:

- Base (Figure 1)
- Upright Frame (Figure 2)
- Tray Frame (Figure 5)
- Two Rods (Figure 6)
- S/S Tray (Figure 9)
- Snap base and upright frame together (Figure 3).
- 2. Screw the rods into tray frame (Figure 7).
- Insert tray frame into the completed base and stand up assembly (Figure 8).
- 4. Place tray on top of Mayo Stand (Figure 9)

