

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

free standing bariatric
trapeze with base

item # 13049

drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com


assembly instructions

1. Connect parts 1, 2 & 3 with springs (A) and secure tube tightly.
2. Connect parts 3 & 4 with bolt (B) and secure tightly.
3. Insert ring (C) over the tube of part 4 and secure in place with bolt (D).
4. Hitch the chain into the gap between (D) and (C).

IMPORTANT

Prior to patient use, be sure to tighten trapeze clamps onto either bed or Trapeze base.

WEIGHT CAPACITY: 650 LBS.

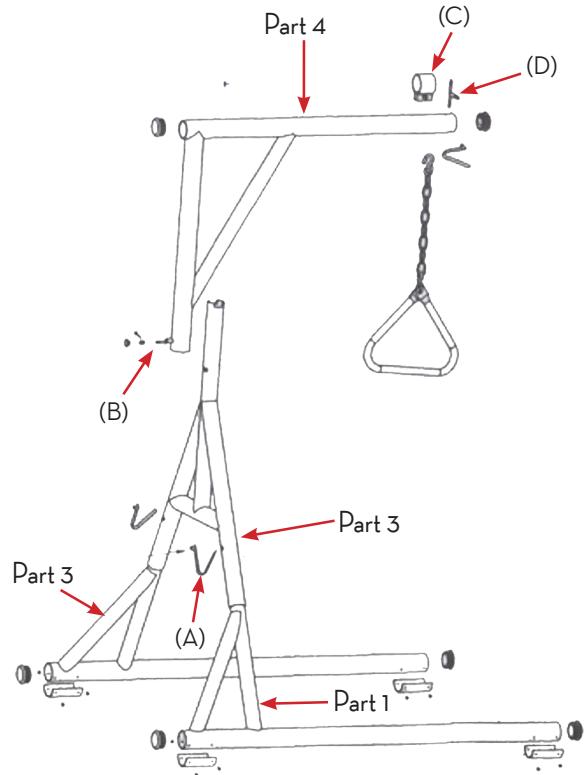

drive

MEDICAL DESIGN & MANUFACTURING

drive

MEDICAL DESIGN & MANUFACTURING