

## Directions For Application, Use and Care

## Application

The Sling Seat Wheelchair Alarm System can be used in conjunction with the Drive Sensor Basic Alarm, and mounting bag to alert staff when a patient gets out of a wheelchair. When sitting on the sling seat, the alarm is inactive. When weight is removed, the alarm sounds, indicating the person may need assistance.

## Installation

1. Remove existing fasteners and sling seat from chair frame (Figure 1).
2. Install new Sling Seat Alarm, and reinstall fasteners (Figure 2).
3. Connect Sling Seat Alarm to Sensor Basic Alarm. Sensor Basic Alarm can be mounted to wheelchair with the supplied mounting bag (Figure 3), or by use of the metal clip on the back of the unit (Figure 4).

figure 1



figure 2



figure 3

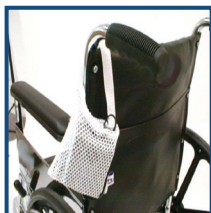


figure 4



## Care

Wipe clean with mild soap or disinfectant. Use a damp cloth or sponge. Do not submerge in any liquid.

## 6 Month Limited Warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for 6 months of the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment. This 6 Month Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.