Doormatic

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AUTOMATIC DOOR OPENER

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OPERATING INSTRUCTIONS AND
SAFETY PROVISIONS

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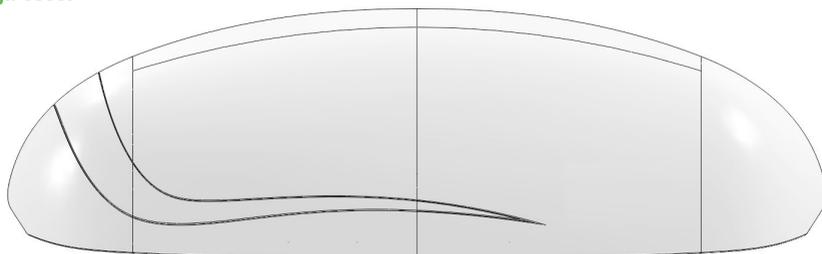
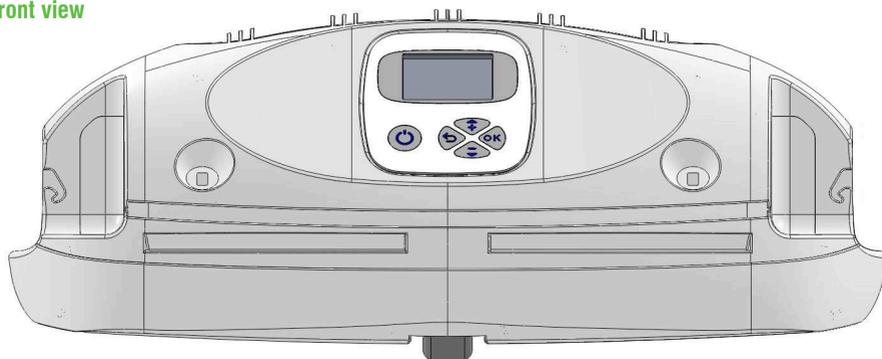
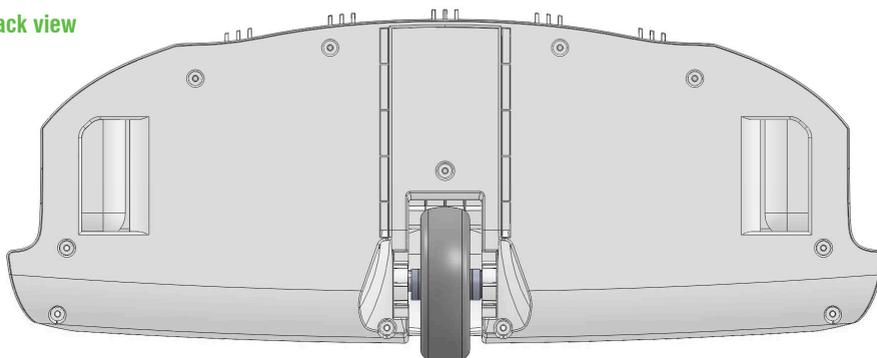
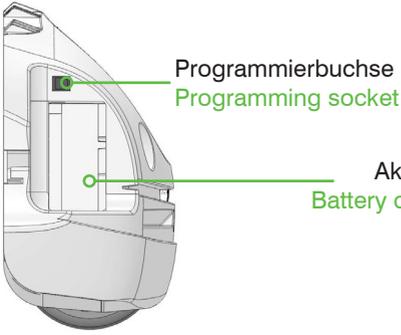
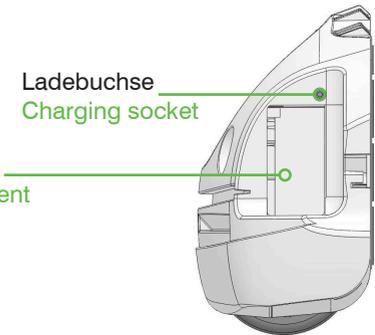
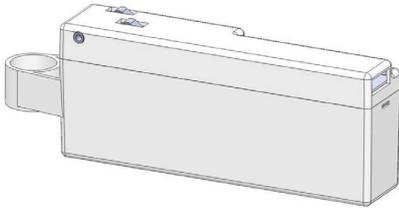
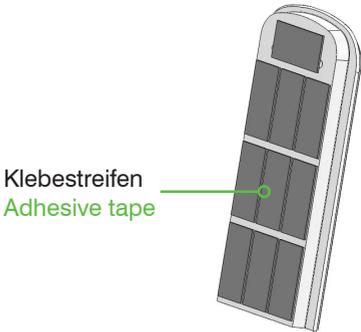
Türhalter Door stay

Türband Door hinge

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You can operate Doormatic as follows

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At the front part of the device, you will see the display with On and Off buttons as well as the navigation buttons just under the design cover.

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The display backlight is activated automatically at the press of a button. If no button is pressed, the backlight will disappear after some time. After a few minutes, the display will switch off. Press any Button to activate.

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Doormatic is designed for automation of standard Interior swing doors with rabbet and a door leaf weight of 50 kg max.

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Doormatic supports with opening and closing these doors. Drive Mobil will not be liable for any damage caused due to any other use beyond what is specified. The operator bears the risk for this.

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The traction wheels will work on even and dry floor surfaces. These must be free from any dirt, dust, oil, grease and other impurities. On wet or dirty floors, performance will be reduced.

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Doormatic is suitable for standard interior swing doors with rabbet

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Do not use doormatic for doors with specific requirements, e.g.:

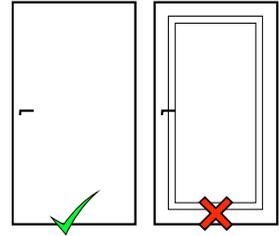
- Fire doors
- Smoke compartment doors
- Escape doors

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
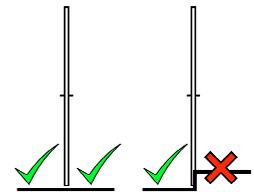
Position am Türblatt Position on the door leaf

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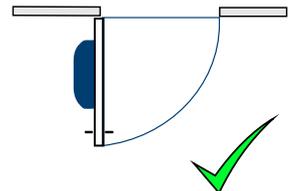
Alternative:

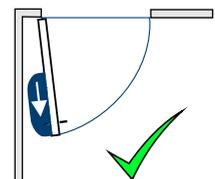If the space between door and the wall or cupboard for example is smaller than the depth of the device, the device shall then be installed at the outer edge of the door.

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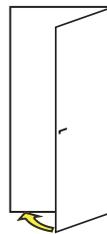
(1) Close the door and make sure that its remains closed during installation.

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(2) Clean the lower door area with the enclosed cleaning cloth.

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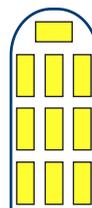
(3) Wait for at least 1 minutes, until the detergent vaporises.

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
(4) Stick the adhesive tape to the guide rails as illustrated and remove the protective foil

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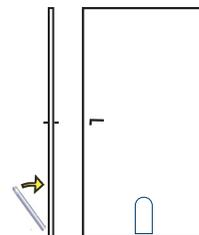(5) Attach the guide rails around 1 cm above the lower edge of the door, with the rounded end of the guide rail pointing upwards. Make sure that it is mounted straight and vertically so that your Doormatic works optimally. Press the guide rails firmly on the door for at least one minute.

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
Optionally, the guide rails can be screwed on. Use the specified drill holes for this purpose.

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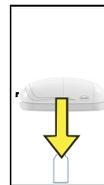(6) Insert Doormatic from the top into the guide rails. The device will be held to the door on the guide rails.

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(7) Remove the design cover from Doormatic. For this, hold it firmly with both the hands on the left and right hand sides and pull it towards yourself.

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Insert the fully charged battery into one of the battery compartments.

Insert the end of battery into the compartment with the battery's contacts pointing upwards.

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You may connect the charging unit with Doormatic permanently or simply just when required. You will find the charging socket at the right hand side of the device.

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Please connect the mains supply for devices without battery.

The socket for the mains connection is present at the right hand side of the device.

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Ensure the cable cannot pose an entrapment or other hazard when the Doormatic is idle or operating.

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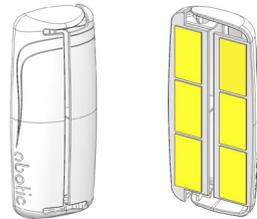
Use the prescribed cable duct at the front part of the device in order to lead the cable towards left or right hand side as required.

Installation des Türhalters und Türbandes Installation of the door stay and door hinge

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(1) Assemble both the halves together. Hold the door stay in your hands and simulate the snapping process of both the halves. In this way, you can easily determine which half has to be fixed to the door and which to the wall.

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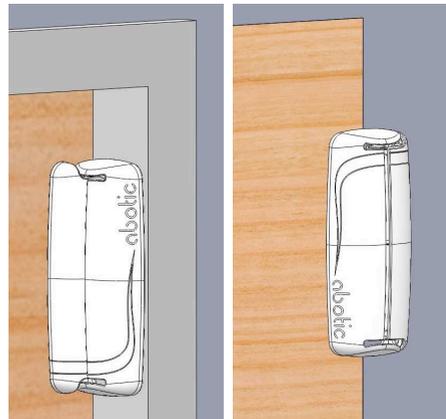
(2) Apply the adhesive tape as illustrated and remove the protective foil.

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(3) Fix the door stay to the closing edge at the upper door area. One half will be fixed to the door and the other to the doorframe or wall.

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(4) If the door was locked, you may unlock it now.

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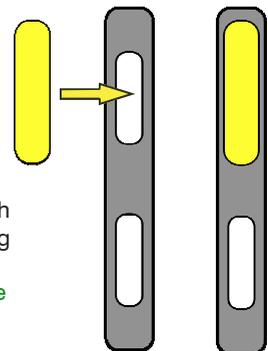
(3) Attach the door hinge over the upper recess of the door handle on the side of the doorframe so that it is completely covered.

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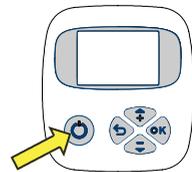
(1) Switch off Doormatic by pressing the On/Off button.

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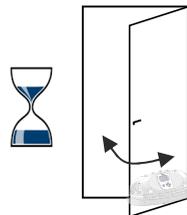
(2) Open the door up to ca. 5 cm (2") - just wait, the Doormatic start the test walkthrough automatically.

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(3) Your door will be completely open and then close. When activated next time, the door will open 2 cm less than during the test walkthrough.

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If you want to automate several closely located doors, press the radio transmitter button during the test walkthrough in order to couple the accessory. See page 18.

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(4) After a successful test walkthrough, the Drive Mobil logo will be shown on the display screen. Put the design cover back on the Doormatic.

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Activate Doormatic directly on your door by pushing or pulling it open. A few centimetres will suffice to activate the device. A waiting period before the door is opened or closed can be set through the menu items "waiting period OPEN and "waiting period CLOSE". Further information regarding the waiting periods has been provided on page 18.

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You can stop or move your door in the opposite direction even during when walking through, your door is always movable.

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Control Doormatic using a remote control device or motion detector.

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Please note that in case of radio activation, the "waiting period OPEN" function is disabled. The door will be opened immediately after the radio signal is received.

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At the press of a button, the door will open and then close after the waiting period has lapsed. In order to stop the device, press the button on the remote control again during when walking through.

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For contactless activation, use the radio motion detector.

On nearing, the door will automatically be opened. The closing cycle takes place 10 seconds after leaving the detection area of the motion detector. A longer waiting period can be set in the settings.

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For this function, the radio connection must be activated as described on page 18, and on page 21 under the heading "properties".

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When the battery level is low, a signal tone will be heard during the opening or closing cycle. Doormatic can still be used about 20 times. Your door can be manually closed or opened at any time as usual even with an empty battery.

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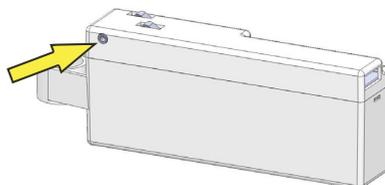
You can charge the battery inside the device as well as externally. Connect the charging unit with a power socket and the charging plug to the Doormatic. The charging socket is located at the right hand side of the device.

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To charge externally, remove the battery from the Doormatic and connect the charging plug with the battery.

Der Ladevorgang dauert bei leeren Akkus ca. 12 Stunden pro Akku.
The charging process takes around 12 hours for each battery from flat.

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Hindrance recognition and force limitation

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The Doormatic is equipped with an automatic hindrance recognition system. On contact with an obstacle, the device immediately stops door movement.

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Hindrance in the opening cycle (when the door is opening)

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If Doormatic recognises a hindrance while the door is being opened, the opening cycle will immediately stopp and the door will close again.

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Hindrance in the closing cycle (when the door is closing)

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If a hindrance is recognised while the door is being closed, the closing cycle will immediately stopp and the door will reopen. The door will close according to the waiting period set. If the hindrance arises again, the door will open and will remain open. The device will be disabled for safety reasons. Before activating it again, check for any hindrance at the doorway. Then activate it through Push & Go or radio. The door will close again as usual. All settings remain unchanged.

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Doormatic is equipped with an auto-correction feature.

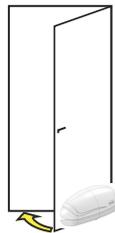
The door is opened up to the maximum possible opening angle and then closed again. Auto-correction is carried out by the Doormatic during an opening cycle when required.

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You can remove the Doormatic from the door as follows

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(1) Close the door and make sure that it remains closed during the removal process. You may lock the door.

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
(2) Remove the design cover and switch off the device.

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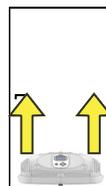
(3) Move the device upwards along the door surface using both hands and lift it out of the guide rails.

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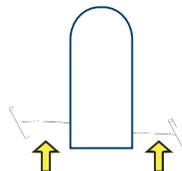(4) Remove the adhesive strip foam carefully with light saw movements from the bottom to the top, using a thin wire or a hard string.

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(5) The adhesive remains can simply be removed without any detergent or scouring agent.

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(6) If the door was locked earlier, you may unlock it now.

Passen Sie die Doormatic individuell an **Customise the Doormatic according to your personal needs**

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All the essential settings are automatically made. Optionally, you can customise the settings according to your individual desires. On the following pages, all menu items and their functions have been described. For navigating and choosing the menu items, use the buttons on the front side.

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Set the time period that shall elapse before the door is opened

↵ **Funkverbindung / Radio link**

Aktivieren Sie Ihre Doormatic bequem per Funk.

Activate your Doormatic comfortably through a radio link

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Radio is inactivate. If no radio accessories are being used, the battery will last longer between charges.

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The device responds to coupled radio transmitters. For

coupling, choose the menu item: “Couple radio accessories”

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Choose whether you want to couple the remote control device or motion detector. Press the remote control device or move the motion detector such that the red LED lightens up. If a radio signal is recognised, a single tone will be heard.

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The coupled accessory is disconnected. If the remote control device or motion detector is not coupled properly, you may remove it again

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This value relates to the 10 cm of space available before the door is closed. If your door cannot be completely closed, increase the closing force. If a loud noise is produced when the door is closed, reduce the force.

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The adjustable sensitivity of the hindrance detection system relates to the 10 cm of space available before the door is closed. Reduce the sensitivity only if the door cannot be closed completely.

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You can adjust the walkthrough path, the distance over which your door is automatically opened or closed.

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Repeat the test walkthrough at the time of initial commissioning. Any radio couplings will be disconnected and reconnected during the test walkthrough. All other settings will remain unchanged. Please pay attention to the instructions displayed on the screen.

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All settings as well as radio connections remain unaffected. Please pay attention to the instructions displayed on the screen.

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Here, you can adjust the walkthrough path in cm steps very precisely. Use the + / - buttons in order to increase or reduce the cm values.

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Adjust the speed of Doormatic to your personal tempo. There are five speed options to choose from:

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Risk of injury!

High tempo settings may lead to injuries and accidents. Select the lowest possible tempo for safety reasons.

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As an additional safety feature, a signal tone can be activated. For the setting “Everything On”, a signal tone can be heard when the door is opened as well as when it is closed.

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Choose from different activation options and operating modes and adjust the hindrance recognition optimally.

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Adjust sensitivity of the hindrance recognition feature to different floors.

For uneven floors (e.g. flow floors), reduce the sensitivity.

For a floor with smooth surface such as laminate, you can increase the sensitivity.

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How shall Doormatic be activated? Choose whether you want to activate the device through the Push & Go feature, radio or both. The setting “fix the door” corresponds to the setting “through radio” with protection against wind, i.e. unintended activation through draught.

nur Push & Go > nur Funk > Push & Go und Funk > Tür fixieren
only Push & Go > only Radio > Push & Go and Funk > Fix the door

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For the standard settings “Open + Close”, your door will be opened with Push & Go as well as through radio activation automatically and then be closed after the waiting period set has elapsed.

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In the operating mode “Door closer”, the door is opened only through radio activation. If the door is opened directly with hands, it will again be closed immediately after the door handle is released.

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Shows the battery voltage as well as the number of opening and closing cycles.

↪ **Softwareversion / Software version**

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Here, you will find the installed software version. This may be required by our service team in order to provide you with quick service support.

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Drive Mobil service partners can install a new software version here.

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Doormatic does not require any maintenance.

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You can clean the device in the course of your normal household cleaning routine with a moist (not wet) cloth without the use of any detergents. Do not use any scouring agent or scrubbing sponges.

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Check regularly whether any hair, dust etc has accumulated around the wheel area and remove this dirt then if any.

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The traction wheel was manufactured using a special plastic material which ensures high adhesion and extremely low abrasion. Nevertheless, some abrasion marks may appear over time. These can be removed with ease however in course of normal floor maintenance activities.

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Read and follow these instructions. Please pay attention to all warnings and retain this document so that you can read it later. If you hand over Doormatic to someone else, do not forget to give them this document as well.

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Doormatic is (if tempo, dynamics and power are set to “very slow“ or. “very weak”) a low energy mode device (**low energy**) according to **DIN 18650**.

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Owing to technical reasons, the batteries discharge themselves over time, even if idle or unconnected. This may reduce their service life. Do not leave the battery discharged. Charge it regularly in order to avoid the discharged state.

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We recommend that you should charge the battery every six months in order to prevent it from getting completely discharged.

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Disposal of batteries

Batteries shall not be disposed off with normal household garbage.

They must be disposed off at an appropriate collection point reserved for recycling purposes. Bear the local regulations in mind. Batteries may contain mercury, cadmium or lead in smaller amounts than specified in the provisions of the Battery Directive 2006/66/EC.

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In order to avoid any danger, the factory settings of Doormatic are set to either low or normal when delivered. Changes in these settings can be made at your own risk. If damage or injuries arise due to higher speeds, Drive Mobil will not be held liable.

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For use in public areas, we recommend notice signs for labelling of the door. These can be obtained from Drive Mobil.

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For use in moist areas, please note that:

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The device as well as the accessories shall not be disposed off with normal household refuse. They must be disposed off at an appropriate collection point where electric and electronic devices are gathered for recycling purposes. Bear the local regulations in mind. You will receive further information about recycling from the responsible authorities on site, from your disposal firm and from your dealer.

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
Limitation of liability

Drive Mobil GmbH accepts no liability for any compensation, indirect damage, incidental or consequential damage, including but not restricted to any loss of profits, revenue or commercial loss due to breach of a tacit guarantee and especially when Drive Mobil GmbH was notified of any such damage.

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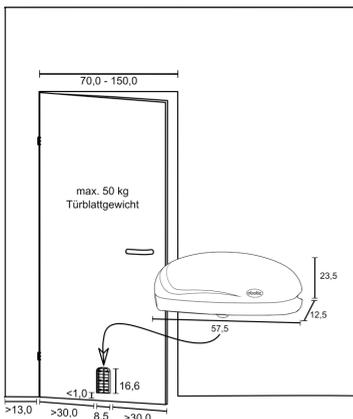The device cannot be switched on

Check whether the battery has been inserted properly and the power cable is connected.

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The device cannot be activated with Push & Go

Check whether the device has been switched on and in the menu “Activation”, “Push & GO” has been selected or not. Open the door up to 5 cm.

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The device cannot be activated with radio

Check whether the device has been switched on and whether “On” has been selected in the menu “Radio connection” and in the menu “Activation”, “Radio” has been selected or not.

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The device gets activated unexpectedly

Check the alignment of the motion detectors. Sometimes, the detector responds to movement of pet animals or the sensor range is extended through a wall mirror unwillingly.

While coupling the radio, the motion detector and remote control may have been swapped.

Disconnect the radio accessory and couple it again as described on page 19.

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Sometimes, the device opens the door more than what has been specified

An auto-correction is being carried out. Through this, the Doormatic readjusts the walkthrough path settings. This function is automatically activated only when required.

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The device opens the door too wide or not wide enough

Readjust the walkthrough path as described on page 20.

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Suddenly, a signal tone is heard

The signal tone is an indication of low battery status. Charge the battery as described on page 14. More information about the signal tones has been provided on page 20.

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Drive Medical GmbH & Co. KG • Leutkircher Straße 44 • D-88316 Isny/Allgäu • Germany