
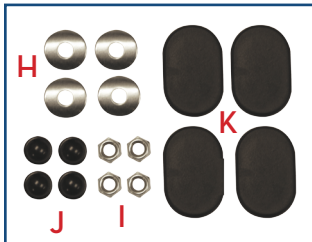


crawlabout

parts list

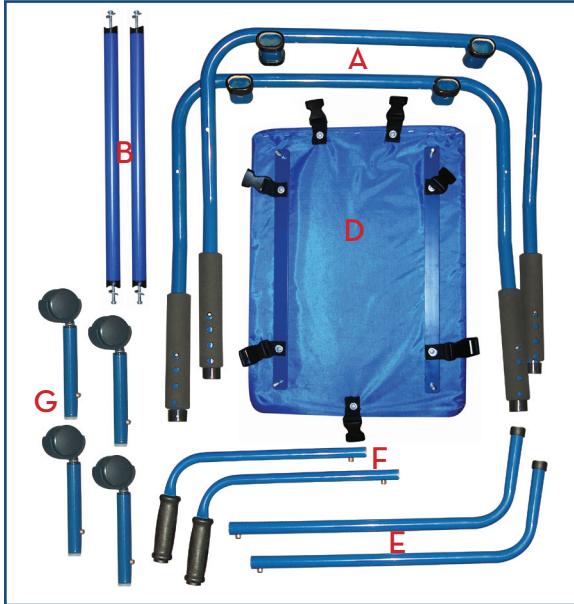

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
MEDICAL DESIGN & MANUFACTURING

Wenzel^{ite}Re/hab

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

crawlabout


item # CAB 2000 & CAB 3000

drive
MEDICAL DESIGN & MANUFACTURING

Wenzel^{ite}Re/hab

www.drivemedical.com

assembly

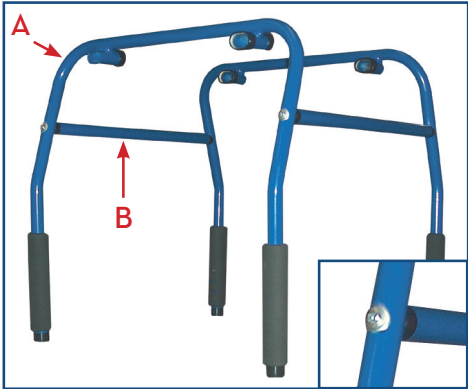

step 1
Remove screws from pole (B), line up end of pole (B) with leg sections (A) and insert screws and curved washers (H). Repeat with second pole (B).
NOTE: Position curve of black plastic end cap of pole (B) to match leg section (A) angle.




step 2
Place cushioned pad (D) on floor face down.



step 3
Place frame over the pad inserting all four screws (C) into frame. Align front of frame with front of pad (Frame and pad are marked front and back.) Install four nuts (I) onto screws (C). Cover nuts with black caps (J).

step 4
Insert all four legs (G) into leg section (A) at desired height.

The CrawlAbout can be used with or without push handles.
To use with push handles:

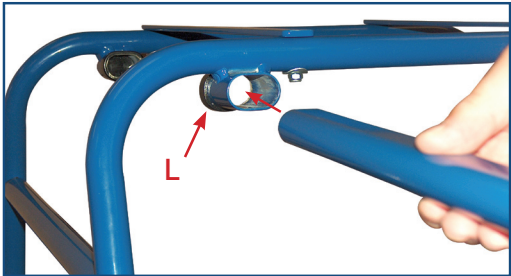

step 1
Insert push handle bracket (E) into mounting tube (L) on the back leg section. Make certain that the push button clicks into hole on underside of mounting tube on the front leg section. Repeat on second side



step 2
Insert push handle (F) into bracket (E) at desired height. Repeat on second side.
When using without push handle insert large black end cap (K) into mounting tube (L) on the back leg section.

operation



step 1
Lay harness (M) on floor and place child on stomach on top of harness. Place CrawlAbout above child. Attach side straps to clasps on the underside of cushioned pad doing front side clasps first. Attach the back straps and then the shoulder

step 2
Adjust harness straps length as needed.

step 3
Adjust leg height to appropriate height.