

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

positioning bar

CE 1052, CE 1053, CE 1054



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

Wenzel *ite* *Re/hab*

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

positioning bar

Positioning Bar is mounted on the handlebar of Safety Roller for adjustable handlebar depth.

CE 1052 - For use with CE 1200

CE 1053 - For use with PE 1200

CE 1054 - For use with PE TYKE 1200

assembly

1. Mount brackets anywhere along the handlebar. (Figure 1)
2. Insert positioning bar into brackets as shown. (Figure 1)
3. Tighten screws. (Figure 2)

adjustments

1. Loosen the screws on the brackets (Figure 2) and slide the bracket with the positioning bar forward or backward. (This allows you to adjust the handlebar depth.)
2. Accessories, pelvic stabilizer and trunk support, can be mounted on the positioning bar. (Figure 3)

