Wenzel Ite Re/hab

one year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



Wenzel te Re/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

back2go item# CRB 2000



assembly

Your Back2Go comes in one carton completely assembled.

use

- Remove both sets of securing straps from pocket and open Fastex[™] buckles. (Fig 1).
- Place Back2Go on any chair or in a stroller, wrap both straps around the back and close the buckles. (Fig 2)
- 3. Adjust straps to fit tight by moving belt clips.
- 4. Open zipper on trunk support.
- 5. Place child in seat and close trunk support.

adjustments

- To adjust height of laterals, remove knobs (A) on the back, place the lateral in the appropriate slot at desired position and replace knobs. (Fig 1)
- To adjust width of laterals, loosen knobs (A) and slide lateral to desired width and tighten knobs to secure in place.
- To raise the trunk harness, remove shoulder straps (B) from Velcro[™] strips in the back and adjust to desired position. (Fig 3)
- 4. To adjust the width of the harness, remove lower straps (C) from Velcro™ straps in the back and adjust to desired position. (Fig 3) You can also open the Fastex™ buckle for additional width adjustablility.

accessories

(sold separately)

CRS 8000 Headrest



figure 1



figure 2



figure 3