

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite Re/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

dolphin bathing chair

Item # DO 2000



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*ite Re/hab

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

recommended use

The Dolphin Bath Chair provides support in the bathtub during bathing. The seat, back and legs are fully adjustable. All adjustments are tool-free.

assembly

No assembly required.

optional accessory

Base, Item # DO 2010.

adjustments

1. Loosen knob (A) to adjust the back angle and knob (B) to adjust the seat angle. Tighten knob to remain at desired angle.
2. There is 5" of seat depth adjustability. Lift the fabric flap (C) under the seat, push in the spring buttons located on the undersides of the seat frame and pull or push bar to preferred seat depth.
3. Headrest strap and positioning belts can be placed anywhere on the bath chair. Securely engage hook and look of strap once positioned. Slide head cushions (D) along the strap for width adjustment.

caution

Adult supervision is required at all times. The straps and supports must be carefully adjusted for the comfort and security of the child. Child must be strapped in at all times.

