

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
WWW.DRIVEMEDICAL.COM

Wenzel^{ite}Re/hab

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

first class chair

item# FC 2000 & FC 4000



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

Wenzel^{ite}Re/hab

Wenzel^{ite}Re/hab

drive
WWW.DRIVEMEDICAL.COM

drive
WWW.DRIVEMEDICAL.COM

first class chair

installation

1. Remove contents from carton.
2. Turn seat upside down and open knobs (A) by turning counter clockwise (figure 1).
3. Slide back support bracket (B) into red seat assembly (C) on the underside of the the seat.
4. Tighten knobs (A) to secure back.
5. Insert armrests as per diagram (figure 2).

adjustments

Seat Depth

Loosen knobs (A), pull/push back to the desired position and tighten knobs to secure in place.

Seat Height

Loosen knobs (D) on each leg, pull/push inner leg to desired position and tighten knobs to secure in place. A 15° anterior or posterior tilt can be achieved by setting front and back legs at different heights.

Armrest Height and Width

Loosen knob (E) and adjust height and width as appropriate and tighten knob (E).

Lap Belt

Adjust belt length through belt buckle and secure with the Velcro. Additional length can be achieved by adjusting the placement of belt on Velcro strips on the underside of seat.

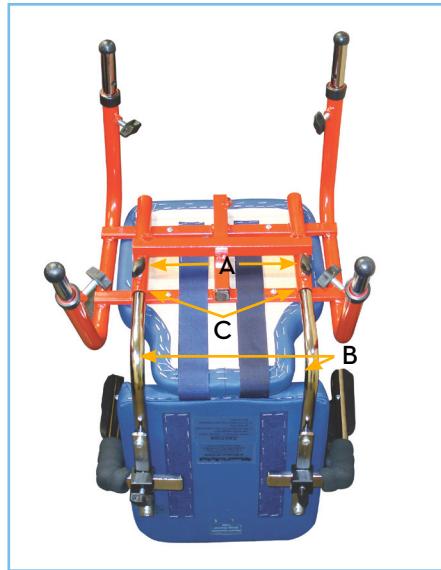


figure 1

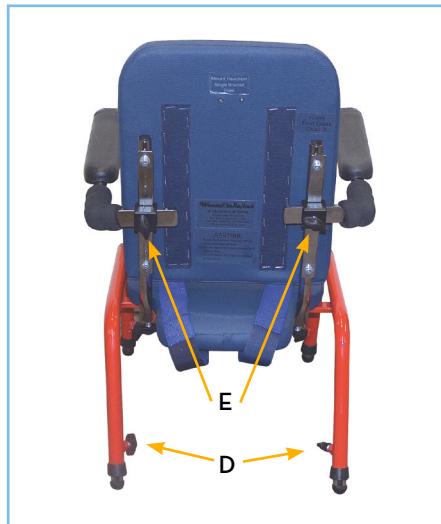


figure 2



accessories (sold separately)

Description	Item #
Tray	FC 2024 & FC 4024
Footrest	FC 2028 & FC 4018
Headrest	FC 8000
Headrest Wedges	WD 8000
Hip Guides	FC 8027
Mobility Base	FC 2029 & FC 4029
Leg Extensions	FC 8050
Anti Tippers	FC 2030 & FC 4030
Support Kit	FC 8025 & FC 8025L
Clear Tray	FC 2026 & FC