

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

mobility base

item # FC 2029 & FC 4029

for use with first class chair
(item # FC 2000 & FC 4000)


Wenzel *ite* *Re/hab*

drive
WWW.DRIVEMEDICAL.COM

set up instructions

NOTE: Chair height affects base's length.

1. Loosen knobs (A) and open Velcro® strap (B). Extend the base so that the chair can be properly inserted.

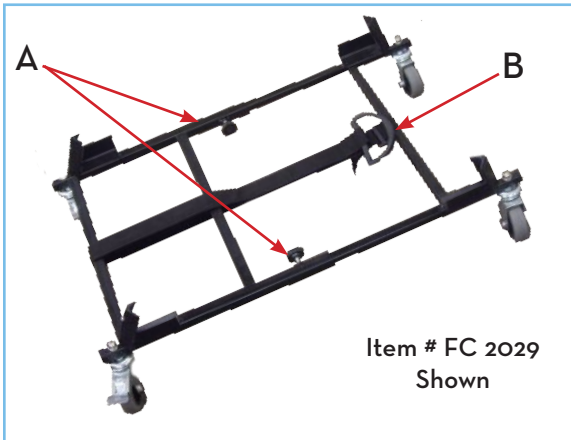

2. Insert chair into base, position front legs flush against the front retaining brackets and pull the Velcro® strap (B) to bring the base together as tight as possible.



3. Close Velcro® strap (B).
4. Tighten knobs (A).