

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

drive
WWW.DRIVEMEDICAL.COM

Wenzel^{ite}Re/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in BULGARIA

headrest for first class school chair

item # FC 8000

for use with FC 2000 & FC 4000



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB


Wenzel^{ite}Re/hab

www.drivemedical.com

drive
WWW.DRIVEMEDICAL.COM

installation

1. Insert mounting bracket bolts (A) into the back of the chair and tighten.
2. Loosen retaining bracket (B) on vertical post.
3. Loosen knob (C) on mounting bracket and slide vertical assembly post (D) into mounting bracket. Tighten knob (C).
4. Position retaining bracket (B) flush against mounting bracket and tighten.
5. Loosen 3 Allen screws (E) on triangular rotating bracket, insert horizontal assembly post (F) and tighten Allen screws.

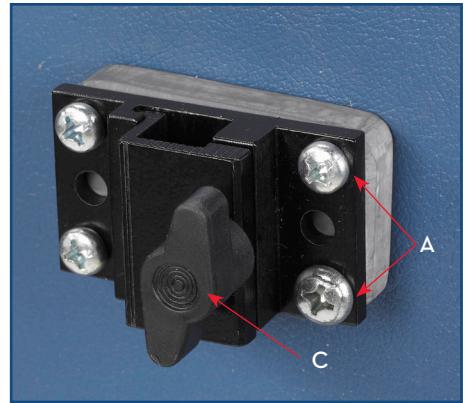

adjustments

Height

Raise/lower vertical assembly pole to desired position.

Depth

Loosen 3 Allen screws (E) on triangular rotating bracket and push/pull headrest to desired position. For additional depth adjustability, mount the vertical assembly pole with the curve either facing inward or outward.

Angle

Loosen 3 Allen screws (G) on circular bracket on headrest and tilt headrest to desired angle. Additional angle adjustability can be achieved by loosening the 3 Allen screws (E) on the triangular rotating bracket and tilting headrest from laterally.

