

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

hip guide


item# FC 8027

(for use with item# FC 2000 & FC 4000)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

Wenzel *Re/hab*

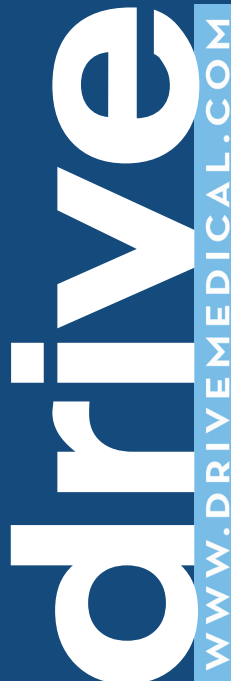


set up instructions

1. Remove pads from bracket. They are attached with velcro-type strips.



2. Loosen right and left knobs (A) on underside of chair.

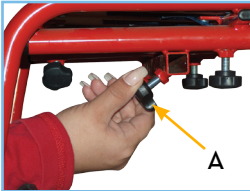

3. Insert hip guide bracket, into square receptacle under the seat. Adjust to desired width and tighten knob to secure.



4. Attach pads to bracket.



Item # FC 8027
Shown

adjustments

1. To adjust width, loosen knob (A), push/pull pad bracket to desired width and tighten knob to secure in place.
2. To adjust depth, place pad towards the front or back of bracket. Pad is secured with velcro-type strips.

