

two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*iteRe/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

nimbo lightweight posterior safety roller

KA 1200N, KA 2200N, KA 3200N,
KA 4200N, KA 5200N



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*iteRe/hab


www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

user instructions

1. Remove Nimbo from carton.
2. To unfold the Nimbo, pull handgrips and cross brace in opposite directions. Continue to unfold until the push pins “click” into place.
3. To install the legs, match the leg numbers with the corresponding frame numbers. Depress the push pins on the legs and slide into the frame. When locked into position, the push pins will “CLICK” through the frame.
4. Adjust height of walker by pressing push pins on each leg and slide leg to desired position.


FIGURE A

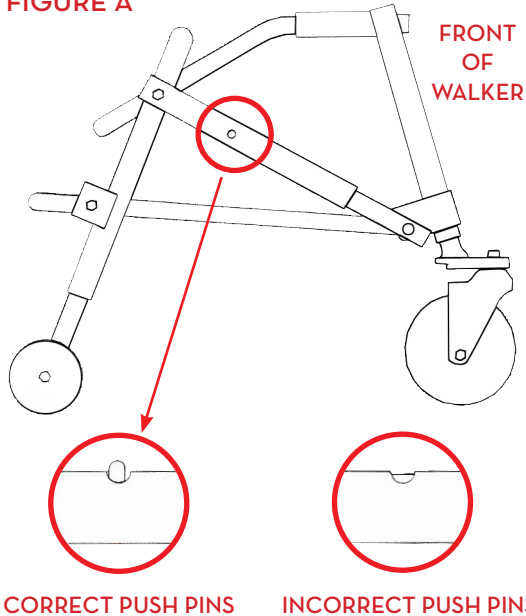

user information


1. Lift and position knob (A) to set front wheels in either the swivel or locked position.

FIGURE B

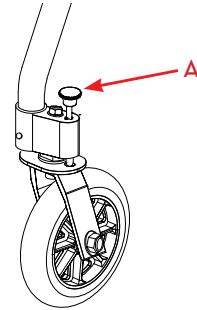

2. To fold the Nimbo, face it from the open end, depress the push pins, tilt walker so that it rests on its rear legs only and push the front legs toward the rear legs.
3. To override the one-directional feature of the rear wheels, lift up aluminum pin and insert into override bracket

FIGURE C

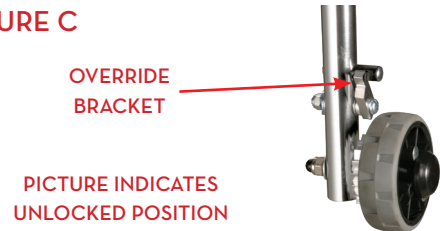

maintenance and care

1. Inspect the Roller regularly. Tighten the nuts and bolts as necessary. If a product needs service or replacement parts, contact an authorized Drive dealer.
2. To clean the Roller, use warm water with a mild detergent. Wipe clean with a soft, dry cloth.