MICHAEL GRAVES
active living
COLLECTION™

Bag Cane Instructions & Warranty

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EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA
Made in China

Item # MG10304

Do not use the Michael Graves Active Living Collection™ Bag Cane without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions, or Instructions, contact a healthcare professional, dealer or technical personnel before using or adjusting this equipment. Otherwise, injury may occur.

Check rubber tip for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tip immediately. Michael Graves Active Living Collection™ Canes can provide ambulatory assistance for an individual weighing up to 300 pounds. This cane is not designed to support the total weight of an individual.

Make sure that the adjustable snap button fully protrudes through adjustment hole of cane. This ensures that the cane is securely locked in position. BEFORE using cane, make sure that it is securely locked in place or a fall could result causing bodily injury and/or damage to the cane.

Always consult your physician or therapist to determine proper adjustments and usage.

Proper Cane Height

For a more comfortable fit, make sure that shoe height is the same as shoe style worn most frequently when using cane.

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Adjust cane so top portion of the handle touches inside of wrist where it meets the palm.

Height Adjustment

1. Press snap button IN (Snap button is inside large cane pocket).
2. Slide extension up or down until snap button protrudes through an adjustment hole on handle end of cane.

Note: an audible “CLICK” will be heard when snap button securely protrudes through the adjustment hole.

3. Repeat STEPS 2-3 until cane is at height determined PROPER CANE HEIGHT on this instruction sheet.

Folding & Unfolding

1. The cane can be folded for convenient transport or storage.
2. Unzip the cane pocket so it is open.
3. Pull the cane segments apart and fold them together. Do this at all three folding points on the cane.

Note: Hold the cane segments securely when folding. The segments are held together with internal elastic cords that will pull the segments back together if they are released.

4. Place all the folded segments within the cane pocket of the bag and zip it closed for convenient storage and transport.
5. To unfold, unzip the cane pocket and pull the cane segments out. Hold the cane by the handle and allow the segments to snap together. Make sure all cane segments are locked together securely before putting weight on the cane.
6. Close the zipper on the cane pocket. Keep the cane pocket closed to prevent personal injury and/or damage to the product.

Maintenance & Care

1. Make sure snap button releases and re-engages properly.
2. Frequently inspect cane tips and elastic cords between cane segments for wear and/or damage.
3. Replace any broken, damaged or worn items immediately.
4. Spot clean bag with a damp cloth and soap. Do not machine wash.
5. Do not place any items in the front pocket that could cause strain or injury.
6. Do not store anything other than the cane in the large cane pocket.

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Lifetime Limited Warranty

Your Michael Graves Active Living Collection™ by Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Michael Graves Active Living Collection™ by Drive device or this warranty, please contact an authorized Drive dealer.

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