MICHAEL GRAVES
active living
COLLECTION™

**Height Adjustable
Tub Rail With Soft Cover
Instructions & Warranty**

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EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA
Made in China

Item # MG12050SC

Installation Instructions

1. Clean bathtub thoroughly and wipe dry before applying rail. Contact surfaces must be dry and free of soapy residue to assure secure installation.
2. Loosen knob (A) until rail is wide enough to slip on side of bathtub.
3. Place Safety Rail on bathtub with knob (A) facing outside of tub and tighten until rail feels tight and secure.
4. To test installation of rail, pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and retighten.

Height Adjustment

After the tub rail is installed on the tub, remove the locking collars. Once removed, pull up or push down from the top of the handle to the desired height. Once the desired height is determined, making sure both holes line up, replace the both locking collars. Be sure the locking pins go through the tubing and protrudes on the opposite side.

NOTE: Safety Rail often requires retightening from time to time. Check periodically before use and retighten as needed.

CAUTION: Please use extreme care when installing on fiberglass tubs. DO NOT OVERTIGHTEN as the clamp may crack the fiberglass.

Limited Lifetime Warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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