Drive Medquip COMPRESSOR NEBULIZER SYSTEM

SISTEMA DE COMPRESOR NEBULIZADOR

The Physician's Choice

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TABLE OF CONTENTS

SPECIAL NOTES	1
INTRODUCTION	2
TYPICAL PRODUCT PARAMETERS	3
SUPPLEMENTAL INFORMATION	4
Compressor	4
Extension Cords	4
HANDLING	4
Unpacking	4
Inspection	4
SECTION 1 – IMPORTANT SAFEGUARDS	5
To Reduce the Risk of Electrocution.....	6
To Reduce the Risk of Burns, Electrocution, Fire and Injury to Persons	6
SECTION 2 – OPERATING INSTRUCTIONS	8
Setup	8
Treatment Instructions	10
SECTION 3 – CLEANING & DISINFECTING ...	11
Cleaning the Outer Case	11
Cleaning the Compressor	11
SECTION 4 – MAINTENANCE	12
Routine Care	12
Storage	12
Filter Change	12
SECTION 5 – TROUBLESHOOTING	13
LIMITED WARRANTY	14

 WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT YOUR EQUIPMENT PROVIDER BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE SERIOUS INJURY OR PROPERTY DAMAGE MAY RESULT.

SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, will result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, will result in property damage.

NOTICE

The information contained in this document is subject to change without notice.

WARNING

Drive products are specifically designed and manufactured for use in conjunction with Drive accessories. Accessories designed by other manufacturers have not been tested by Drive and are not recommended for use with Drive products.

INTRODUCTION

The Drive Compressor Nebulizer System is designed to efficiently deliver physician-prescribed medication to the bronchial lung passages. The nebulizer system consists of the following components:

- Compressor
- Disposable Nebulizer Kit (MQ5800 & MQ5900)
- Reusable Nebulizer Kit (MQ5900)
- Air Tubing
- Mouthpiece
- 5 Extra Filters

MODELS	DESCRIPTIONS
MQ5800	Compact Compressor Nebulizer with Disposable Neb Kit
MQ5900	Compact Compressor Nebulizer with Disposable and Reusable Neb Kit

TYPICAL PRODUCT PARAMETERS

Electrical Requirements	115 VAC, 60 Hz, Single Phase
Running Current:	1.5 A @ 14 psig @ 115 volts (approx.)
Sound Level:	50 to 63 dBA
Power Wattage:	75 watts @ 14 psig
Foam Intake Filter:	Foam
Safety Systems:	High temperature compressor shutdown
Width:	5.9 inches
Height:	3.8 inches
Depth:	5.7 inches
Regulatory Listing:	In compliance with UL 1431 Personal Hygiene and Healthcare Appliances.
Maximum Pressure:	33 psi
Maximum Flow Rate:	8.0 L/min
Operating Flow Rate:	4.0 L/min @ 16 psi operating pressure
Storage Conditions:	-13° F to 158° F (-25° C to 70° C); 10 to 95% RH
Operating Conditions:	50° F to 104°F (10° C to 40° C); 10 to 95% RH
Altitude:	0 to 6500 ft (2000 m) above sea level
Mode of Operation:	Intermittent Use (30 min. ON / 30 min. OFF)

NOTE: Specifications and appearance are subject to change without notice for purposes of improvement.

NOTE: Nebulization rate and particle size are dependent on the characteristics of the nebulizer chosen.

SUPPLEMENTAL INFORMATION

Compressor

If the unit shuts down frequently, you may have a low voltage situation.

Low voltage can also be suspected when:

- The motor does not get up to full power or speed.
- Home fuses or circuit breakers activate when starting compressor.
- Lights dim or remain dim when compressor is started.
- Other motor operated appliances fail to operate properly or too many motor operated appliances are on same circuit.

Extension Cords

Make certain extension cord is in good condition. Make certain extension cord wire is NOT LESS than 18 gauge for 25 feet.

HANDLING

Unpacking

1. Check for any obvious damage to the carton or its contents. If damage is evident, DO NOT use. Notify Drive or your local Dealer.
2. Remove all the loose packing from the carton.
3. Carefully remove all the components from the carton.

NOTE: Retain carton and packing materials for use in shipping of storing the unit until use is required.

Inspection

1. Examine exterior for nicks, dents, scratches or other damage. Inspect all components.

SECTION I – IMPORTANT SAFEGUARDS

WARNING

Section I – Important Safeguards contains this important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as Owner's Manuals, Service Manuals or Instruction Sheets supplied with this product or optional equipment. If you are unable to understand the Dangers, Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

Federal law restricts this device to sale by or on the order of a physician.

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

DANGER

Risk of electric shock. DO NOT disassemble. Refer servicing to qualified service personnel.

FIRE RISK - DO NOT use this product in an oxygen enriched environment (i.e. – oxygen tent).

TO REDUCE THE RISK OF ELECTROCUTION:

ALWAYS unplug this product IMMEDIATELY after using.

DO NOT use while bathing.

DO NOT place or store product where it can fall or be pulled into a tub or sink.

DO NOT reach for product that has fallen into water. Unplug immediately.

Do not place in or drop into water or other liquid.

WARNING

TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE AND INJURY TO PERSONS:

This product should NEVER be left unattended when plugged in. Close supervision is necessary when this product is used by, on, or near children and physically challenged individuals.

Use this product only for its intended use as described in this manual and only with the prescribed medication. Use this product only under physician's direction.

Drive products are specifically designed and manufactured for use in conjunction with Drive accessories. Accessories designed by other manufacturers have not been tested by Drive and are not recommended for use with Drive products.

NEVER operate this product with a damaged cord or plug, if it is not working properly, if it was dropped or damaged, or if it was dropped into water. Call qualified service personnel for examination and repair.

 WARNING

Keep the nebulizer tubing, cord, and unit away from heated or hot surfaces, including space heaters, electric blankets, stoves and similar electrical appliances.

NEVER drop or insert objects into any opening or hose.

NEVER use while sleeping or drowsy.

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening, may be clogged. Keep the air openings free from lint, hair and the link. Blocked openings may cause the unit to shut down. Contact dealer immediately.

The Drive compressor nebulizer system uses the surrounding air to effectively administer the prescribed medication through the nebulizer. The user and administrator should be aware of surroundings so as not to have contaminated air filtered through the compressor nebulizer system. (Examples can be air contaminated with paint cleaning fumes, weed killer, car exhaust, etc.)

DO NOT use the Drive compressor nebulizer system in these environments. Otherwise, serious bodily injury may occur.

This unit is oil-less. DO NOT lubricate.

 WARNING

This equipment is not suitable for use in the presence of a flammable anesthetic mixture with air or with nitrous oxide.

DO NOT operate the compressor while it is still in the carrying case.

When not administering treatment, keep out of the reach of small children. Some of the parts of the nebulizer are small enough for children to swallow. The tubing may present a strangulation hazard.

NEVER touch the unit with wet hands.

DO NOT pull on the power cord to unplug the unit from an electrical outlet.

DO NOT overload wall outlets.

Pump maintenance **MUST** be performed by a qualified service technician.

SAVE THESE INSTRUCTIONS

SECTION 2 – OPERATING INSTRUCTIONS

WARNING

Remove the compressor from the carrying case before use.

Do not operate unit without filter.

Avoid operating this product in a dusty environment.

Otherwise, premature compressor wear could occur.

NOTE: Clean and disinfect the nebulizer and mouthpiece before using the device for the first time. Refer to Cleaning and Disinfecting.

Setup

NOTE: For this procedure, refer to FIGURE 2.1

1. Place the unit on a table or other flat, stable surface. Be sure you can easily reach the controls when you are seated.

CAUTION

Make sure the power switch is in the off "O" position.

2. Remove the nebulizer kit from the carrying case.
3. Gently twist the nebulizer to separate it into two sections.
4. Be sure the pick-up funnel is in place in the bottom section of the nebulizer cup.

CAUTION

DO NOT add over 8ml of medication to the medication cup. The maximum capacity of the nebulizer medication cup is 8ml.

5. Add the prescribed amount of medication to the cup of the nebulizer using an eyedropper, pre-measured dose or ampoule (Figure 2.1).

6. Gently twist the top and bottom sections together to assemble the nebulizer. Be sure the two sections fit securely.

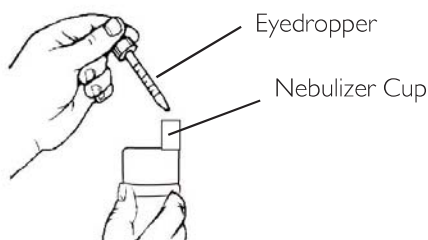
FIGURE 2.1 Adding Medication

7. Attach one end of the air tubing to the air tubing connector located on the front of the compressor.

8. Attach the other end of the air tubing to the base of the nebulizer.

WARNING

In the case of high humidity, condensation may form inside the internal tubing. Run the unit for two minutes before connecting the nebulizer to the tubing.

9. Attach the mouthpiece to the top section of the nebulizer.

NOTE: Before starting a cold unit, let the unit reach room temperature in a heated area.

10. With the power switch off, plug the power cord into an appropriate electrical outlet.

CAUTION

This device is for intermittent use: 30 min. ON/ 30 min. OFF. Failure to operate within these limits could damage the device beyond repair.

Treatment Instructions

NOTE: For this procedure, refer to FIGURE 2.2.

1. Turn the unit on, pressing the ON/OFF switch to the "-" position.

2. Place mouthpiece between your teeth close your mouth and tilt the nebulizer toward your mouth (Figure 2.2).

FIGURE 2.2

Using the Nebulizer

3. As aerosol begins to flow, inhale deeply, slowly, and hold breath for ten seconds. Then exhale slowly through the mouthpiece.

NOTE: If you need to rest during a treatment, place the nebulizer in the round holder located within the handle opening.

NOTE: Tighten caps on medication jars.

4. When treatment is complete, turn the unit off by setting the switch to the "O" position and unplug the unit from the electrical outlet.

5. Before storing supplies between treatments, clean and dry the nebulizer and accessories. Refer to Cleaning and Disinfecting instructions supplied with the nebulizer.

SECTION 3 – CLEANING AND DISINFECTING

Following the cleaning instructions after each use will prevent any remaining medication in the bottle from drying and causing the device to nebulize ineffectively.

Cleaning the Outer Case

WARNING

Unplug unit before cleaning.

DO NOT submerge in water to clean. Clean outer case with damp cloth.

DANGER

DO NOT remove outer case of this unit. Removing the case creates a serious electrical shock hazard. All disassembly and maintenance of this unit **MUST** be done by a qualified technician.

Cleaning the Compressor

CAUTION

Unplug unit before cleaning. **DO NOT** submerge in water to clean.

DO NOT clean using abrasive cleaners.

1. Clean the casing of the main unit daily by using a soft cloth moistened with water or a mild detergent.
2. Wipe the casing immediately dry using a soft clean cloth.

SECTION 4 – MAINTENANCE

DANGER

DO NOT remove outer case of this unit. Removing the case creates a serious electrical shock hazard. All disassembly and maintenance of this unit **MUST** be done by a qualified technician.

Routine Care

To keep your device in the best condition and protect the unit from damage, follow these directions:

- **DO NOT** subject the device or the parts to any strong shocks such as dropping the device on the floor.
- **DO NOT** store the device in extreme hot or cold temperatures, high humidity or under direct sunlight.

Storage

1. Put the nebulizer and mouthpiece in a carrying case.
2. Coil the air tubing.
3. Wrap the AC cord in a coil.
4. Zip the bag closed.
5. Store the device in a safe area.

***NOTE:** Store the repackaged device in a dry area. **DO NOT** place other objects on top of the repackaged device.*

Filter Change

1. The filter **MUST** be replaced after approximately 30 hours of use or when it turns grey.
2. Open the filter holder to replace the filter with a new one.

CAUTION

DO NOT use cotton or any other material.

DO NOT wash or clean the filter.

Use only Drive filters.

DO NOT operate unit without an air filter.

SECTION 5 – TROUBLESHOOTING

PROBLEM	SOLUTION
Device does not switch ON.	1. Make sure the plug is firmly fitted to the wall socket. 2. Try plugging into another wall socket that is known to work. 3. Contact your Drive representative.
Device does not nebulize, or it nebulizes weakly.	1. Make sure that the ends of the air tubing are fitted tightly onto the main unit and onto the nebulizer. 2. Check whether the nebulizer is empty of medication or has too much medication (MAX 8ml). 3. Check whether the nebulizer nozzle is obstructed. 4. Check whether the funnel is inserted. If these possible solutions DO NOT work, contact your medical dealer.

LIMITED WARRANTY

Your Drive brand product is warrantied to be free of defects in materials and workmanship for five years from the date of purchase for the original consumer purchaser. This device was built to exacting standards and carefully inspected prior to shipment.

This 5 year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as filters, rubber accessories, and rubber grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive Dealer.

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