

## two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Two Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel* *ite* *Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

## straddler bolster chair

item # SD 2000



EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP

*Wenzel* *ite* *Re/hab*

[www.drivemedical.com](http://www.drivemedical.com)

drive  
WWW.DRIVEMEDICAL.COM

The Straddler Bolster Chair was designed as a seating choice for individuals whose legs tend to adduct. It promotes sitting in an upright position, increases the hip to back angle and permits the pelvis to tilt anteriorly. It assists with sit to stand activities and encourages independence in getting on and off the chair.

Your Straddler Bolster Chair comes in one carton completely assembled.

---

### adjustments:

---

1. Seat angle can be adjusted by opening knob (A) on each side and positioning in proper groove.
2. Seat height can be adjusted by opening knob (B) under the front face plate.
3. Seat depth can be adjusted by opening knob under seat and sliding pelvic stabilizer to desired position.
4. Pelvic stabilizer can be removed if not required.
5. Pelvic stabilizer width can be adjusted by opening 4 knobs (C) and moving side panels to desired position. The side panels can be removed if desired.
6. Pelvic belt length can be adjusted by sliding belts through belt buckles.
7. Leg retainers (D) can be raised or lowered for additional lower leg positioning and alignment.

