

# move-X

Enjoy • Walking  
User Manual



## FOLDING PROCEDURE

### How to unfold:

1. Release the folding webbing button.
2. Hold the handle and pull up until a click sound to ensure it's good to be used.
3. Push the safety lock down to lock the folding handle.
4. Push the seat side tubes down until a click sound.
5. Adjust the handle to the proper position.
6. Pull out the bag holding rod.
7. Clip the bag on the bag holding rod and attach the Velcro on the seat Velcro loop.
8. Push down the castor lock to free the castor .

### How to fold:

1. Pull up the front castor lock to fix the direction of caster(better but not a must).
2. **Take off the shopping bag (better but not a must):** To grab the bag handle and take off the bag.
3. **Push in the bag holding rod (better but not a must):** When bag is taken off, push the holding rod inside.  
When bag is taken off, push the holding rod inside
4. Adjust the handle to the lowest position.
5. **Pull up the folding webbing :** The folding webbing is knitted at the left hand side of the seat. Pull it to fold the rollator horizontally to the Storage status.



6. Push the button of folding webbing on the seat tube button to fix the folding position.
7. Pull up the safety lock on outer side of the handles to release the safety lock.
8. **Pull the X folding handle:** Use your fingers to pull up the X-folding handle at each side of the rollator handle.
9. Fold down the rollator frame.



10. **After folding, use the buckle webbing to fix the rollator:** Put the webbing on the rivet each side to secure the folded position



## FEATURES


### Brake

Pull up the brake lever to activate running brake.  
Push down the brake lever to activate parking brake.



### Handle height adjustment

1. Pull up the fixation lock to release.
2. Pull the fixation knob and turn 90 degree.
3. Adjust the handle to proper position.
4. Turn the fixation knob 90 degree and let go the knob to fixation position and a click sound. (If no click sound, fine tune the handle for a click sound and it shows the handle is fixed at proper position )
5. Try the handle to ensure it doesn't move
6. Push back the fixation lock to tighten the handle.

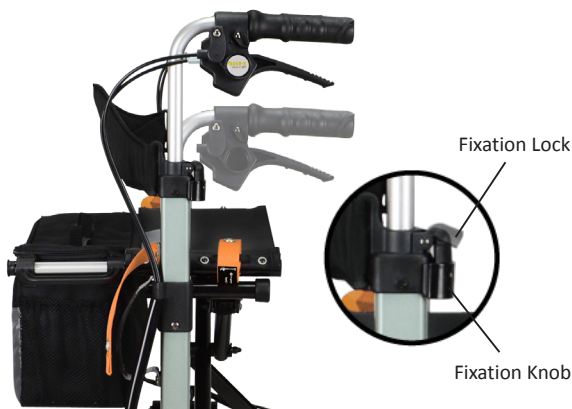

### Folding mechanism

Pull to disengage the rollator frame to start folding.

### The safety lock

Up position: prepare to fold the rollator

Down position: After unfolded, push down the safety lock to secure the folding handle



### Reflector



### Cane & Umbrella Holder




### Installation of Basket

1. Pull out both tubes attached to the seat and hook the basket up.
2. Hear the click sound to ensure the basket is secured onto the tube.
3. Attach the Velcro on the seat Velcro loop.



## SPECIFICATION

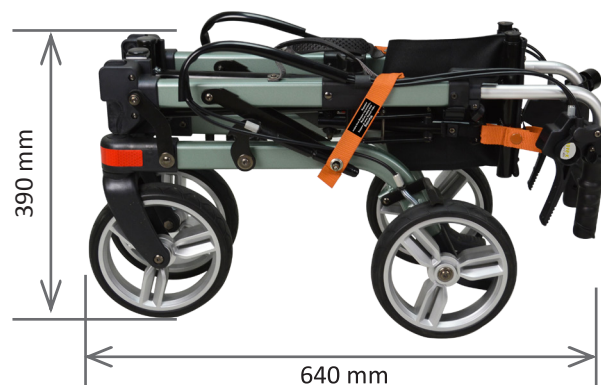

### Osann Move X

Test according to ISO11199-2

Weight Capacity : 125kg

Bag Capacity: 5kg

Overall Dimension:

L580 X W640 X H775~895mm

Folded Dimension:

L640 X W280 X H390mm

Seat Dimension:

W425 X D190 X H580mm

Backrest:L365 X W255mm







## **CLEANING METHOD**

1. Wipe down your equipment weekly (or more often as needed) with a cloth dampened in a mild detergent/ water solution.
2. Allow the equipment to dry before using or storing.
3. Store your equipment safely away in a clean, dry location when not in use.

## **WARNINGS**

### **FOR PERSONAL SAFETY**

- For your health and comfort, be sure to use good posture and hold the rollator so that you can operate the brakes with ease.
- Please store the rollator in a safe environment.
- Unfold the rollator completely into the locked position before use.
- Possible risk of finger trapping - do not put fingers or hands into the hinge of the rollator when folding or opening for use.
- Ensure that the wheels are stable and on the ground before using.
- Ensure the brakes are adjusted properly and tested before using.
- Ensure that the seat is locked in position before sitting.
- Always sit in the center of the seat, with the seat adjusted that the user has their feet on the floor.
- Always have the parking brakes locked in position before sitting.
- Never sit on the rollator when it is folded for storage.
- Never stand on the rollator, or use it as a step.
- Never transport a person on the seat of the rollator.
- Use caution when lifting and carrying the rollator.

## **WARRANTY**

There is a comprehensive twelve-month warranty from the date of delivery of your new rollator. the warranty covers the rollator for repairs and replacement during this period. More detailed warranty information as follows:

1. Any work or replacement part installation must be carried out by an authorized dealer or service agent.
2. Warranty claims must be placed at the outlet from which the rollator was purchased.
3. Any parts of the rollator that require repair or replacement as a result of manufacturing or material defect within the warranty period will be supplied free of charge.
4. Any repaired or replaced parts will be covered by the balance of the warranty period on the rollator.
5. Under normal circumstances, warranty claims will not be accepted where the following situations occur:
  - a) The rollator has not been maintained in accordance with the manufacturer's recommendation.
  - b) Failure to use the manufacturer's specified parts.
  - c) The rollator is damaged due to neglect, accident, or improper use.
  - d) The rollator or parts having been altered from the manufacturer's specification or repairs having been attempted before the dealer is notified.

The manufacturer reserves the right to alter without notice any weights, measurements, or other technical data shown in this manual. All figures, measurements, and capacities shown here are approximate and do not constitute exact specifications. All rights reserved.

