

INSTRUCTIONS FOR INSTALLATION AND USE

BEFORE YOU BEGIN:

1. Maximum weight limit is 850 pounds.
2. Read and follow all instructions before use. Do not use if ramp is damaged or unstable.
3. For use with wheelchairs and scooters only. Ramp is intended for residential use.
4. **⚠ WARNING:** Use ramp only with a qualified helper and always use your lap belt.
5. **⚠ WARNING:** It is important that you refer to your equipment's (wheelchair or scooter) Owner Guide for the proper degree of incline/decline and chair direction before using ramp. Never exceed its recommendations.

FITTING A RUBBER THRESHOLD RAMP:

1. Ramp can be easily trimmed to accommodate lower heights and narrower widths and, if necessary, notched to suit the doorjamb configuration.
2. If door threshold is equal to the height of the ramp, use Threshold Ramp as is.
3. If door threshold is less than the height of the ramp, trim ramp to the desired height as shown in DIAGRAM 1 (serrations in the bottom of the ramp can be used as a guide). **⚠ WARNING:** Use caution when operating any non-manual saw as serious injury may result due to unexpected binding or pinching of rubber ramp material. Always follow tool manufacturer's safety guidelines.

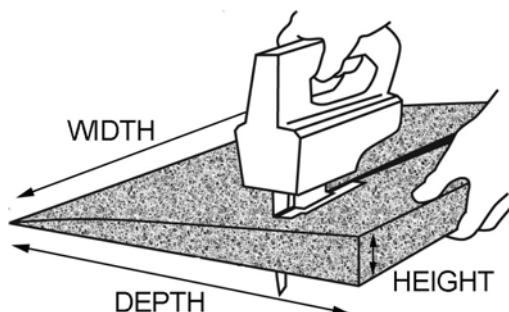


DIAGRAM 1

SECURING THE RAMP: It is recommended that the ramp be securely attached.

1. ATTACHING TO WOOD:

- a. Place ramp in desired location and ensure that surface is free of dirt and debris.
- b. Secure ramp using supplied Phillips head screws (two each). Space screws evenly (see DIAGRAM 2 for placement) and drive until head is ¼" below the ramp surface.

2. ATTACHING TO CONCRETE:

- a. Place ramp in desired location and ensure that surface is free of dirt and debris.
- b. Using a ¼" diameter masonry bit, drill a hole 2" deep 1" from the edges of the corners at the taper as shown in DIAGRAM 2. **NOTE:** Ensure the holes are free of any residue.
- c. Drive the supplied ¼" diameter mushroom head pin anchors (two each) into the holes with a hammer until the head is seated.

CLEANING AND MAINTENANCE:

1. At all times, keep ramp clear of snow, ice, dirt, leaves, and other debris that may accumulate on the surface.
2. Clean ramp with mild soap and warm water. Rinse well and use extra caution when ramp surface is wet.

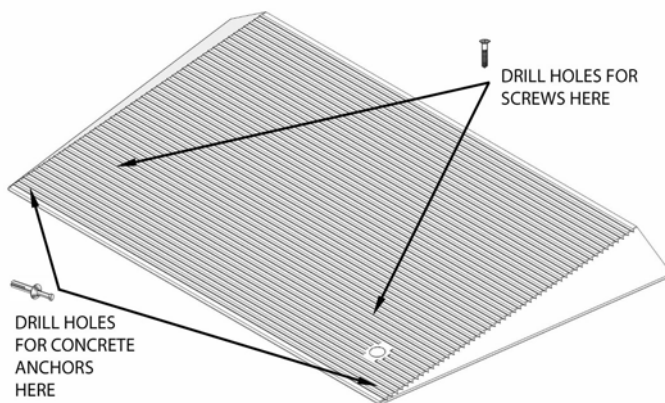


DIAGRAM 2

