

330 Preferred Care Recliner

Operating Instructions



330-INS-LAB

© Graham-Field Health Products 2005

**IMPORTANT: READ THIS MANUAL BEFORE OPERATING
YOUR ORTHO-BIOTIC PREFERRED CARE RECLINER**

WARNINGS

- ⚠ **Warning: Read entire sheet before assembly or operation.**
- ⚠ **Warning: Occupant weight should not exceed 250 lbs.**
- ⚠ **Warning: Periodically inspect all parts and replace as needed.**
- ⚠ **Warning: Casters must be locked at all times, except during transit.**
- ⚠ **Warning: Do not use near open flame, as the chair may be a “combustion source”.**
- ⚠ **Warning: Never stand on chair or transport anything other than a person.**

CARE & MAINTENANCE

Every three months:

- Check and clean casters • tighten nuts • check brakes on casters •
- check and clean lock recline mechanism •

Check recliner mechanism for operation in all positions with assistant seated in recliner.

For upholstery maintenance and stain removal, the following hints will be helpful:

Ordinary Dirt: Wash with warm water and a mild soap or detergent.

Stubborn Dirt: Using a soft bristle brush, follow directions above.

Ground-in Dirt: Scrub with powdered cleanser or similar detergent.

Chewing Gum: Scrape carefully; remove with kerosene or naphtha.

Nail Polish and Remover: Blot immediately to prevent damage.

Ball Point Pen: Blot immediately with a white cloth dampened in water or mineral oil.

Paint, Shoe Polish, Heel Marks: Remove immediately! Use a white cloth dampened in kerosene, naphtha or turpentine. Do not use paint remover or liquid brush cleaners.

Tars, Asphalt, Creosote: Remove to prevent staining. Clean area with kerosene or naphtha.
Caution: When using kerosene or naphtha, do so in a well ventilated area and keep away from fire and flame.

Waxing and Refinishing: Improver wearability. Use any quality paste wax.

A diluted solution of 10% (or less) of bleach or Clorox may be used when cleaning this chair.

RESIDENTIAL RECLINER

The recliner has an interior & exterior frame design. The interior frame is welded steel (7/8" square tubing) on which the recliner hardware is mounted to. The exterior frame is made of a poplar (7/8" x 2 1/2") construction that is mounted to the outside of the Interior frame, with exposed upholstery sides. All vinyl material is made of a sulfide stain resistant naugahyde fabric. Foam & fabric material are flame retardant.

The recliner main components are: an upholstered legrest and an upholstered back with a single pillow for easy cleaning. The back is KD for shipping and replacement. The upholstered seat is made up of 4" medium density foam cushion with straps that are attached to the seat deck for easy cleaning & removability. Both back frame and seat deck are made with flexolator and spring suspension for better seating.

All wood components are made of 7/8" thick poplar.

The recliner is available with four casters (2

locking, 2 non-locking). The height of the casters are 4" from the floor. which allows most overbed tables to be used.

This recliner will have a push handle.

Transfer arms with push button release have provision for locking table. Arms are fully padded on top.

A table top is being offered which is a locking adjustable design/high pressure laminate and "T" molding, and can be stored behind the back of the chair.

Recliner hardware provides three positions, First - SITTING, Second - TV, Third - FULL RECLINE (for sleeping).

Modified trendelenburg position.

KD back, one piece back with built in lumbar.

KD "loose pillow look" high styled, triple rolled back.

DIMENSIONS & SPECIFICATIONS

Back height from floor42"
Back height from seat26"
Overall width31"
Seat width21 1/2"
Seat height19"
Seat depth20"

Arm height24"
Overall depth38-62"
Caster width front center to center .24 3/4"
Front to rear caster center to center ...30"
Weight (uncartoned)99 lbs.
Cartoned weight109 lbs.

OPTIONS & ACCESSORIES

Model #	Description
3310	Table
3311	Storage Pouch

ASSEMBLY INSTRUCTIONS

- 1 Remove recliner back and base unit from cotton. Remove thumb screws from two recliner mechanism uprights.
- 2 Open velcro panel at lower half of the chair back (Fig 1).
- 3 Locate brackets on each side of backrest and slide backrest (in vertical movement) onto the two recliner uprights. Be sure the uprights are fully inserted into the brackets. Replace the thumb screws onto the recliner uprights to secure the backrest onto the mechanism.
- 4 Tightly close velcro panel.
- 5 Chair is now ready for use.



Figure 1



Figure 2

Armrest

1. To raise armrest lift up on central portion of armrest until locking pin on armrest locks.
2. To lower armrest, lift slightly on arm to disengage pull pin. Push armrest down while pin is pulled out.



Figure 3

Recliner

1. The standard model of this recliner has three positions that can be achieved by leaning back in the unit while pushing back on the armrest. **IMPORTANT:** Do not push chair into reclining positions when chair is unoccupied or recliner mechanism may be damaged.
2. To return unit to upright sitting position, pull forward on armrest and push down on foot rest as you decrease your pressure on the recliner's back.



Figure 4

If you ordered a recliner with modified Trendelenburg position:

3. The modified Trendelenburg position maybe obtained by reclining one step past the third position. (See Fig 4). As a safety feature, you must hold chair in the modified Trendelenburg position. This is so that patients will not accidentally go to this position and get stuck.

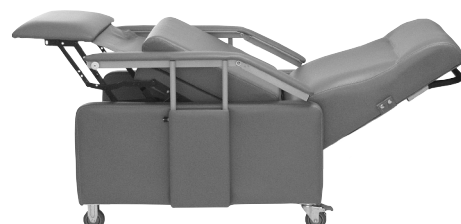


Figure 5

U.S.A., Corporate Headquarters:

Graham-Field Health Products

2935 Northeast Parkway
Atlanta, Georgia 30360
telephone: 800-347-5678, 770-447-1609
fax: 800-726-0601, 678-291-3232



<http://www.grahamfield.com>