



# Model 2210CART

---

Neo-natal Scale Cart  
For 2210KL

INSTRUCTION MANUAL

## Introduction

The 2210CART cart is designed for use with your 2210KL Neo-natal Scale.

Included in this kit are:

- (1) Top shelf with handle
- (1) Middle shelf with 8 leg posts
- (1) Bottom shelf with caster inserts
- (4) Fluted legs (16-1/2") for cabinet
- (4) Square legs (8") for top shelf
- (1) Cabinet back panel
- (2) Cabinet side panels
- (1) Cabinet door panel
- (2) Cabinet door keys
- (2) 4" swivel casters with brakes
- (2) 4" swivel casters

You will need a rubber mallet.  
(Optional: metal hammer with a block of wood to cushion impact.)



## Assembly Instructions

1. Locate the bottom shelf. Place upside down (ribbed surface up) on a level and firm floor surface. Insert the stems of the (2) casters with brakes on the left short side of the shelf. Insert the other (2) casters on the right short side. Tap casters into socket with rubber mallet until fully seated.

Casters with brakes




2. Turn cart right side up. Place the four fluted legs over the corner posts so that the channels are facing each other. Use a rubber mallet to tap legs in place. Make sure that legs are even and fully seated.



3. Locate back panel and slide into grooves of two back legs.

Casters with brakes

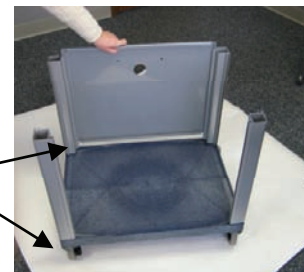

4. Take the two side panels and slide each into place.



5. Take front panel and slide into place. The lock should be positioned on the left as shown.



Casters with  
brakes


6. Take the middle shelf and place ribbed side down onto legs so that the posts are fitting into the holes in the top of the legs. Push shelf down as far as you can by hand. Gently tap with mallet if necessary.



7. Place the four short square legs over the corner posts on the middle shelf. Tap gently with mallet until fully seated.



8. Take the top shelf with the handle to the left (over the casters with brakes) as shown and place onto legs so that the posts are fitting into the holes in the top of the legs. Push shelf down as far as you can by hand. Gently tap with mallet if necessary.

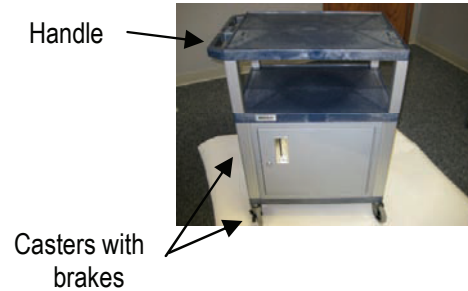


Handle

Casters with  
brakes

9. Position scale on top shelf so that the white scale platform is all the way towards the front and right safety lip. This provides maximum access to the shelf handle and assures that the clear cradle does not touch any wall surface which will affect accurate weight measurement.



10. The cabinet may be used with or without the keys. The door has a positive latching action when used without the key.

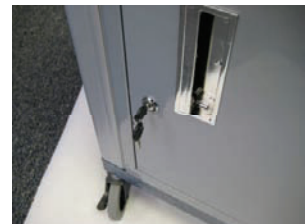

## Safety

Never use the cart to transport the infant.  
Use only on firm and level flooring.

## Cleaning

Clean the cart as required using soap and water or a commercially available disinfectant. Follow the instructions from the cleaner or disinfectant manufacturer.

## Specifications

Size: 32"w x 24"d x 34"h  
Weight Capacity: 200 pounds

## Need Help?

If you have any questions on the assembly or use of this cart, please contact customer service at 800-638-3722.



Pelstar LLC  
11800 South Austin Avenue Unit B.  
Alsip, IL 60803

Phone: 708-377-0600  
Toll free: 800-815-6615  
Fax: 708-377-0601  
[www.homscales.com](http://www.homscales.com)

Health o meter® is a registered trademark of  
Sunbeam Products Inc., Boca Raton, FL 33431