Reduced Gap Half-Length Bed Rail

Assembly, Installation and Operating Instructions

Model Nos. 6630

Yes, you can.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the dealer/carrier for further instructions.

To ensure the safe use of the Reduced Gap Half Length Bed Rail, these instructions **MUST** be followed:

⚠ WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, death, injury or property damage may occur.

Invacare recommends the use of two sets of half length bed rails: one set at the head end of the bed, and one set at the foot end of the bed where permitted.

These bed rails are intended to prevent an individual from inadvertently rolling out of bed. **DO NOT** use for restraint purposes.

ENTRAPMENT MAY OCCUR. Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions, and mattress thickness, size or density could increase the risk of entrapment. Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Review **A Guide to Bed Safety**, published by the Hospital Bed Safety Workgroup, located at www.invacare.com. Use the link located under each bed rail product entry to access this bed safety guide.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Although bed rails are not rated to any specific weight limitation, the bed rails may become deformed or broken if excessive side pressure is exerted on the bed rails. This bed rail is not an assist rail for getting into or out of bed. **DO NOT** use the bed rails as push handles when moving the bed.

After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely. Bed rails with dimensions different from the original equipment supplied or specified by the bed manufacturer may not be interchangeable and may result in entrapment or other injury.

To reduce the risk of entrapment make certain that the crossbraces **DO NOT** exceed the width of the mattress. **Mattress MUST** fit bed frame and bed rails snugly to reduce the risk of entrapment.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

⚠ WARNINGS (CONTINUED)

Unless the bed rail is in the lowest position, ensure that the spring loaded pins fully protrude through the same respective adjustment hole of each bed rail tube. This ensures that the bed rail is securely locked in position and an even height adjustment is achieved. (The lowest position for the bed rail occurs when the bed rail rests against the bed rail stop. Spring loaded pins **DO NOT engage into an adjustment hole when the bed rail in the lowest position.)**

Installing the Crossbraces

READ AND UNDERSTAND WARNINGS before proceeding with procedure.

NOTE: For this procedure, refer to FIGURE 1 on page 3.

Tools required are 13 mm socket wrench.

1. Remove the mattress from the bed.
2. Place the bed in the flat position.
3. Bolt the crossbraces to the half rails (Detail "A" in FIGURE 1).
4. Position the crossbraces in the following manner (Detail "B" in FIGURE 1):
 - A. Head End - The crossbrace should be approximately 11 inches (28 cm) from the inside edge of the bed frame.
 - B. Foot End - The crossbrace should be approximately 11 inches (28 cm) from the inside edge of the bed frame.


NOTE: Avoid the bed springs when installing the bed rails. If the 11-inch dimension, stated above, falls in alignment with a bed spring then the bed rail needs to be mounted on either side of the bed spring.

5. Loosen the wingnut under the crossbrace.
6. Slide the long side of the clamp under the bed frame, making sure the bent portion of the clamp is positioned towards the inside of the bed (Detail "C" in FIGURE 1).
7. Tighten the wingnut to secure the clamp to the bed frame.
8. Repeat STEPS 3-7 for the remaining three wingnuts and clamps.

Using the Bed Rails

NOTE: For this procedure, refer to FIGURE 1 on page 3 .

1. Ensure the bed rails are flush against the bed frame.
2. While holding bed rail with one hand, pull spring-loaded knob with other hand to lower bed rail (Detail "B" in FIGURE 1).
3. Pull bed rail up to lock into position.
4. Periodically check and retighten mounting hardware.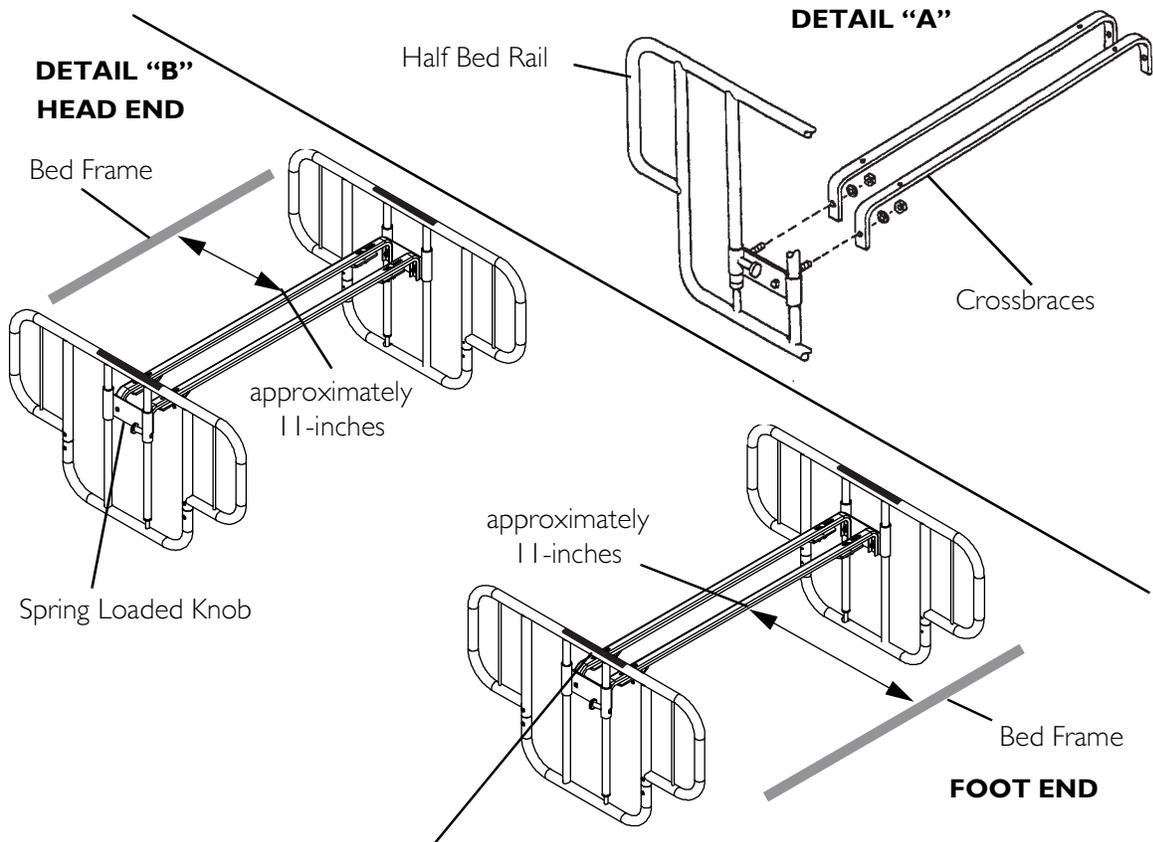

⚠ WARNING
 Patient entrapment with bed side rails may cause injury or death. Mattress must fit bed frame and side rails snugly to prevent patient entrapment. Follow the manufacturer's instructions. Monitor patient frequently. Read and understand the Owner's/Operator's Manual prior to using this equipment. Manual is available at www.invacare.com or your dealer.
PN 110103 Rev A-1104

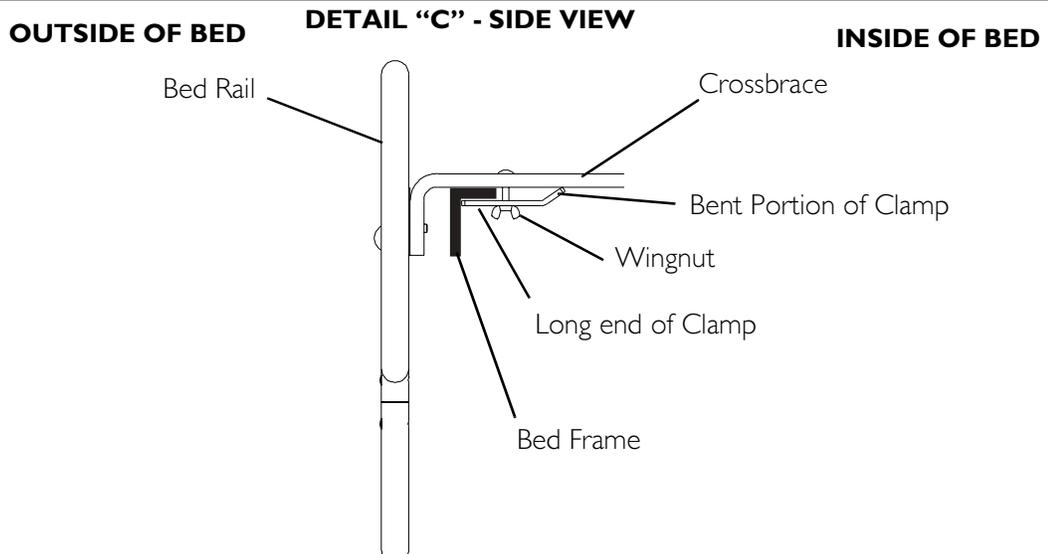
FIGURE I Installing the Crossbraces/Using the Bed Rails

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for two years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS: PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE. THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2006 Invacare Corporation
Design are registered trademarks of