User Manual

DEALER: This manual MUST be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

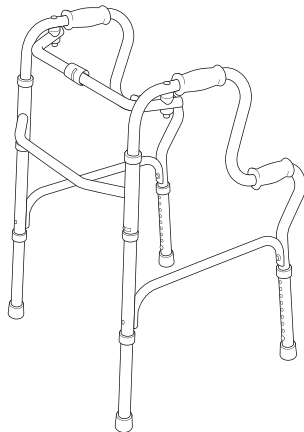
Yes, you can.

Two Step Walker Model P421B

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I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

Gives useful tips, recommendations and information for efficient, trouble-free use.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

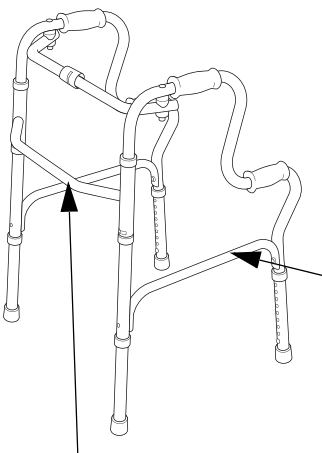
This product complies with Directive 93/42/EEC concerning medical devices.

I.2 Life

The expected service life of the product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. Effective service life can vary according to frequency and intensity of use. Maintenance and inspection should be done at least every six months.

I.3 Product Labeling

Yes, you can.

 = 130 kg
 286 lbs

Model / REF
P421B

LOT
GTYYMMDD

INVACARE REA AB
BOX 200
SE 343 75 DÖ, SWEDEN
DISTRIBUTED BY
INVACARE
CORPORATION
ELYRIA, OHIO 44035
USA
MADE IN CHINA

Product Weight Limitation

Product Model Number

Supplier and Date Code

EU Representative

WARNING
BEFORE use, both sides of the walker
MUST be open and in locked position.

I.4 User Height Specifications and Weight Limitation

WARNING

ALWAYS observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.

USER HEIGHT MIN. / MAX	WEIGHT LIMITATION
5'3" - 6'4" (160-193 cm)	286 lb (130 kg)

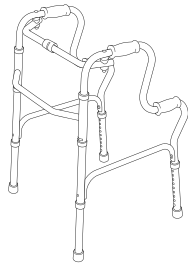
2 Safety

 Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/ dealer for further instruction.

2.1 Intended Use

The walker is intended to be used as a walking aid or toilet safety frame and all other use can cause risk of injury. The walker can be used inside and outside the house.

Use as a Walking Aid

Use as a Toilet Safety Frame

2.2 General Warnings

WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

WARNING

You should ALWAYS consult with your physician or therapist to determine proper adjustment and usage.

ALWAYS use caution when using the walker, especially on wet or icy surfaces.

If the walker is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips DO NOT twist on side frame otherwise damage or injury may occur. The handgrips MUST be tight.

Ensure that the rubber tips are not ripped, worn or missing. Replace the tips as necessary before using walker or toilet safety frame. DO NOT use plastic glide tips or wheeled leg extensions with this product.

Rear leg extensions with rubber tips MUST be in contact with floor surface at all times. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one inch lower than the front.

Make sure that the user's weight is distributed evenly and directly over the walker or toilet safety frame.

The walker is not intended to support the full weight of the user.

The toilet safety frame is designed to provide support, increase stability and assist the end-user.

The toilet safety frame is NOT designed to support the total weight of an individual.

Check that all labels are present and legible. Replace if necessary.

DO NOT hang anything on either the left or right side frames (area that includes the release paddles) or the left and right side braces. This will cause side frames side braces to bend and prevent the walker or toilet safety frame from locking when opened. This may also cause the walker or toilet safety frame to tip or be unstable, resulting in injury or damage.

DO NOT sit on the crossbar. DO NOT use the crossbar as support.

Exercise caution when walking on sloping surfaces.

DO NOT use the walker to go up or down stairs.

DO NOT use the walker on an escalator.

DO NOT leave or store the walker outside.

Users with limited physical capabilities should be supervised or assisted when using the walker as a toilet safety frame.

DO NOT rest the toilet safety frame on the toilet.

When using the toilet safety frame, the bathroom floor must be dry and free of debris to avoid injury due to slipping on the floor.

3 Usage

3.1 Opening/Folding the Walker

WARNING

Pinch points exist between the frame tubes when opening and folding the walker. Make sure hands and body are clear of all pinch points.

1. Swing both side frames (A) out away from the front crossbrace (B) until an audible "click" is heard.

WARNING

BEFORE use, both sides of the walker **MUST** be open and in locked position.

2. Test the walker by grasping the side frames (A) and, with some force, try to fold them inward.
3. To fold the walker, press down on the locking knob (C) and fold the side frames (A) in toward the front crossbrace (B).

3.2 Installing/Adjusting Leg Extensions

WARNING

Rear leg extensions with rubber tips **MUST** be in contact with floor surface at all times. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one inch lower than the front.

Pinch points exist when installing and adjusting leg extensions. Make sure hands and body are clear of all pinch points.

DO NOT use plastic glide tips or wheeled leg extensions with this product.

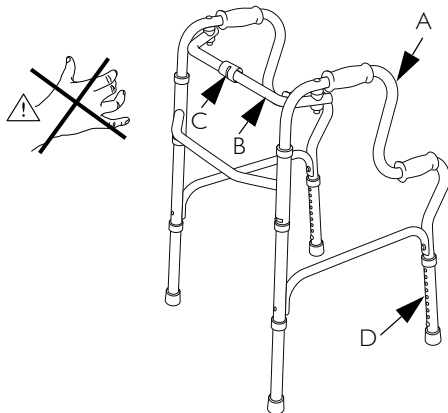
1. Press the snap button and slide the leg extension (D) onto the walker leg.

2. Adjust the leg extension to the desired height and allow the snap button to snap into the height adjustment hole.


An audible "click" will be heard.


3. Repeat STEPS 1 and 2 as necessary.
4. Make sure that the walker is level.

DETAIL "B" - LEG EXTENSION SNAP BUTTON

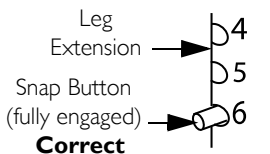Snap Button
(not engaged)
Incorrect

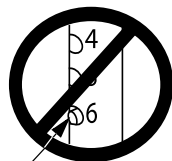4 Care and Maintenance

Maintenance should be done at least every six months.

DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the walker.

1. Regular cleaning of the walker is recommended. Wipe all surfaces with a damp cloth and allow all surfaces to dry before use.
2. Regular inspection of parts including hardware, brackets, and plastics, for deformation, corrosion, breakage, wear or compression is recommended. Replace the walker if any of these conditions exist.
3. Ensure the handgrips DO NOT twist on side frame. The handgrips MUST be tight. DO NOT use the walker if the handgrips are loose. Replace the walker if the handgrips are loose on the side frame(s).
4. If rubber tips are worn or cracked, replace the rubber tips IMMEDIATELY.

5 After Use

5.1 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Reuse

WARNING: Risk of Injury

Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, DO NOT use the product. Otherwise injury may occur.

6 Limited Warranty - North America

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for five years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE. THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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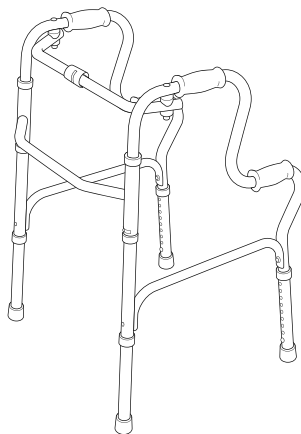
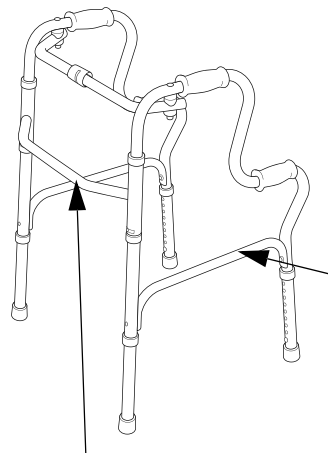
Model / REF
P421B

LOT
GTYYMMDD

INVACARE REA AB
BOX 200
SE 343 75 DIÖ, SWEDEN
DISTRIBUTED BY
INVACARE
CORPORATION
ELYRIA, OHIO 44035
USA
MADE IN CHINA

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© 2010 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare Rea AB